

State of North Dakota)
)ss
County of Burleigh)

I, Charles E. Eder, do hereby certify that I am a duly-appointed State Toxicologist for the State of North Dakota and an official custodian of the records and files of the office thereof, that I have carefully compared the

MEMO TO EMERGENCY ROOM SUPERVISORS AND PERSONNEL (SEPTEMBER 29, 2011)

hereto attached with the respective original as the same appears of record on file in the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, and find the same to be a true and correct copy thereof and of the whole thereof. In witness whereof I have set my hand at the city of Bismarck, in said county this:

29th day of SEPTEMBER, 2011

Charles E. Eder
Charles E. Eder, State Toxicologist


State of North Dakota)
)ss
County of Burleigh)

On this 29 day of September, 2011, before me personally appeared Charles E. Eder, known to me to be the State Toxicologist for the State of North Dakota, and acknowledged to me that he has executed the same.

Subscribed to and sworn before me on this:

29 day of September, 2011

Stacie L Fleck-Merkel
Stacie L. Fleck-Merkel
Notary Public, State of North Dakota
My Commission Expires August 16, 2017

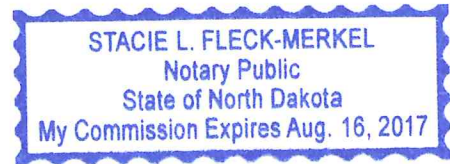

Notary seal/stamp



OFFICE OF ATTORNEY GENERAL
Crime Laboratory Division
2641 East Main Avenue (58501)
P.O. Box 937
Bismarck, ND 58502-0937

Tel. (701) 328-6159
(800) 296-2054
Fax (701) 328-6185

MEMO TO: Emergency Room Supervisors & Personnel
FROM: Charles E. Eder, State Toxicologist
DATE: September 29, 2011
REGARDING: Blood Alcohol Collection Kits

Charles E. Eder
29 SEP 11

Since the inception of this office in 1961, the State Toxicology Laboratory has been supplying the blood collection kits used for alcohol testing. The present blood collection kits contain most of the components needed for withdrawing blood. On occasion, you may need to use a substitute for an item in the kit. If a substitute component is used, that information should be documented on the Submission for Blood (104) found inside the kit boxes. Listed below are the kit components and their acceptable alternatives:

KIT COMPONENTS

- 1) Submission for Blood (104)
- 2) Vacutainer or blood tube
- 3) Needle
- 4) Disinfectant*

- 5) Guide (vacutainer or blood tube holder)
- 6) Specimen label
- 7) Absorbent pad
- 8) Plastic bag
- 9) Return address label or shipping seal

ACCEPTABLE ALTERNATIVES

- From another kit of the same Kit Lot No.
From another kit of the same Kit Lot No.
From your supplies (any sterile needle and/or syringe)
From another kit of the same Kit Lot No. or germicidal soap, water, or hydrogen peroxide
From your supplies
From your supplies or another kit
From your supplies or another kit
From your supplies or another kit
From your supplies or another kit

These kits are assembled and sealed by an approved supplier and distributed by the Office of Attorney General, Crime Laboratory Division. An expiration date is marked on the kits. Do not use the kit after this date. Using the kit after its expiration date will not affect the alcohol test, but it could result in insufficient blood being collected. Please note that either the specimen collector and/or specimen submitter should verify that the kit was intact before use and make checks appropriately on the Submission for Blood (104).

A blood sample obtained according to the instructions listed on the form and utilizing a blood collection kit supplied by the Lab will be considered properly obtained.

This memorandum supersedes the memoranda of July 2, 1992, March 29, 1993, May 2, 1994, September 27, 1999, September 1, 2000, June 15, 2003, January 2, 2004 and December 10, 2010.

*Disinfectant products containing 0.02 percent or less ethyl alcohol will be considered non-alcoholic for this purpose.