

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004951

Location: TOX LAB

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5190 Lot No. ✓ Rep AC 0.019 AC ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5113 Lot No. — Rep AC 0.285 AC ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. — Rep AC 0.10 AC + 0.05% Acetone ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 709352 Cyl No. 36 Exp. Date 23 Dec 2011

Test 1 0.079 Test 4 0.079 Test 7 0.079

Test 2 0.078 Test 5 0.079 Test 8 0.079

Test 3 0.078 Test 6 0.078 Test 9 0.079

Average 0.079

L. Remarks/Maintenance record: Annual Inspection

12 March 2010
Date

Mary Beum
Field Inspector's Signature

15 Mar 2010
Date

Deb Kashner
Reviewed by

80-004951
23/31 152

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 11:23

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 123456780
Standard Cyl #? 1234567
Standard Expiration? 11/11/2011
Oper No? 107501

Flow Cal. Date: 01/19/2010
Slope 698
Intercept -658617

IR Calibration Date: 01/19/2010
 3um 9um

	3um	9um
0th Coef(*100):	-24390	-26496
1st Coef(*100):	258099	132326
2nd Coef(*100):	1745	1171
H2O adj(mg/l*10k):	509	459

***** Printer Test End *****

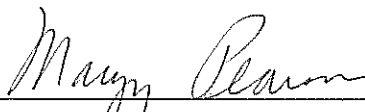

Operator Signature
MARGY PEARSON

Remarks:

Print

Form 106-I8000

80-004951
2/3/11 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 13:55

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:56
02 Std. Sol.	0.016	13:57
03 Room Air	0.000	13:58
04 Std. Sol.	0.017	13:58
05 Room Air	0.000	13:59
06 Std. Sol.	0.017	14:00
07 Room Air	0.000	14:00


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 08

Oper No. = 107501

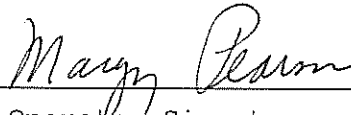

Operator Signature
MARGY PEARSON

Remarks:

LOW AC 0.019AC

Form 106-I8000

80-004951
25/31 15M

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 14:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:15
02 Std. Sol.	0.286	14:17
03 Room Air	0.000	14:17
04 Std. Sol.	0.288	14:18
05 Room Air	0.000	14:19
06 Std. Sol.	0.289	14:19
07 Room Air	0.000	14:20


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 0000

County = 08

Oper No. = 107501

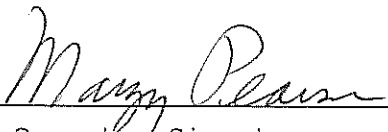
Operator Signature
MARGY PEARSON

Remarks:

LINEARITY 0.285 AC

Form 106-I8000

80-004951
26/31 YPL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 15:05

Test	AC	Time
01 Room Air	0.000	15:05
02 *Subject Test	INT*	15:06
03 Room Air	0.000	15:06

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 709352
Cyl No = 36
Expiration Date = 12/23/2011
County = 08 Oper No. = 107501

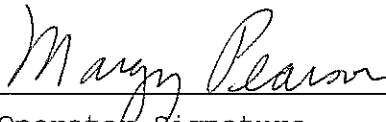

Operator Signature
MARGY PEARSON

Remarks:

Interfer Alc + Acetone

Form 106-I8000

80-004951
27/31 YAL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 14:54

Test	AC	Time
01 Room Air	RFI*	14:54
02 Room Air	0.000	14:55

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 11/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 709352
Cyl No = 36
Expiration Date = 12/23/2011
County = 08 Oper No. = 107501

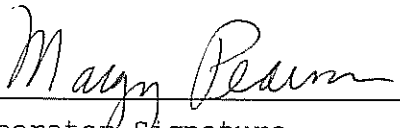

Operator Signature
MARGY PEARSON

Remarks:

RFI

Form 106-I8000

80-004951
28/31 HZ


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 14:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:25
02 Std. Gas	0.079	14:26
03 Room Air	0.000	14:26
04 Std. Gas	0.078	14:26
05 Room Air	0.000	14:27
06 Std. Gas	0.078	14:27
07 Room Air	0.000	14:28

Lot No = 709352
Cyl No = 36
Exp Date = 12/23/2011
County = 08 Oper No. = 107501

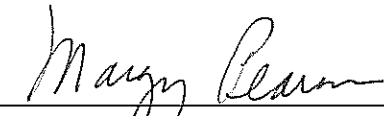

Operator Signature
MARGY PEARSON

Remarks:

ACA Cal Check

Form 106-I8000

80-004951
29/31 PL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 14:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Gas	0.079	14:32
03 Room Air	0.000	14:32
04 Std. Gas	0.079	14:33
05 Room Air	0.000	14:33
06 Std. Gas	0.078	14:34
07 Room Air	0.000	14:34

Lot No = 709352
Cyl No = 36
Exp Date = 12/23/2011
County = 08 Oper No. = 107501

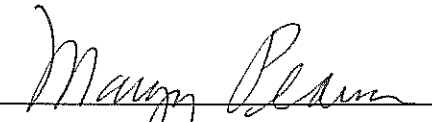
Operator Signature
MARGY PEARSON

Remarks:

Cal Check A/C

Form 106-I8000

80-004951
39/31 FAL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = TOXL 8164.13.00 06/09
03/12/2010 14:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Gas	0.079	14:46
03 Room Air	0.000	14:46
04 Std. Gas	0.079	14:47
05 Room Air	0.000	14:47
06 Std. Gas	0.079	14:47
07 Room Air	0.000	14:48

Lot No = 709352
Cyl No = 36
Exp Date = 12/23/2011
County = 08 Oper No. = 107501

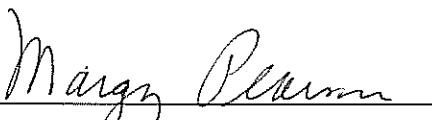

Operator Signature
MARGY PEARSON

Remarks:

Cal Check AC A

Form 106-I8000

80-004951
3/31/12