

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006491

Location: TOXL

Check When Done

- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5114 Lot No. 201505B Rep AC 0.030 ✓
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016

Test 1 0.079 Test 4 0.081 Test 7 0.080

Test 2 0.079 Test 5 0.080 Test 8 0.081

Test 3 0.080 Test 6 0.080 Test 9 0.080

Average 0.080

J. Tank pressure, Level 3 Func D Sub G

P: 418 psi Reg <sup>~450</sup>475 CEE psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4.12.2016  
Date

Charles E. ...  
Field Inspector's Signature

12 Apr 2016  
Date

Deb Shanaver  
Reviewed by



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006491  
Location = TOXL      8164.13.00 06/09  
04/12/2016      16:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:33
02 Std. Sol.	0.028	16:34
03 Room Air	0.000	16:35
04 Std. Sol.	0.028	16:36
05 Room Air	0.000	16:36
06 Std. Sol.	0.028	16:37
07 Room Air	0.000	16:38

08 Sim Temp = 34.0°C

Simul Ser No = DR5114  
Std Sol No = 201505B  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Low AC 0.030 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006491  
Location = TOXL      8164.13.00 06/09  
04/12/2016      16:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:40
02 Std. Sol.	0.404	16:40
03 Room Air	0.000	16:41
04 Std. Sol.	0.406	16:42
05 Room Air	0.000	16:43
06 Std. Sol.	0.404	16:43
07 Room Air	0.000	16:44

08 Sim Temp = 34.0°C

Simul Ser No = MP3068

Std Sol No = 14102

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006491  
Location = TOXL      8164.13.00 06/09  
04/12/2016      16:45

Test	AC	Time
01 Room Air	0.000	16:45
02 *Subject Test	INT*	16:46
03 Room Air	0.000	16:47

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*CR0-100 0.10AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006491  
Location = TOXL                    8164.13.00 06/09  
04/12/2016                            16:47

Test	AC	Time
01 Diagnostic	OK	16:48
02 Room Air	RFI*	16:48
03 Room Air	0.000	16:48

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006491  
Location = TOXL      8164.13.00 06/09  
04/12/2016      16:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:49
02 Std. Gas	0.079	16:49
03 Room Air	0.000	16:50
04 Std. Gas	0.079	16:50
05 Room Air	0.000	16:51
06 Std. Gas	0.080	16:51
07 Room Air	0.000	16:52

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006491  
Location = TOXL      8164.13.00 06/09  
04/12/2016      16:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:53
02 Std. Gas	0.081	16:53
03 Room Air	0.000	16:54
04 Std. Gas	0.080	16:54
05 Room Air	0.000	16:55
06 Std. Gas	0.080	16:55
07 Room Air	0.000	16:56

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006491  
Location = TOXL      8164.13.00 06/09  
04/12/2016      16:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:56
02 Std. Gas	0.080	16:57
03 Room Air	0.000	16:57
04 Std. Gas	0.081	16:58
05 Room Air	0.000	16:58
06 Std. Gas	0.080	16:59
07 Room Air	0.000	16:59

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC