

INTOXILYZER 8000 INSPECTION

Serial No.: 80-004951 Location: TOXL

- Check When Done
- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)  
Sim SN DR3455 Lot No. 10290 Rep AC 0.020 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)  
Sim SN DR3847 Lot No. 11110 Rep AC 0.300 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3377 Lot No. — Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 23411080A1 Cyl No. 55 Exp. Date 10/01/2013

Test 1 0.082 Test 4 0.081 Test 7 0.081

Test 2 0.081 Test 5 0.081 Test 8 0.081

Test 3 0.081 Test 6 0.081 Test 9 0.081

Average 0.081

L. Tank pressure, Level 3 Func D Sub G P: 570 psi Reg 550 psi

M. Remarks/Maintenance record: ANNUAL INSPECTION

18 APR 12 Date Charles E. [Signature] Field Inspector's Signature

18 Apr 12 Date D. B. Kashner Reviewed by



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = TOXL      8164.13.00 06/09  
04/18/2012      13:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:35
02 Std. Sol.	0.022	13:36
03 Room Air	0.000	13:36
04 Std. Sol.	0.021	13:37
05 Room Air	0.000	13:38
06 Std. Sol.	0.022	13:38
07 Room Air	0.000	13:39


08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 000

County = 08

Oper No. = 666666

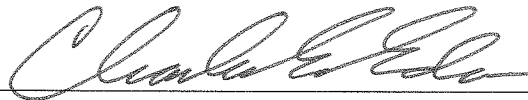

Operator Signature

CHARLES EDER

Remarks:

*Low AC*

*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = TOXL      8164.13.00 06/09  
04/18/2012      13:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:49
02 Std. Sol.	0.302	13:49
03 Room Air	0.000	13:50
04 Std. Sol.	0.304	13:51
05 Room Air	0.000	13:51
06 Std. Sol.	0.304	13:52
07 Room Air	0.000	13:52

08 Sim Temp = 34.0°C

Simul Ser No = 000000

Std Sol No = 000

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

LINEARITY TEST

0.300 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = TOXL      8164.13.00 06/09  
04/18/2012      13:56

Test	AC	Time
01 Room Air	0.000	13:57
02 *Subject Test	INT*	13:57
03 Room Air	0.000	13:58

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = 123456  
Dr. Lic. = ND/DIS821456  
Simul Ser No = 000000  
Std Sol No = 000  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *INTERFERENT CHECK*  
*0.100 AC + ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = TOXL      8164.13.00 06/09  
04/18/2012      13:59

Test	AC	Time
01 Diagnostic	OK	14:00
02 Room Air	RFI*	14:01
03 Room Air	0.000	14:01

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = 123456  
Dr. Lic. = ND/DIS821456  
Simul Ser No = 000000  
Std Sol No = 000  
County = 08      Oper No. = 666666

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = TOXL      8164.13.00 06/09  
04/18/2012      14:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:04
02 Std. Gas	0.082	14:04
03 Room Air	0.000	14:05
04 Std. Gas	0.081	14:05
05 Room Air	0.000	14:06
06 Std. Gas	0.081	14:06
07 Room Air	0.000	14:06

Lot No = 23411080A1  
Cyl No = 55  
Exp Date = 10/01/2013  
County = 08      Oper No. = 666666

  
Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = TOXL      8164.13.00 06/09  
04/18/2012      14:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:10
02 Std. Gas	0.081	14:10
03 Room Air	0.000	14:10
04 Std. Gas	0.081	14:11
05 Room Air	0.000	14:11
06 Std. Gas	0.081	14:12
07 Room Air	0.000	14:12

Lot No = 23411080A1  
Cyl No = 55  
Exp Date = 10/01/2013  
County = 08      Oper No. = 666666

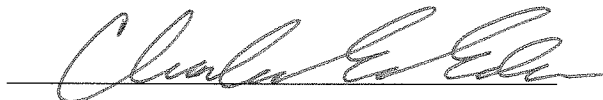

Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = TOXL      8164.13.00 06/09  
04/18/2012      14:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:17
02 Std. Gas	0.081	14:18
03 Room Air	0.000	14:18
04 Std. Gas	0.081	14:19
05 Room Air	0.000	14:19
06 Std. Gas	0.081	14:20
07 Room Air	0.000	14:20


Lot No = 23411080A1

Cyl No = 55

Exp Date = 10/01/2013

County = 08

Oper No. = 666666

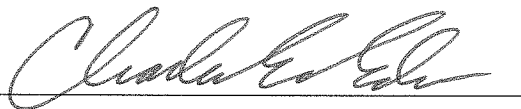

Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000