

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-004951

Location: DEVL

(LAKE REGION  
CORR. CENTER)

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use  $<0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5115 Lot No. 12111 Rep AC 0.020 ✓
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR5143 Lot No. 13080 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.  
(Attach test record.)  
Sim SN DR3375 Lot No. IC5#2 Rep AC 0.100 AC + 0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio on first room air.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.)

Lot No. 16512080A2 Cyl No. 004 Exp. Date 08/01/2014

Test 1 0.081 Test 4 0.080 Test 7 0.081

Test 2 0.079 Test 5 0.079 Test 8 0.080

Test 3 0.079 Test 6 0.079 Test 9 0.079

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 595 psi Reg 600 psi

K. Remarks/Maintenance record: REPLACED<sup>CEL</sup> BREATH HOSE AND SIMULATOR

RETURN O-RINGS, ANNUAL INSPECTION

15 MAY 14  
Date

19 May 14 ✓  
Date

Charles E. Edr  
Field Inspector's Signature

Deb Kashner  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004951  
Location = DEVL            8164.13.00 06/09  
05/15/2014            13:39

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            16512080A2  
Standard Cyl #?            4  
Standard Expiration?        08/01/2014  
Oper No?                    666666

Flow Cal. Date:            06/27/2011  
Slope                        704  
Intercept                   -660586

IR Calibration Date:        05/08/2013  
                                    3um                      9um

0th Coef(*100):	-19649	-21799
1st Coef(*100):	272934	139905
2nd Coef(*100):	-428	720
H2O adj(mg/l*10k):	446	535

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = DEVL      8164.13.00 06/09  
05/15/2014      13:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:42
02 Std. Sol.	0.018	13:43
03 Room Air	0.000	13:43
04 Std. Sol.	0.019	13:44
05 Room Air	0.000	13:45
06 Std. Sol.	0.019	13:45
07 Room Air	0.000	13:46

08 Sim Temp = 34.0°C

Simul Ser No = DR5115

Std Sol No = 12111

County = 36

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = DEVL      8164.13.00 06/09  
05/15/2014      13:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:48
02 Std. Sol.	0.291	13:48
03 Room Air	0.000	13:49
04 Std. Sol.	0.295	13:50
05 Room Air	0.000	13:50
06 Std. Sol.	0.297	13:51
07 Room Air	0.000	13:51

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 13080

County = 36

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = DEVL      8164.13.00 06/09  
05/15/2014      13:56

Test	AC	Time
01 Room Air	0.000	13:57
02 *Subject Test	INT*	13:58
03 Room Air	0.000	13:58

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 4  
Expiration Date = 08/01/2014  
County = 36      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*INTERFERENT CHECK*  
*0.100 AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = DEVL      8164.13.00 06/09  
05/15/2014      13:59

Test	AC	Time
01 Diagnostic	OK	14:00
02 Room Air	RFI*	14:00
03 Room Air	0.000	14:00

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 16512080A2  
Cyl No = 4  
Expiration Date = 08/01/2014  
County = 36      Oper No. = 666666

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = DEVL      8164.13.00 06/09  
05/15/2014      13:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:53
02 Std. Gas	0.081	13:54
03 Room Air	0.000	13:54
04 Std. Gas	0.079	13:55
05 Room Air	0.000	13:55
06 Std. Gas	0.079	13:55
07 Room Air	0.000	13:56

Lot No = 16512080A2  
Cyl No = 4  
Exp Date = 08/01/2014  
County = 36      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION  
CHECK  
0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = DEVL      8164.13.00 06/09  
05/15/2014      14:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:01
02 Std. Gas	0.080	14:01
03 Room Air	0.000	14:02
04 Std. Gas	0.079	14:02
05 Room Air	0.000	14:03
06 Std. Gas	0.079	14:03
07 Room Air	0.000	14:04

Lot No = 16512080A2

Cyl No = 4

Exp Date = 08/01/2014

County = 36

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION  
CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004951  
Location = DEVL      8164.13.00 06/09  
05/15/2014      14:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:04
02 Std. Gas	0.081	14:05
03 Room Air	0.000	14:05
04 Std. Gas	0.080	14:06
05 Room Air	0.000	14:06
06 Std. Gas	0.079	14:06
07 Room Air	0.000	14:07


Lot No = 16512080A2

Cyl No = 4

Exp Date = 08/01/2014

County = 36

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION  
CHECK  
0.080 AC*

Form 106-I8000