

✓CML

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004196

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓

- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5144 Lot No. Rep AC 0.021
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5187 Lot No. Rep AC 0.290
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DRS114 Lot No. Rep AC 0.100
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. <u>617774</u>	Cyl No. <u>48</u>	Exp. Date <u>28 May 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.079</u>	Test 5 <u>0.078</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.080</u>		

L. Remarks/Maintenance record: annual inspection

26 Aug 09
Date

Dob Kashur
Field Inspector's Signature

4 Sept 2009
Date

Mary Plam
Reviewed by

08/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 16:44

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Wet
Standard Value? 0.000
Standard Solution #? 000
Sim Serial #? DR0000
Oper No? 777777

Flow Cal. Date: 06/04/2009
Slope 727
Intercept -600779

IR Calibration Date: 06/03/2009
 3um 9um

0th Coef(*100): -16078 -19878
1st Coef(*100): 252494 128485
2nd Coef(*100): 3196 1141
H2O adj(mg/l*10k): 686 431

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print Test

Form 106-I8000

80-004196
P. 2 of 9 KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 16:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:58
02 Std. Sol.	0.017	16:59
03 Room Air	0.000	17:00
04 Std. Sol.	0.017	17:00
05 Room Air	0.000	17:01
06 Std. Sol.	0.018	17:02
07 Room Air	0.000	17:02

08 Sim Temp = 34.0°C

Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-I8000

*80-004196
P. 3 of 9 PK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 17:05

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:06
02 Std. Sol.	0.284	17:07
03 Room Air	0.000	17:07
04 Std. Sol.	0.288	17:08
05 Room Air	0.000	17:09
06 Std. Sol.	0.288	17:09
07 Room Air	0.000	17:10

08 Sim Temp = 34.0°C

Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

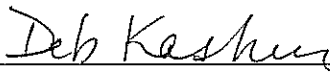

Operator Signature
DEB KASHUR

Remarks:

Linearity Test

Form 106-18000

*80-004196
P. 4 of 9 AM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 17:14

Test	AC	Time
01 Room Air	0.000	17:15
02 *Subject Test	INT*	17:15
03 Room Air	0.000	17:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

^{DK}
Interferent *Interferent check*

Form 106-I8000

80-004196
P. 5/9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 16:50

Test	AC	Time
01 Diagnostic	OK	16:52
02 Room Air	RFI*	16:52
03 Room Air	0.000	16:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004196
P. Kashur

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 16:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:27
02 Std. Gas	0.080	16:27
03 Room Air	0.000	16:28
04 Std. Gas	0.079	16:28
05 Room Air	0.000	16:29
06 Std. Gas	0.080	16:29
07 Room Air	0.000	16:30

Lot No = 17774
Cyl No = 48
Exp Date = 05/28/2011
County = 08 Oper No. = 77777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004196
P. 2 of 9 FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 16:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:32
02 Std. Gas	0.080	16:32
03 Room Air	0.000	16:33
04 Std. Gas	0.078	16:33
05 Room Air	0.000	16:34
06 Std. Gas	0.079	16:34
07 Room Air	0.000	16:35

Lot No = 17774
Cyl No = 48
Exp Date = 05/28/2011
County = 08 Oper No. = 77777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-18000

*80-009 4196
P. 8/29/09 FM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.12.00 06/09
08/26/2009 16:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Gas	0.081	16:36
03 Room Air	0.000	16:37
04 Std. Gas	0.080	16:37
05 Room Air	0.000	16:38
06 Std. Gas	0.079	16:38
07 Room Air	0.000	16:38

Lot No = 17774
Cyl No = 48
Exp Date = 05/28/2011
County = 08 Oper No. = 77777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004196
P. 9 of 9 AM