

✓ emm

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003069

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5144 Lot No. — Rep AC 0.021 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5187 Lot No. — Rep AC 0.290 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.) ✓
Sim SN DR5114 Lot No. — Rep AC 6.100
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 617774 Cyl No. 48 Exp. Date 28 May 2011

Test 1 <u>0.079</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>

Average 0.079

L. Remarks/Maintenance record: Annual inspection

03 Sept 2009 Deb Keshu
Date Field Inspector's Signature

4 Sept 2009 Mary Pearson
Date Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.12.00 06/09
09/03/2009 12:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:58
02 Std. Sol.	0.017	13:03
03 Room Air	0.000	13:04
04 Std. Sol.	0.017	13:04
05 Room Air	0.000	13:05
06 Std. Sol.	0.018	13:05
07 Room Air	0.000	13:06


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777

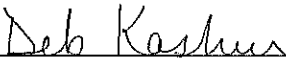

Operator Signature
DEB KASHUR

Remarks:

Low AC

Form 106-I8000

*80-003069
p. 3 of 9
FM*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.12.00 06/09
09/03/2009 15:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:46
02 Std. Sol.	0.283	15:47
03 Room Air	0.000	15:48
04 Std. Sol.	0.285	15:48
05 Room Air	0.000	15:49
06 Std. Sol.	0.285	15:50
07 Room Air	0.000	15:50


08 Sim Temp = 34.0°C

Simul Ser No = DR0000

Std Sol No = 000

County = 08

Oper No. = 777777



Operator Signature
DEB KASHUR

Remarks:

Linearity Test

Form 106-I8000

80-003069
p. 4 of 9
JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.12.00 06/09
09/03/2009 09:45

Test	AC	Time
01 Room Air	0.000	09:46
02 *Subject Test	INT*	09:47
03 Room Air	0.000	09:48

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent test

Form 106-I8000

80-003069
P. Saf 9
AK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.12.00 06/09
09/03/2009 15:43

Test	AC	Time
01 Diagnostic	OK	15:43
02 Room Air	RFI*	15:44
03 Room Air	0.000	15:44

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = DR0000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-003069
p. 6 of 9
JTH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.12.00 06/09
09/03/2009 09:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:33
02 Std. Gas	0.079	09:33
03 Room Air	0.000	09:34
04 Std. Gas	0.079	09:34
05 Room Air	0.000	09:35
06 Std. Gas	0.079	09:35
07 Room Air	0.000	09:35

Lot No = 617774
Cyl No = 48
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003069
P. Jaf 9/3/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003069
Location = TOXL 8164.12.00 06/09
09/03/2009 09:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:40
02 Std. Gas	0.080	09:41
03 Room Air	0.000	09:41
04 Std. Gas	0.079	09:42
05 Room Air	0.000	09:42
06 Std. Gas	0.079	09:43
07 Room Air	0.000	09:43

Lot No = 617774
Cyl No = 48
Exp Date = 05/28/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-003069
P. 7 of 9 JFM