

INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004196

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓

- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5144 Lot No. --- Rep AC 0.019
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
Sim SN DR5187 Lot No. --- Rep AC 0.285
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
Sim SN DR5114 Lot No. --- Rep AC 0.100
- J. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode. (Attach test record.)

Lot No. 659358 Cyl No. 46 Exp. Date 03 Sept 2011
Test 1 0.082 Test 4 0.082 Test 7 0.082
Test 2 0.082 Test 5 0.082 Test 8 0.082
Test 3 0.082 Test 6 0.082 Test 9 0.082
Average 0.082

L. Remarks/Maintenance record: Software 8164.13 installed.

19 NOV 2009
Date

Deb Kashur
Field Inspector's Signature

1 Dec 2009
Date

Mary Pearson
Reviewed by

08/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 10:27

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes

of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659357
Standard Cyl #? 45
Standard Expiration? 09/03/2011
Oper No? 777777

Flow Cal. Date: 06/04/2009
Slope 727
Intercept -600779

IR Calibration Date: 06/03/2009
 3um 9um

0th Coef(*100):	-16078	-19878
1st Coef(*100):	252494	128485
2nd Coef(*100):	3196	1141
H2O adj(mg/l*10k):	686	431

***** Printer Test End *****

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 11:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Sol.	0.017	11:16
03 Room Air	0.000	11:17
04 Std. Sol.	0.017	11:17
05 Room Air	0.000	11:18
06 Std. Sol.	0.018	11:19
07 Room Air	0.000	11:19
08 Sim Temp = 34.0°C		

Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks: *Low AC 0.019 AC*

Form 106-I8000

*80-004196
p. 3 of 9 FR*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 14:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Sol.	0.284	14:43
03 Room Air	0.000	14:43
04 Std. Sol.	0.289	14:44
05 Room Air	0.000	14:45
06 Std. Sol.	0.288	14:45
07 Room Air	0.000	14:46

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.285 AC

Form 106-18000

80-004196
P. 4 of 9 JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 16:16

Test	AC	Time
01 Room Air	0.000	16:16
02 *Subject Test	INT*	16:17
03 Room Air	0.000	16:17

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 46
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

DK
RFL test Interferent test

Form 106-I8000

80-004196
P. Sep 9 JAH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 16:19

Test	AC	Time
01 Diagnostic	OK	16:20
02 Room Air	RFI*	16:20
03 Room Air	0.000	16:20

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 46

Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004196
P. 6 of 9 KH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 16:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:25
02 Std. Gas	0.082	16:26
03 Room Air	0.000	16:26
04 Std. Gas	0.082	16:26
05 Room Air	0.000	16:27
06 Std. Gas	0.082	16:27
07 Room Air	0.000	16:28

Lot No = 659358
Cyl No = 46
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

*80-004196
p. 2 of 9 ym*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 16:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:29
02 Std. Gas	0.082	16:29
03 Room Air	0.000	16:30
04 Std. Gas	0.082	16:30
05 Room Air	0.000	16:30
06 Std. Gas	0.082	16:31
07 Room Air	0.000	16:31

Lot No = 659358
Cyl No = 46
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Keshur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-004196
P. P. J. K.

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.13.00 06/09
11/19/2009 16:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:32
02 Std. Gas	0.082	16:33
03 Room Air	0.000	16:33
04 Std. Gas	0.082	16:34
05 Room Air	0.000	16:34
06 Std. Gas	0.082	16:34
07 Room Air	0.000	16:35

Lot No = 659358
Cyl No = 46
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check

Form 106-I8000

80-00 4196
P. 9 of 9 PK