



INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-004188

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5144 Lot No. Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5187 Lot No. Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5114 Lot No. Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. <u>659358</u>	Cyl No. <u>35</u>	Exp. Date <u>03 Sept 2011</u>
Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

L. Remarks/Maintenance record: Software upgrade 8164.13 Annual inspection
14 Dec 2009 Deb Kashner
Date Field Inspector's Signature
15 Dec 2009 Mary Pearson

08/09

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 07:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 659358
Standard Cyl #? 35
Standard Expiration? 09/03/2011
Oper No? 777777

Flow Cal. Date: 06/04/2009
Slope 689
Intercept -630047

IR Calibration Date: 12/11/2009

	3um	9um
0th Coef(*100):	-36132	-37983
1st Coef(*100):	283422	138305
2nd Coef(*100):	-135	610
H2O adj(mg/l*10k):	688	672

***** Printer Test End *****

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-004188
P. 2 of 9 ym

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 10:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:35
02 Std. Sol.	0.021	10:36
03 Room Air	0.000	10:37
04 Std. Sol.	0.020	10:37
05 Room Air	0.000	10:38
06 Std. Sol.	0.021	10:39
07 Room Air	0.000	10:39

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC 0.019 AC

Form 106-18000

80-004188
P-349 KM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 10:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:42
02 Std. Sol.	0.288	10:47
03 Room Air	0.000	10:47
04 Std. Sol.	0.290	10:48
05 Room Air	0.000	10:48
06 Std. Sol.	0.290	10:49
07 Room Air	0.000	10:50

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.285 AC

Form 106-I8000

80-004188
P. 4/9 KH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 12:56

Test	AC	Time
01 Room Air	0.000	12:57
02 *Subject Test	INT*	12:57
03 Room Air	0.000	12:57

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 35
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Interferent test

Form 106-18000

80-004188
P. 5 of 9 PM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 14:40

Test	AC	Time
01 Diagnostic	OK	14:40
02 Room Air	RFI*	14:41
03 Room Air	0.000	14:41

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 35
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-004188
P. 6 of 7 1/14

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 15:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:40
02 Std. Gas	0.080	15:41
03 Room Air	0.000	15:41
04 Std. Gas	0.080	15:41
05 Room Air	0.000	15:42
06 Std. Gas	0.079	15:42
07 Room Air	0.000	15:43

Lot No = 659358
Cyl No = 35
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-004188
A. Tol 9 Feb

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 15:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:45
02 Std. Gas	0.080	15:45
03 Room Air	0.000	15:46
04 Std. Gas	0.080	15:46
05 Room Air	0.000	15:47
06 Std. Gas	0.080	15:47
07 Room Air	0.000	15:47

Lot No = 659358
Cyl No = 35
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

*80-004188
P. Def 7/11*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004188
Location = TOXL 8164.13.00 06/09
12/14/2009 16:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:12
02 Std. Gas	0.080	16:13
03 Room Air	0.000	16:13
04 Std. Gas	0.080	16:14
05 Room Air	0.000	16:14
06 Std. Gas	0.080	16:14
07 Room Air	0.000	16:15

Lot No = 659358
Cyl No = 35
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

0.080 AC calibration check

Form 106-I8000

80-004188
p. 9 of 9 KM