



INTOXILYZER 5000 G and 8000 INSPECTION

Serial No.: 80-003071

Location: TOXL

Check When Done

- A. Is the warm-up time less than 20 minutes? ✓
- B. Is a three-pronged grounded outlet used? ✓
- C. Is the breath tube heated? ✓
- D. Is the diagnostic check complete? ✓
- E. Is the time, date, and year correct (re-set if necessary)? ✓
- F. Print test. (Attach test record.) ✓
- G. Low AC. Use <0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5144 Lot No. Rep AC 0.019 ✓
- H. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR5187 Lot No. Rep AC 0.285 ✓
- I. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode.
(Attach test record.)
Sim SN DR5114 Lot No. Rep AC 0.100 ✓
- J. RFI check. Run CMS mode. Key radio on first room air.
(Attach test record.) ✓
- K. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.)

Lot No. 659358 Cyl No. 34 Exp. Date 03 Sept 2011

Test 1 0.080 Test 4 0.080 Test 7 0.081

Test 2 0.080 Test 5 0.080 Test 8 0.080

Test 3 0.080 Test 6 0.079 Test 9 0.080

Average 0.080

L. Remarks/Maintenance record: Software upgrade 8164.13. Annual inspection

15 Dec 2009
Date

Deb Kashur
Field Inspector's Signature

15 Dec 2009
Date

Maryn Blum
Reviewed by

80-003071
p. 1 of 9

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 09:16

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Wet
Standard Value? 0.000
Standard Solution #? 000
Sim Serial #? 00000
Oper No? 777777

Flow Cal. Date: 04/08/2008
Slope 696
Intercept -591525

IR Calibration Date: 12/15/2009

	3um	9um
0th Coef(*100):	-23386	-16310
1st Coef(*100):	273044	136944
2nd Coef(*100):	-333	360
H2O adj(mg/l*10k):	568	414

***** Printer Test End *****

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Print test

Form 106-I8000

80-003071
P. 2 of 9 STM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 10:36

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:37
02 Std. Sol.	0.019	10:37
03 Room Air	0.000	10:38
04 Std. Sol.	0.019	10:39
05 Room Air	0.000	10:39
06 Std. Sol.	0.019	10:40
07 Room Air	0.000	10:40

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

low AC 0.019 AC

Form 106-18000

80-003071
P. 3 of 9 JK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 10:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:19
02 Std. Sol.	0.283	10:20
03 Room Air	0.000	10:21
04 Std. Sol.	0.283	10:21
05 Room Air	0.000	10:22
06 Std. Sol.	0.284	10:23
07 Room Air	0.000	10:23

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Linearity test

Form 106-18000

80-003071
A. 4/2/9 10/12

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 13:11

Test	AC	Time
01 Room Air	0.000	13:12
02 *Subject Test	INT*	13:12
03 Room Air	0.000	13:13

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 34
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent Check

Form 106-I8000

80-003071
P. 5 of 9 JKH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 13:24

Test	AC	Time
01 Diagnostic	OK	13:25
02 Room Air	RFI*	13:25
03 Room Air	0.000	13:26

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 659358
Cyl No = 34
Expiration Date = 09/03/2011
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

RFI test

Form 106-I8000

80-003071
P. 6 of 9 JAH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 10:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:50
02 Std. Gas	0.080	10:51
03 Room Air	0.000	10:51
04 Std. Gas	0.080	10:51
05 Room Air	0.000	10:52
06 Std. Gas	0.080	10:52
07 Room Air	0.000	10:53

Lot No = 659358
Cyl No = 34
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kasher
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 Ac

Form 106-I8000

80-003071
P. 7 of 9 1/12

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 10:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:55
02 Std. Gas	0.080	10:55
03 Room Air	0.000	10:56
04 Std. Gas	0.080	10:56
05 Room Air	0.000	10:56
06 Std. Gas	0.079	10:57
07 Room Air	0.000	10:57

Lot No = 659358

Cyl No = 34

Exp Date = 09/03/2011

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003071
P. 8 of 9 JDR

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = TOXL 8164.13.00 06/09
12/15/2009 10:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:58
02 Std. Gas	0.081	10:58
03 Room Air	0.000	10:59
04 Std. Gas	0.080	10:59
05 Room Air	0.000	11:00
06 Std. Gas	0.080	11:00
07 Room Air	0.000	11:01

Lot No = 659358
Cyl No = 34
Exp Date = 09/03/2011
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003071
P. 9 of 9 pm