

INTOXILYZER® 8000 INSPECTION

Serial No.: 20-004193

Location: TOXL

Check When Done

- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR 3846 Lot No. 16200 Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR 7111 Lot No. 16060 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR 5144 Lot No. 1C55 Solution 0.16 AC + 0.05% acetone ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 27 Exp. Date 08/05/2018

Test 1 0.080 Test 4 0.080 Test 7 0.081

Test 2 0.080 Test 5 0.081 Test 8 0.081

Test 3 0.081 Test 6 0.080 Test 9 0.080

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 179 psi Reg 180 psi

K. Remarks/Maintenance record: Annual inspection, cleaned intox, cleaned particulate screen, replaced breath hose o-ring & handle, ^{is recalibrated} recal tank sensor (regulator).

09 June 17
Date

Deb Shanaver
Field Inspector's Signature

09 JUNE 2017
Date

Charles E. Edin
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 10:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 777777

Flow Cal. Date: 06/04/2009
Slope 692
Intercept -611138

IR Calibration Date: 04/23/2015
 3um 9um

0th Coef(*100): -21817 -20283
1st Coef(*100): 254491 124899
2nd Coef(*100): 4701 1877
H2O adj(mg/l*10k): 502 469

***** Printer Test End *****

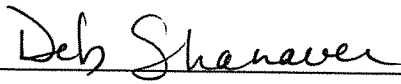

Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 11:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:02
02 Std. Sol.	0.018	11:02
03 Room Air	0.000	11:03
04 Std. Sol.	0.018	11:04
05 Room Air	0.000	11:04
06 Std. Sol.	0.018	11:05
07 Room Air	0.000	11:05

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 10:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:54
02 Std. Sol.	0.295	10:55
03 Room Air	0.000	10:56
04 Std. Sol.	0.296	10:56
05 Room Air	0.000	10:57
06 Std. Sol.	0.296	10:58
07 Room Air	0.000	10:58

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 16060

County = 08

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 11:08

Test	AC	Time
01 Room Air	0.000	11:08
02 *Subject Test	INT*	11:09
03 Room Air	0.000	11:09

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Interferent detected 0.10AC + 0.05% acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 11:11

Test	AC	Time
01 Diagnostic	OK	11:11
02 Room Air	RFI*	11:11
03 Room Air	0.000	11:12

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
DEB SHANAVER

Remarks:

RFI check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 11:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Gas	0.080	11:15
03 Room Air	0.000	11:15
04 Std. Gas	0.080	11:16
05 Room Air	0.000	11:16
06 Std. Gas	0.081	11:17
07 Room Air	0.000	11:17

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 11:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:22
02 Std. Gas	0.080	11:23
03 Room Air	0.000	11:23
04 Std. Gas	0.081	11:23
05 Room Air	0.000	11:24
06 Std. Gas	0.080	11:24
07 Room Air	0.000	11:25

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
06/09/2017 11:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:30
02 Std. Gas	0.081	11:31
03 Room Air	0.000	11:31
04 Std. Gas	0.081	11:32
05 Room Air	0.000	11:32
06 Std. Gas	0.080	11:32
07 Room Air	0.000	11:33

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC