

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-003071

Location: TOXL

**A. General Setup and Checks:**

- 1.  Diagnostic Tests Pass and Instrument in Ready Mode
- 2.  Breath tube heated
- 3.  Date, time and location code (Level 2,E). Re-set if necessary.
- 4.  Print test (Level 1,P). Sign and attach test record.
- 5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 188 psi Regulator: 200 psi

**B. Tests (Sign and attach test records):**

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3063 Lot #: 16200 Exp. Date: 17 Aug 18
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: mp3065 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

- 2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR5144 Lot #: 1055 Exp. Date: 15 May 19  
 Display reads "Interferent Detect"
- 3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
- 4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 20016080A2 Cyl #: 48 Exp. Date: 5 Sep 18

|                      |              |        |              |        |              |
|----------------------|--------------|--------|--------------|--------|--------------|
| Test 1               | <u>0.080</u> | Test 4 | <u>0.080</u> | Test 7 | <u>0.080</u> |
| Test 2               | <u>0.079</u> | Test 5 | <u>0.080</u> | Test 8 | <u>0.080</u> |
| Test 3               | <u>0.079</u> | Test 6 | <u>0.080</u> | Test 9 | <u>0.080</u> |
| Average <u>0.080</u> |              |        |              |        |              |

C. Remarks/Maintenance: Annual inspection. Replaced o ring on simulator return.

Instrument is acceptable to be used in the field.

Roberts Hugo - Nimmo  
Breath Analyst Signature

17 APR 18  
Date

Charles E. Edm  
Reviewed by

18 APR 18  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      11:46

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      20016080A2  
Standard Cyl #?      48  
Standard Expiration?      09/05/2018  
Oper No?      888888

Flow Cal. Date:      07/09/2010  
Slope      806  
Intercept      -994737

IR Calibration Date:      04/10/2015

|                    | 3um    | 9um    |
|--------------------|--------|--------|
| 0th Coef(*100):    | -11328 | -10938 |
| 1st Coef(*100):    | 268087 | 134278 |
| 2nd Coef(*100):    | 3093   | 1368   |
| H2O adj(mg/l*10k): | 242    | 233    |

\*\*\*\*\* Printer Test End \*\*\*\*\*

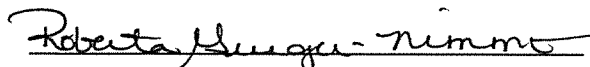

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      12:10

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 12:10 |
| 02 Std. Sol. | 0.020 | 12:11 |
| 03 Room Air  | 0.000 | 12:11 |
| 04 Std. Sol. | 0.019 | 12:12 |
| 05 Room Air  | 0.000 | 12:13 |
| 06 Std. Sol. | 0.020 | 12:13 |
| 07 Room Air  | 0.000 | 12:14 |


08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16200

County = 08

Oper No. = 888888

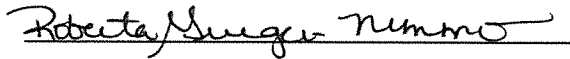

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: low AC  
0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      12:16

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 12:16 |
| 02 Std. Sol. | 0.297 | 12:17 |
| 03 Room Air  | 0.000 | 12:18 |
| 04 Std. Sol. | 0.296 | 12:19 |
| 05 Room Air  | 0.000 | 12:19 |
| 06 Std. Sol. | 0.297 | 12:20 |
| 07 Room Air  | 0.000 | 12:21 |

08 Sim Temp = 34.0°C

Simul Ser No = MP3065

Std Sol No = 17350

County = 08

Oper No. = 888888

*High Roberta Grieger-Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *High AC*  
*0.300 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      11:59

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 12:00 |
| 02 *Subject Test | INT*  | 12:01 |
| 03 Room Air      | 0.000 | 12:01 |

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08      Oper No. = 888888

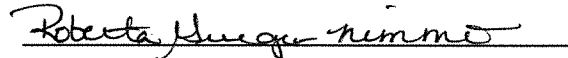  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*  
*0.10 AC + 0.05% Acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      12:02

| Test          | AC    | Time  |
|---------------|-------|-------|
| 01 Diagnostic | OK    | 12:03 |
| 02 Room Air   | RFI*  | 12:03 |
| 03 Room Air   | 0.000 | 12:03 |

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08      Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

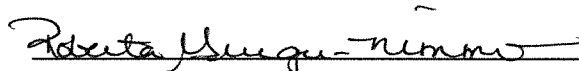

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      11:47

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 11:47 |
| 02 Std. Gas | 0.080 | 11:48 |
| 03 Room Air | 0.000 | 11:48 |
| 04 Std. Gas | 0.079 | 11:48 |
| 05 Room Air | 0.000 | 11:49 |
| 06 Std. Gas | 0.079 | 11:49 |
| 07 Room Air | 0.000 | 11:50 |

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

Roberta Grieger Nimmro  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      11:50

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 11:51 |
| 02 Std. Gas | 0.080 | 11:51 |
| 03 Room Air | 0.000 | 11:52 |
| 04 Std. Gas | 0.080 | 11:52 |
| 05 Room Air | 0.000 | 11:53 |
| 06 Std. Gas | 0.080 | 11:53 |
| 07 Room Air | 0.000 | 11:53 |

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

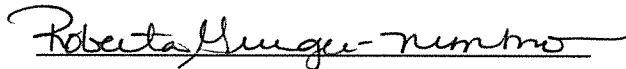

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
04/17/2018      11:54

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 11:54 |
| 02 Std. Gas | 0.080 | 11:55 |
| 03 Room Air | 0.000 | 11:55 |
| 04 Std. Gas | 0.080 | 11:56 |
| 05 Room Air | 0.000 | 11:56 |
| 06 Std. Gas | 0.080 | 11:56 |
| 07 Room Air | 0.000 | 11:57 |

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

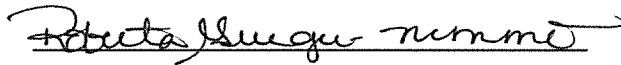

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC