



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (5-2011)

Chemical Test Operator Name (Print) <i>Kirby, Amber</i>		
Location <i>FYPD</i>	Intoxilyzer® Serial Number <i>80-004193</i>	
Gas Lot Number <i>33913080A4</i>	Gas Cylinder Number <i>15</i>	Gas Expiration Date <i>01/01/2016</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.078* AC
 - B. *0.078* AC
 - C. *0.078* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>9/8/14</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>15 Sept 14</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = FYPD 8164.13.00 06/09
09/08/2014 17:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:27
02 Std. Gas	0.078	17:28
03 Room Air	0.000	17:28
04 Std. Gas	0.078	17:29
05 Room Air	0.000	17:29
06 Std. Gas	0.078	17:30
07 Room Air	0.000	17:30

Lot No = 33913080A4
Cyl No = 15
Exp Date = 01/01/2016
County = 61 Oper No. = 021112

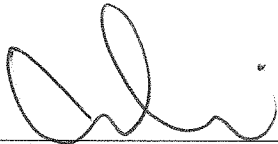

Operator Signature
AMBER KIRBY

Remarks: