



**ETHANOL BREATH STANDARD CYLINDER REPORT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59282 (5-2011)

Chemical Test Operator Name (Print) <i>Johnny Z Lawson</i>		
Location <i>BOWMAN SO</i>	Intoxilyzer® Serial Number <i>80-004188</i>	
Gas Lot Number <i>234/1080A1</i>	Gas Cylinder Number <i>69</i>	Gas Expiration Date <i>10/01/13</i>

Check When Done:

- JUL*  1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.079* AC  
 B. *0.079* AC  
 C. *0.079* AC
3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
5. Send the Following to the Crime Laboratory:  
 A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G).  
 B. ACA Test Record.

Chemical Test Operator Signature <i>Johnny Z Lawson</i>	Date <i>9-26-12</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>09 oct 2012</i>

ATTN: Deb


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004188  
Location = BOWM                8164.13.00 06/09  
09/26/2012                        10:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:36
02 Std. Gas	0.079	10:36
03 Room Air	0.000	10:37
04 Std. Gas	0.079	10:37
05 Room Air	0.000	10:37
06 Std. Gas	0.079	10:38
07 Room Air	0.000	10:38

Lot No = 23411080A1  
Cyl No = 69  
Exp Date = 10/01/2013  
County = 06                            Oper No. = 020814

  
\_\_\_\_\_  
Operator Signature  
JOHNNY Z. LAWSON

Remarks: