



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Eberle Frank T</i>		
Location <i>Bowman County Sheriff's office</i>	Intoxilyzer® Serial Number <i>80-006496</i>	
Gas Lot Number <i>34418080A2</i>	Gas Cylinder Number <i>41</i>	Gas Expiration Date <i>02-05-2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.081 AC
 B. 0.081 AC
 C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Frank Eberle</i>	Date <i>8-5-2019</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles Eberle</i>	Date <i>8.5.2019</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006496
Location = BOWM 8164.13.00 06/09
08/05/2019 11:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:13
02 Std. Gas	0.081	11:13
03 Room Air	0.000	11:14
04 Std. Gas	0.081	11:14
05 Room Air	0.000	11:15
06 Std. Gas	0.081	11:15
07 Room Air	0.000	11:16

Lot No = 34418080A2
Cyl No = 41
Exp Date = 02/05/2021
County = 06 Oper No. = 130015

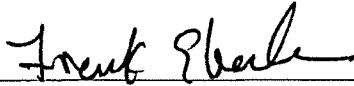

Operator Signature
FRANK EBERLE

T.

Remarks:

Form 106-I8000



INTOXILYZER® RECORD
ND OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
SFN 50496 (05/2018)

Intoxilyzer® Serial Number 80-0064 96		Location Bowman County Sheriff's Office	
Gas Lot Number 34418080A2	Gas Cylinder Number 41	Gas Expiration Date 02-05-2021	

Test Date	Chemical Test Operator Number	Operator's Name (PRINT Last Name, First, MI)	
8-5-2019	130015	<i>Frank T. Eberle</i>	Eberle, Frank T <i>set up</i>