



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Shane Johnson</i> 4331		
Location <i>COOP</i>		Intoxilyzer® Serial Number <i>80-004952</i>
Gas Lot Number <i>26021080A1</i>	Gas Cylinder Number <i>27</i>	Gas Expiration Date <i>10-05-2023</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.029* AC
 B. *0.080* AC
 C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i> 4331	Date <i>11-15-21</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>11/16/2021</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = COOP 8164.14.00 09/16
11/15/2021 17:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:19
02 Std. Gas	0.079	17:19
03 Room Air	0.000	17:20
04 Std. Gas	0.080	17:20
05 Room Air	0.000	17:21
06 Std. Gas	0.080	17:21
07 Room Air	0.000	17:21

Lot No = 26021080A1
Cyl No = 27
Exp Date = 10/05/2023
County = 20 Oper No. = 131650

 4331

Operator Signature
SHANE JOHNSON

Remarks: ACA test

Form 106-I8000