



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (1-2010)

Serial Number: 80-004184 Instrument Location: Williams Co. LEC

Reason for Install/Repair:

Install After Lab Repair/Inspection

Other (Specify) Replaced old Intoxilyzer sent in for calibration

Check When Done:

- A. Surge Protector Installed/Properly Grounded.
- B. Telephone Line Connected to Intoxilyzer® 8000.
- C. Breath Tube Heated.
- D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- E. Review/Enter Gas Setup (Level1, Function S).
- F. Print Test (Level 1, Function P). (Attach Test Record.)
- G. ACA Test (Level 1, Function C). (Attach Test Record.)
- H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)
- I. Repair and/or Maintenance Performed: _____

Sammie M. Husell
Field Inspector Signature

7 Jan 2011 ^{2011 DK}
Date

Deb Kashur
Reviewed By

28 Jan 2011
Date

80-004184
P. 98 of 101
AM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = WILL 8164.13.00 06/09
01/07/2011 14:01

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-==|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-==|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

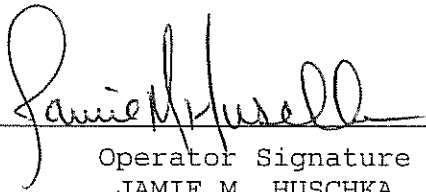
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 15410080A1
Standard Cyl #? 6
Standard Expiration? 07/01/2012
Oper No? 100406

Flow Cal. Date: 07/30/2010
Slope 804
Intercept -925008

IR Calibration Date: 07/30/2010
 3um 9um

0th Coef(*100):	-19612	-18975
1st Coef(*100):	270159	137378
2nd Coef(*100):	1174	963
H2O adj(mg/l*10k):	427	372

***** Printer Test End *****


Operator Signature
JAMIE M. HUSCHKA

Remarks:

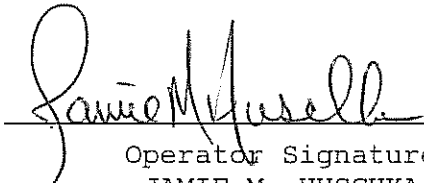
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = WILL 8164.13.00 06/09
01/07/2011 14:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:03
02 Std. Gas	0.080	14:04
03 Room Air	0.000	14:04
04 Std. Gas	0.080	14:05
05 Room Air	0.000	14:05
06 Std. Gas	0.080	14:05
07 Room Air	0.000	14:06

Lot No = 15410080A1
Cyl No = 6
Exp Date = 07/01/2012
County = 53 Oper No. = 100406


Operator Signature
JAMIE M. HUSCHKA

Remarks:

Form 106-I8000

80-004184
P. 100 of 101
VAM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

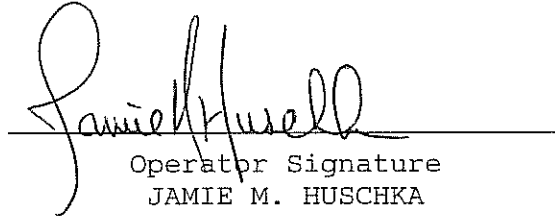
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004184
Location = WILL 8164.13.00 06/09
01/07/2011 14:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:08
02 Reference	RFI*	14:08
03 Room Air	0.000	14:09

*Invalid Test
Inhibited - RFI

Lot No = 15410080A1
Cyl No = 6
Exp Date = 07/01/2012
County = 53 Oper No. = 100406


Operator Signature
JAMIE M. HUSCHKA

Remarks:

Form 106-18000

80-004184
P. 106-1101
JMH