



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (1-2010)

Serial Number: 80-004951

Instrument Location: New Town

Reason for Install/Repair:

☒ Install After Lab Repair/Inspection

☐ Other (Specify) _____

Check When Done:

- ☒ A. Surge Protector Installed/Properly Grounded.
- ☒ B. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ C. Breath Tube Heated.
- ☒ D. Review/Enter Preliminary Data Entry (Level 2, Function E).
- ☒ E. Review/Enter Gas Setup (Level 1, Function S).
- ☒ F. Print Test (Level 1, Function P). (Attach Test Record.)
- ☒ G. ACA Test (Level 1, Function C). (Attach Test Record.)
- ☒ H. RFI Test (Use CMS Mode-First Room Air). (Attach Test Record.)

I. Repair and/or Maintenance Performed: _____

Stirling Reed
Field Inspector Signature

3-19-10
Date

Deb Kasher
Reviewed By

29 Mar 2010
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = NEWT 8164.13.00 06/09
03/29/2010 21:14

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 709352
Standard Cyl #? 36
Standard Expiration? 12/23/2011
Oper No? 020717

Flow Cal. Date: 01/19/2010
Slope 698
Intercept -658617

IR Calibration Date: 01/19/2010
3um 9um


0th Coef(*100):	-24390	-26496
1st Coef(*100):	258099	132326
2nd Coef(*100):	1745	1171
H2O adj(mg/l*10k):	509	459

***** Printer Test End *****

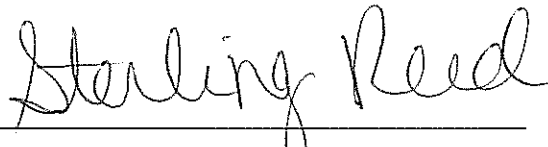

Operator Signature
STERLING R. REED

Remarks:

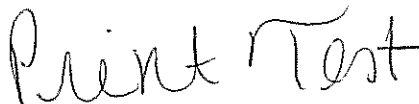

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = NEWT 8164.13.00 06/09
03/29/2010 21:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:16
02 Std. Gas	0.079	21:16
03 Room Air	0.000	21:17
04 Std. Gas	0.079	21:17
05 Room Air	0.000	21:17
06 Std. Gas	0.078	21:18
07 Room Air	0.000	21:18

Lot No = 709352
Cyl No = 36
Exp Date = 12/23/2011
County = 60 Oper No. = 020717

Sterling Reed

Operator Signature
STERLING R. REED

Remarks:

ACA Test

Form 106-I8000

80-004951
36/37 1-11


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004951
Location = NEWT 8164.13.00 06/09
03/29/2010 21:21

Test	AC	Time
01 Diagnostic	OK	21:24
02 Room Air	RFI*	21:25
03 Room Air	0.000	21:25

*Invalid Test
Inhibited - RFI

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

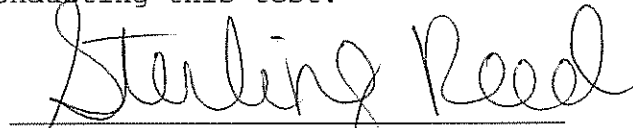

Operator Signature
STERLING R. REED

Remarks:

RFI Test

Form 106-I8000

80-004951
37/37 KM