



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (03-2014)

Serial Number <i>80-006494</i>	Instrument Location <i>VALL</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Dan Rutell</i>	Date <i>5/19/2016</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>14 Oct 16</i>



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006494  
Location = VALL      8164.13.00 06/09  
05/19/2016      14:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:52
02 Std. Gas	0.079	14:52
03 Room Air	0.000	14:53
04 Std. Gas	0.079	14:53
05 Room Air	0.000	14:54
06 Std. Gas	0.079	14:54
07 Room Air	0.000	14:55


Lot No = 01316080A2

Cyl No = 31

Exp Date = 02/05/2018

County = 02

Oper No. = 100012



Operator Signature  
DANA RUSTEBAKKE

Remarks:

ACA Test

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006494  
Location = VALL      8164.13.00 06/09  
05/19/2016      14:58

Test	AC	Time
01 Diagnostic	OK	15:00
02 Room Air	0.000	15:01
03 *Subject Test	RFI*	15:01
04 Room Air	0.000	15:02

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, RFI NA  
Sub DOB = 01/01/2000  
Sub Sex = Male      Weight = 175  
Test = DUI      Cit = 123456  
Dr. Lic. = ND/123456  
Lot No = 01316080A2  
Cyl No = 31  
Expiration Date = 02/05/2018  
County = 02      Oper No. = 100012

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
DANA RUSTEBAKKE

Remarks:

RFI Test

Form 106-I8000