

Alco-Sensor FST

Quick Reference Status Messages

E03	Blank Timeout (> 60 seconds)
E06	Exceeded Sample Attempt Allotment
E07	Bad Calibration (Invalid Sample)
E09	Temp Too Cold for Test Type Being Performed
E10	Temp Too Hot for Test Type Being Performed
E11	Failed Air Blank (Replace Mouthpiece with New Mouthpiece & Begin Test Again)
E12	RFI Detection Error
E21	Invalid Sample
E22	Invalid Calibration Target (Target value entered must be between .015 & .200)
E29	Fuel Cell Out of Range
E30	Pressure Sensor Baseline Out of Range
E31	Result Over Range

Refer to the Operators Manual for any Status Messages not shown above or Contact Intoximeters.

(314) 429-4000

Intoximeters Inc.

Alco-Sensor FST[®]

Basic Operation Checklist

1. Attach Mouthpiece

Insert closed end of mouthpiece into end of channel, single hole facing up. Lever/push down to lock into place.

2. Power Instrument ON.

Press the ON button (large button) with trigger/index finger to power unit ON. Hold down for several seconds if back lighting is preferred.

3. Battery condition and temperature will display.

4. When 'blo' appears, instruct subject on how to properly provide a breath sample.

"Take a deep breath, hold it for a moment and blow steadily into the mouthpiece for as long as you can".

5. Note the subject's result.

6. Remove & discard the mouthpiece.

7. Turn the instrument OFF.

Press the OFF button (small button under display) or the instrument will auto-power down after fifteen seconds.

For On Line Certified Training and a printable Operators Manual go to:

<http://training.intox.com/asfst>

Intoximeters Inc.

(314) 429-4000

www.intox.com

Alco-Sensor FST

Accessing On-Line Training & Manual

go to: <http://training.intox.com/asfst>

The following options are available without a login:

- 1 ASFST Operators Cheat Sheet
- 2 Calibration Log Sheet
- 3 NHTSA Approval Letter

Creating a login is simple and free. After logging in you will have access to:

- 1 Individual Online Training
- 2 Group Online Training
- 3 ASFST Operators Manual

Call **(314) 429-4000** for:

- * Technical Assistance
- * Supplies
- * Repair Information

Intoximeters Inc.

Alco-Sensor FST

Accessing Menu Options

1. Hold down OFF button while pressing ON button to access MENU features.
2. Press ON button to cycle through MENU options (listed below).
3. Press OFF button to select displayed option.

MENU Listing

(A password may be required)

rcL Recall

Display last test.

PAS Passive Test Mode

Replace mouthpiece with Passive Sampling. Cup. Ask subject to blow at cup until instrument samples.

rbL Roadblock Test Mode

Allows rapid repeat testing. Press ON button after test result to begin another test.

ACC Accuracy Check

For use by Maintenance Technician.

CAL Calibration

For use by Maintenance Technician.

dSr Description

For use by Maintenance Technician.

Intoximeters Inc.