

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-007096

A. GENERAL Setup and Checks:

- Power On
 - 5 min. countdown during initial warm-up
 - 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
- Password Security:
 - Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
- Barcode Reader (Level 1,B):
 - Operator ID ND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
- Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-007096 DSP Ver: 1362.14
 - Core Ver: 8164.16 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

- DVM Monitor (Level 3,D,D): 3: 12909 s: 1 C: 47.00
9: 13374 s: 1 B: 45 0
(RFI indicator with keyed radio): 1
- Function (F) Keys: : F5 – Tone F9 - Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid
- Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 963 CF: 1.05
- Flow Monitor (Level 3,D,F): Flow Sensor, R: 208 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 47.5 I: 0.0
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 639 psi P (gauge): 650 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: DR3377 Lot #: MILLI Q H₂O Exp date: NA
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: DR7111 Lot #: 17160 Exp date: 6.8.19
Known value: 0.020 AC
Results: 0.020 AC 0.019 AC 0.019 AC
3. High AC (Level 1,C):
Sim ser #: DR3375 Lot #: 17350 Exp date: 10.11.19
Known value: 0.300 AC
Results: 0.299 AC 0.301 AC 0.301 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR7089 Lot #: ICS #5 Exp date: 5.15.19
Known value: 0.10 AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR3846 Lot #: ACE 1 Exp date: NA
Known value: 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR3455 Lot #: MET 2 Exp date: NA
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR5189 Lot #: I PA 2 Exp date: NA
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):
Ave: 0.0808 Std Dev: 0.0006 Rel Std Dev: 0.69%

Maintenance/Repairs/Notes: INITIAL INSPECTION AFTER RECEIVING
FROM CMI, INC.

Instrument is acceptable to be used in the field.

Charles E. Eden
Field Inspector's Signature

10/22/2018
Date

Robert Yungqu-Nimmo
Reviewed by

22 Oct 18
Date

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 11:39

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

24 characters per line
abcdefghijklmnopqrstuvw
ABCDEFGHIJKLMNQRSTUVW

32 characters per line
abcdefghijklmnopqrstvwxyz123456
ABCDEFGHIJKLMNQRSTUvwxyz123456

42 characters per line
abcdefghijklmnopqrstvwxyz1234567890abcdef
ABCDEFGHIJKLMNQRSTUvwxyz1234567890ABCDEF

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 80
Standard Cyl #? 111
Standard Expiration? 11/11/2019
Oper No? 666666

Flow Cal. Date: 10/02/2018
Slope 695
Intercept -692605

IR Calibration Date: 10/02/2018
3um 9um

0th Coef(*100):	-33544	-18475
1st Coef(*100):	261464	138054
2nd Coef(*100):	3020	1612
H2O adj(mg/l*10k):	573	386

***** Test End *****



Operator Signature
CHARLES EDER

Remarks: INTERNAL
PRINT TEST

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 11:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%\$^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%\$^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 80
Standard Cyl #? 111
Standard Expiration? 11/11/2019
Oper No? 666666

Flow Cal. Date: 10/02/2018
Slope 695
Intercept -692605

IR Calibration Date: 10/02/2018
 3um 9um

0th Coef(*100): -33544 -18475
1st Coef(*100): 261464 138054
2nd Coef(*100): 3020 1612
H2O adj(mg/l*10k): 573 386

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

EXTERNAL PRINT
TEST


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 11:44

Test	AC	Time
01 Room Air	0.000	11:46
02 *Subject Test	REF*	11:46
03 Room Air	0.000	11:46

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = R KEY REFUSAL
Dr. Lic. = ND/DIS821456
Lot No = 80
Cyl No = 111
Expiration Date = 11/11/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *R KEY REFUSAL*
BARCODE READER

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 11:48

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

DIAGNOSTIC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 12:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:01
02 Std. Sol.	0.000	12:02
03 Room Air	0.000	12:03
04 Std. Sol.	0.000	12:03
05 Room Air	0.000	12:04
06 Std. Sol.	0.000	12:05
07 Room Air	0.000	12:05

08 Sim Temp = 34.0°C

Simul Ser No = DR3377
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

ZERO AC
0.000 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 12:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:07
02 Std. Sol.	0.020	12:08
03 Room Air	0.000	12:09
04 Std. Sol.	0.019	12:09
05 Room Air	0.000	12:10
06 Std. Sol.	0.019	12:11
07 Room Air	0.000	12:11


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17160

County = 08

Oper No. = 666666

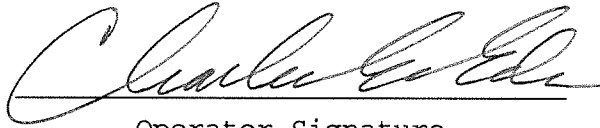

Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 12:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:14
02 Std. Sol.	0.299	12:14
03 Room Air	0.000	12:15
04 Std. Sol.	0.301	12:16
05 Room Air	0.000	12:16
06 Std. Sol.	0.301	12:17
07 Room Air	0.000	12:18

08 Sim Temp = 34.0°C

Simul Ser No = DR3375
Std Sol No = 17350
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:01

Test	AC	Time
01 Room Air	0.000	13:02
02 *Subject Test	INT*	13:02
03 Room Air	0.000	13:03

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferant Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:04

Test	AC	Time
01 Room Air	0.000	13:05
02 *Subject Test	INT*	13:05
03 Room Air	0.000	13:06

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:06

Test	AC	Time
01 Room Air	0.000	13:07
02 *Subject Test	INT*	13:07
03 Room Air	0.000	13:08

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: 0.10% METHANOL
INTERFERENT CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:08

Test	AC	Time
01 Room Air	0.000	13:09
02 *Subject Test	INT*	13:09
03 Room Air	0.000	13:10

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

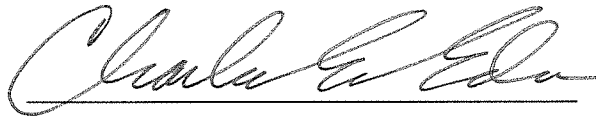

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10% ISOPROPAOL

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:11

Test	AC	Time
01 Room Air	AMB*	13:12
02 Room Air	0.000	13:12

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *AMBIENT FAIL*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:13

Test	AC	Time
01 Room Air	0.000	13:14
02 Invalid Sample	X.XXX	13:14
03 Room Air	0.000	13:15

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INVALID SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INVALID SAMPLE*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:15

Test	AC	Time
01 Room Air	0.000	13:16
02 *Subject Test	ABT*	13:16
03 Room Air	0.000	13:16

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = SEQUENCE ABORT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: SEQUENCE ABORTED

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:16

Test	AC	Time
01 Room Air	0.000	13:17
02 Subject Test 1	0.098	13:18
03 Room Air	0.000	13:18
04 Reported AC	0.098	13:18

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = ABA MODE
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

ABA MODE TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:18

Test	AC	Time
01 Diagnostic	OK	13:19
02 Room Air	0.000	13:20
03 Subject Test 1	0.098	13:20
04 Room Air	0.000	13:22
05 Std. Gas	0.000*	13:23
06 Room Air	0.000	13:24

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CAL CHK OUT TOL
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:24

Test	AC	Time
01 Diagnostic	OK	13:25
02 Room Air	0.000	13:26
03 *Subject Test	RFI*	13:26
04 Room Air	0.000	13:26

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI DETECTED
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 21
Expiration Date = 09/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: RFI DETECTED

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:30

Test	AC	Time
01 Diagnostic	OK	13:31
02 Room Air	0.000	13:32
03 *Subject Test	0.000*	13:35
04 Room Air	0.000	13:37
05 Std. Gas	0.080	13:38
06 Room Air	0.000	13:40
07 *Subject Test	0.000*	13:43
08 Room Air	0.000	13:44


No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DEFICIENT SAMP
Dr. Lic. = ND/DIS821456
Lot No = 17618080A1
Cyl No = 28
Expiration Date = 08/05/2020
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

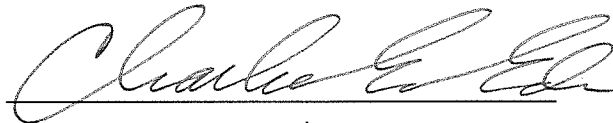

Operator Signature
CHARLES EDER

Remarks: DEFICIENT SAMPLES

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 13:44

Test	AC	Time
01 Diagnostic	OK	13:45
02 Room Air	0.000	13:46
03 Reference	IPS*	13:46
04 Room Air	0.000	13:46

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = IMPROPER SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 17618080A1
Cyl No = 28
Expiration Date = 08/05/2020
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *Improper Sample*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 14:03

Test	AC	Time
01 Diagnostic	OK	14:04
02 Room Air	0.000	14:04
03 Subject Test 1	0.249*	14:04
04 Room Air	0.000	14:07
05 Std. Gas	0.079	14:08
06 Room Air	0.000	14:09
07 Subject Test 2	0.096*	14:10
08 Room Air	0.000	14:11

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS821456
Lot No = 17618080A1
Cyl No = 28
Expiration Date = 08/05/2020
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

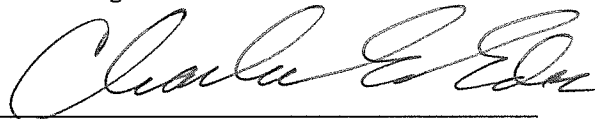

Operator Signature
CHARLES EDER

Remarks: *DIFFERENCE TOO GREAT*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 14:11


Test	AC	Time
01 Diagnostic	OK	14:12
02 Room Air	0.000	14:12
03 Subject Test 1	0.097	14:13
04 Room Air	0.000	14:15
05 Std. Gas	0.080	14:16
06 Room Air	0.000	14:18
07 Subject Test 2	0.096	14:18
08 Room Air	0.000	14:19
09 Reported AC	0.096	14:18

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CMS MODE
Dr. Lic. = ND/DIS821456
Lot No = 17618080A1
Cyl No = 28
Expiration Date = 08/05/2020
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

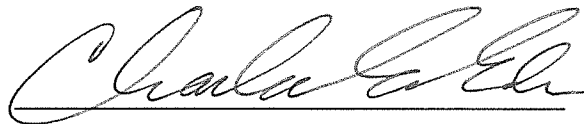

Operator Signature
CHARLES EDER

Remarks: *CMS MODE
TEST*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = TOXL 8164.16.00 09/18
10/22/2018 14:20

Test	AC	Time
01 Room Air	0.000	14:20
02 Std. Gas	0.080	14:21
03 Room Air	0.000	14:21
04 Std. Gas	0.080	14:21
05 Room Air	0.000	14:22
06 Std. Gas	0.080	14:22
07 Room Air	0.000	14:23
08 Std. Gas	0.081	14:23
09 Room Air	0.000	14:24
10 Std. Gas	0.081	14:24
11 Room Air	0.000	14:25
12 Std. Gas	0.081	14:25
13 Room Air	0.000	14:25
14 Std. Gas	0.080	14:26
15 Room Air	0.000	14:26
16 Std. Gas	0.080	14:27
17 Room Air	0.000	14:27
18 Std. Gas	0.081	14:28
19 Room Air	0.000	14:28
20 Std. Gas	0.081	14:28
21 Room Air	0.000	14:29
22 Std. Gas	0.080	14:29
23 Room Air	0.000	14:30
24 Std. Gas	0.081	14:30
25 Room Air	0.000	14:31
26 Std. Gas	0.081	14:31
27 Room Air	0.000	14:32
28 Std. Gas	0.081	14:32
29 Room Air	0.000	14:32
30 Std. Gas	0.082	14:33
31 Room Air	0.000	14:33
32 Std. Gas	0.081	14:34
33 Room Air	0.000	14:34
34 Std. Gas	0.081	14:34
35 Room Air	0.000	14:35
36 Std. Gas	0.081	14:35
37 Room Air	0.000	14:36
38 Std. Gas	0.081	14:36
39 Room Air	0.000	14:37
40 Std. Gas	0.081	14:37
41 Room Air	0.000	14:38
42 Cal Check Stats:		
43 Average	0.0808	
44 Std Dev	0.0006	
45 Rel Std Dev	0.69%	



Operator Signature
CHARLES EDER

Remarks: STABILITY
TEST

Form 106-I8000

0.080AC

Lot: 17618080A1

Exp: ~~5/8/2020~~ CEE
8/5/2020

CYL: 028