

**INTOXILYZER® 8000 INITIAL INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-007097

**A. GENERAL Setup and Checks:**

- Power On  
 5 min. countdown during initial warm-up  
 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:  
 Level 1 (Operator)     Level 2 (Field Inspector)     Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

**B. LEVEL 1 Setup and Checks:**

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):  
 Operator ID     ND License     R key: Refusal. Attach record.

**C. LEVEL 2 Setup and Checks:**

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):  
Inst Ser#: 80-007097 DSP Ver: 1362.14  
Core Ver: 8104.16 CPLD Ver: 7

**D. LEVEL 3 Setup and Checks:**

- DVM Monitor (Level 3,D,D): 3: 12645 s: 1 C: 47.00  
9: 13240 s: 1 B: 44 0  
(RFI indicator with keyed radio): 1
2. Function (F) Keys: :  F5 – Tone                       F9 - Fan  
 F6 – Green LED                       F10 – Pump  
 F7 – Red LED                       F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 939 CF: 1.07
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 189 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.5 I: 0.0  
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 596 psi P (gauge): 600 psi
7.  General Setup (Level 3,S,G)
8.  Configure Start Test (Level 3,S,T)
9.  Network (Level 3,S,N)

**E. TESTS (attach all test records):**

**Configure wet bath simulator standards before running tests (Level 1,S):**

1. Zero AC (Level 1,C):  
Sim ser #: DR3377 Lot #: milliQ Exp date: NA  
Known value: 0.000 AC  
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):  
Sim ser #: DR7111 Lot #: 17180 Exp date: 8 Jun 19  
Known value: 0.020 AC  
Results: 0.018 AC 0.018 AC 0.018 AC
3. High AC (Level 1,C):  
Sim ser #: DR3375 Lot #: 17350 Exp date: 11 Oct 19  
Known value: 0.300 AC  
Results: 0.295 AC 0.296 AC 0.295 AC

**Configure dry gas standard before running tests (Level 1,S):**

4. Ethanol + 0.05% Acetone (Level 1,B):  
Sim ser #: DR7089 Lot #: 1C55 Exp date: 15 May 19  
Known value: 0.10 AC + 0.05% Acetone  
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):  
Sim ser #: DR3846 Lot #: ACE1 Exp date: NA  
Known value: 0.05% Acetone  
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):  
Sim ser #: DR3455 Lot #: MET2 Exp date: NA  
Known value: 0.10% methanol  
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):  
Sim ser #: DR5189 Lot #: IPA2 Exp date: NA  
Known value: 0.10% Isopropanol  
 Display reads "Interferent Detect"

- 8.  Ambient Fail (Level 1,B): Display and test record read correct.
- 9.  Invalid Sample (Level 1,B): Display and test record read correct.
- 10.  Sequence Aborted (Level 1,B): Display and test record read correct.
- 11.  ABA (Level 1,B): Test record reads correct.
- 12.  Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13.  RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15.  Deficient Samples (CMS mode): Display and test record read correct.
- 16.  Improper Sample (CMS mode): Display and test record read correct.
- 17.  Difference Too Great (CMS mode): Display and test record read correct.
- 18.  CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0802 Std Dev: 0.0004 Rel Std Dev: 0.52%

Maintenance/Repairs/Notes: Initial inspection after receiving  
instrument new from CMI, Inc.  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

Instrument is acceptable to be used in the field.

Roberta Huenig-Rimmo  
Field Inspector's Signature

26 Nov 18  
Date

Charles E. Eden  
Reviewed by

28 Nov. 2018  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-007097  
Location = TOXL 8164.16.00 09/18  
11/23/2018 14:59

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMNQRSTU

24 characters per line  
abcdefghijklmnopqrstuvw  
ABCDEFGHIJKLMNQRSTU

32 characters per line  
abcdefghijklmnopqrstvwxyz123456  
ABCDEFGHIJKLMNQRSTUWXYZ123456

42 characters per line  
abcdefghijklmnopqrstvwxyz1234567890abcdef  
ABCDEFGHIJKLMNQRSTUWXYZ1234567890ABCDEF

Current Instrument Setup

Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On MemFull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 03417080A3  
Standard Cyl #? 7  
Standard Expiration? 03/05/2019  
Oper No? 888888

Flow Cal. Date: 09/27/2018  
Slope 671  
Intercept -612394

IR Calibration Date: 09/27/2018

	3um	9um
--	-----	-----

0th Coef(*100):	-31035	-29197
1st Coef(*100):	248456	136978
2nd Coef(*100):	2388	1456
H2O adj(mg/l*10k):	964	610

\*\*\*\*\* Test End \*\*\*\*\*

*Roberta Grieger-Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:

Internal Printer Test

Form 106-18000

Intoxilyzer Test Record and Checklist  
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
 North Dakota Model 8000            SN 80-007097  
 Location = TOXL                    8164.16.00 09/18  
 11/23/2018                            14:57

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&\*()\_+?

Current Instrument Setup

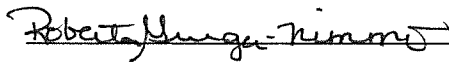
Data Entry Mode:            Enabled  
 Start Test Sequence:        DABACABA  
 Display Prelim Rslt?        Yes  
 Display Third Digit?        Yes  
 Inhib Printer(Y/N)?        No  
 Display Volume?            No  
 Disable On Memfull?        Yes  
 # of Print Copies?          1  
 Select Std (D/W/I)?        Dry  
 Standard Value?            0.080  
 Standard Lot #?            03417080A3  
 Standard Cyl #?            7  
 Standard Expiration?       03/05/2019  
 Oper No?                    888888

Flow Cal. Date:            09/27/2018  
 Slope                      671  
 Intercept                  -612394

IR Calibration Date:        09/27/2018  
                               3um                            9um

	3um	9um
0th Coef(*100):	-31035	-29197
1st Coef(*100):	248456	136978
2nd Coef(*100):	2388	1456
H2O adj(mg/l*10k):	964	610

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
 Operator Signature  
 ROBERTA GRIEGER-NIMMO

Remarks: External Print Test

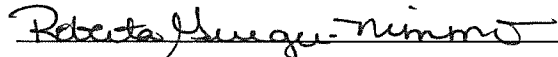
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = TOXL      8164.16.00 09/18  
11/23/2018      12:26

Test	AC	Time
01 Room Air	0.000	12:26
02 *Subject Test	REF*	12:26
03 Room Air	0.000	12:27

\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = BARCODE RDR  
Dr. Lic. = ND/DIS821456  
Lot No = 80  
Cyl No = 11  
Expiration Date = 11/11/2019  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Barcode Reader*

Form 106-I8000

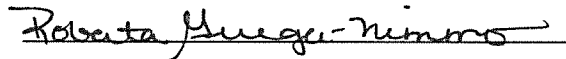
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-007097  
Location = TOXL                8164.16.00 09/18  
11/23/2018                       12:28

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 888888



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Diagnostic*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

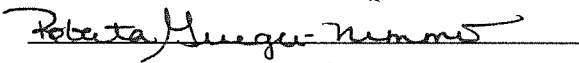
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = TOXL      8164.16.00 09/18  
11/23/2018      14:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Sol.	0.000	14:30
03 Room Air	0.000	14:30
04 Std. Sol.	0.000	14:31
05 Room Air	0.000	14:31
06 Std. Sol.	0.000	14:32
07 Room Air	0.000	14:32

08 Sim Temp = 34.0°C

Simul Ser No = DR3377  
Std Sol No = MILLIQ  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Zero AC*  
*0.000AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = TOXL      8164.16.00 09/18  
11/23/2018      14:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:23
02 Std. Sol.	0.018	14:24
03 Room Air	0.000	14:25
04 Std. Sol.	0.018	14:26
05 Room Air	0.000	14:26
06 Std. Sol.	0.018	14:27
07 Room Air	0.000	14:27

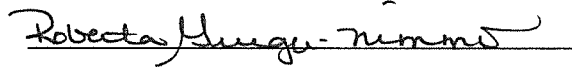
08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17180

County = 08

Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

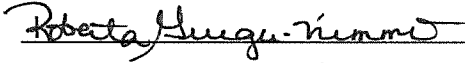
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = TOXL      8164.16.00 09/18  
11/23/2018      14:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:40
02 Std. Sol.	0.295	14:41
03 Room Air	0.000	14:42
04 Std. Sol.	0.296	14:42
05 Room Air	0.000	14:43
06 Std. Sol.	0.295	14:44
07 Room Air	0.000	14:44

08 Sim Temp = 34.0°C

Simul Ser No = DR3375  
Std Sol No = 17350  
County = 08      Oper No. = 888888

  
\_\_\_\_\_  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.300AC

Form 106-I8000

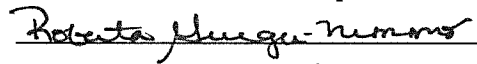
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = TOXL      8164.16.00 09/18  
11/23/2018      14:46

Test	AC	Time
01 Room Air	0.000	14:47
02 *Subject Test	INT*	14:48
03 Room Air	0.000	14:48

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08      Oper No. = 888888

  
\_\_\_\_\_  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: 0.10 AC+0.059 Acetone

Form 106-I8000

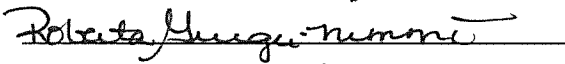
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL            8164.16.00 09/18  
11/23/2018            14:49

Test	AC	Time
01 Room Air	0.000	14:50
02 *Subject Test	INT*	14:51
03 Room Air	0.000	14:51

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female            Weight = 150  
Test = DUI            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08            Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: 0.059% Acetone

Form 106-I8000

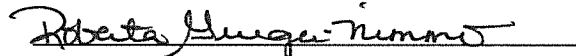
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = TOXL      8164.16.00 09/18  
11/23/2018      14:52

Test	AC	Time
01 Room Air	0.000	14:52
02 *Subject Test	INT*	14:53
03 Room Air	0.000	14:54

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08      Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:  
0.10% methanol

Form 106-I8000

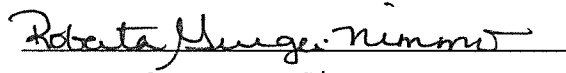
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                      8164.16.00 09/18  
11/23/2018                              14:55

Test	AC	Time
01 Room Air	0.000	14:55
02 *Subject Test	INT*	14:56
03 Room Air	0.000	14:57

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                              Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: 0.10% Isopropanol

Form 106-I8000

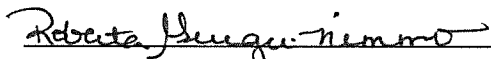
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/23/2018                            15:04

Test	AC	Time
01 Room Air	AMB*	15:05
02 Room Air	0.000	15:06

\*Invalid Test  
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Ambient Fail

Form 106-I8000

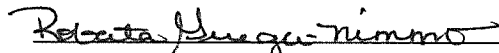
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/23/2018                            15:07

Test	AC	Time
01 Room Air	0.000	15:08
02 Invalid Sample	X.XXX	15:08
03 Room Air	0.000	15:09

\*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Invalid Sample

Form 106-I8000



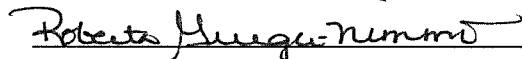
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/23/2018                            15:09

Test	AC	Time
01 Room Air	0.000	15:10
02 *Subject Test	ABT*	15:10
03 Room Air	0.000	15:10

\*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = SEQ ABT  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 888888

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Sequence Aborted*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                    Alcohol Analyzer  
North Dakota Model 8000                SN 80-007097  
Location = TOXL                            8164.16.00 09/18  
11/26/2018                                    08:19

Test	AC	Time
01 Room Air	0.000	08:20
02 Subject Test 1	0.000	08:21
03 Room Air	0.000	08:21
04 Reported AC	0.000	08:21

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                            Weight = 150  
Test = DUI                                    Cit = ABA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                                 Oper No. = 888888



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *ABA*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

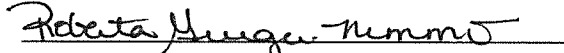
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/26/2018                            08:22

Test	AC	Time
01 Diagnostic	OK	08:22
02 Room Air	0.000	08:23
03 Subject Test 1	0.000	08:24
04 Room Air	0.000	08:27
05 Std. Gas	0.090*	08:28
06 Room Air	0.000	08:29

\*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = CAL OUT TOL  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check out of  
Tolerance

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

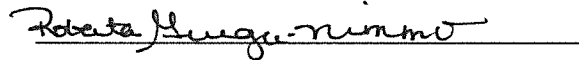
CMI, Inc. Intoxilyzer                 Alcohol Analyzer  
North Dakota Model 8000                 SN 80-007097  
Location = TOXL                         8164.16.00 09/18  
11/26/2018                                 08:29

Test	AC	Time
01 Diagnostic	OK	08:30
02 Room Air	0.000	08:30
03 *Subject Test	RFI*	08:30
04 Room Air	0.000	08:31

\*Invalid Test  
   Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                         Weight = 150  
Test = DUI                                 Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                                 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *RFI detected*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/23/2018                            15:17

Test	AC	Time
01 Diagnostic	OK	15:18
02 Room Air	0.000	15:18
03 *Subject Test	0.000*	15:22
04 Room Air	0.000	15:24
05 Std. Gas	0.079	15:25
06 Room Air	0.000	15:26
07 *Subject Test	0.000*	15:29
08 Room Air	0.000	15:30

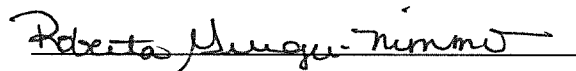
No RFI Detected

\*Deficient Sample - Value Printed was  
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = DEFICIENT  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 888888

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Deficient Samples

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

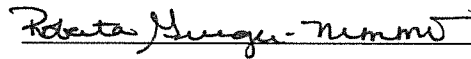
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/26/2018                            08:31

Test	AC	Time
01 Diagnostic	OK	08:32
02 Room Air	IPS*	08:32
03 Room Air	0.000	08:33

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                            Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Improper Sample

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

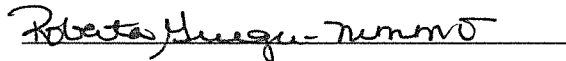
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/26/2018                            08:34

Test	AC	Time
01 Diagnostic	OK	08:34
02 Room Air	0.000	08:35
03 Subject Test 1	0.246*	08:35
04 Room Air	0.000	08:38
05 Std. Gas	0.082	08:39
06 Room Air	0.000	08:40
07 Subject Test 2	0.304*	08:41
08 Room Air	0.000	08:42

\*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                          Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Difference too Great*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007097  
Location = TOXL                    8164.16.00 09/18  
11/26/2018                            08:42

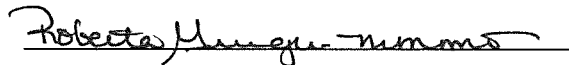
Test	AC	Time
01 Diagnostic	OK	08:43
02 Room Air	0.000	08:44
03 Subject Test 1	0.000	08:44
04 Room Air	0.000	08:46
05 Std. Gas	0.081	08:48
06 Room Air	0.000	08:49
07 Subject Test 2	0.000	08:49
08 Room Air	0.000	08:50
09 Reported AC	0.000	08:44

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 03417080A3  
Cyl No = 7  
Expiration Date = 03/05/2019  
County = 08                    Oper No. = 888888

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: CMS Test

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = TOXL      8164.16.00 09/18  
11/23/2018      15:31

Test	AC	Time
01 Room Air	0.000	15:31
02 Std. Gas	0.080	15:31
03 Room Air	0.000	15:32
04 Std. Gas	0.080	15:32
05 Room Air	0.000	15:33
06 Std. Gas	0.081	15:33
07 Room Air	0.000	15:34
08 Std. Gas	0.080	15:34
09 Room Air	0.000	15:35
10 Std. Gas	0.080	15:35
11 Room Air	0.000	15:35
12 Std. Gas	0.080	15:36
13 Room Air	0.000	15:36
14 Std. Gas	0.080	15:37
15 Room Air	0.000	15:37
16 Std. Gas	0.081	15:38
17 Room Air	0.000	15:38
18 Std. Gas	0.080	15:38
19 Room Air	0.000	15:39
20 Std. Gas	0.080	15:39
21 Room Air	0.000	15:40
22 Std. Gas	0.080	15:40
23 Room Air	0.000	15:41
24 Std. Gas	0.080	15:41
25 Room Air	0.000	15:42
26 Std. Gas	0.080	15:42
27 Room Air	0.000	15:42
28 Std. Gas	0.080	15:43
29 Room Air	0.000	15:43
30 Std. Gas	0.080	15:44
31 Room Air	0.000	15:44
32 Std. Gas	0.080	15:45
33 Room Air	0.000	15:45
34 Std. Gas	0.080	15:45
35 Room Air	0.000	15:46
36 Std. Gas	0.081	15:46
37 Room Air	0.000	15:47
38 Std. Gas	0.080	15:47
39 Room Air	0.000	15:48
40 Std. Gas	0.081	15:48
41 Room Air	0.000	15:49
42 Cal Check Stats:		
43 Average	0.0802	
44 Std Dev	0.0004	
45 Rel Std Dev	0.52%	

*Roberta Hugué Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Stability Test  
0.080A2  
Lot 93417080A3  
Cyl 7  
Exp 5 Mar 19

Form 106-I8000