



# ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Allen, Kathryn M.</i>		#090801
Location <i>Cass County Jail</i>		Intoxilyzer® Serial Number <i>80-007097</i>
Gas Lot Number <i>13518080A6</i>	Gas Cylinder Number <i>14</i>	Gas Expiration Date <i>8-5-2020</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0.*082* AC
  - B. 0.*083* AC
  - C. 0.*082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>12-28-2018</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>12-31-2018</i>


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007097  
Location = CASS      8164.16.00 09/18  
12/28/2018      21:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:09
02 Std. Gas	0.082	21:10
03 Room Air	0.000	21:10
04 Std. Gas	0.083	21:11
05 Room Air	0.000	21:11
06 Std. Gas	0.082	21:12
07 Room Air	0.000	21:12

Lot No = 13518080A6  
Cyl No = 14  
Exp Date = 08/05/2020  
County = 09      Oper No. = 090801

  
\_\_\_\_\_  
Operator Signature  
KATHRYN ALLEN

Remarks: