



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-007097	Instrument Location Cass County Jail
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (If any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 12-29-2018
-------------------------------	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date 12-31-2018
---------------------------	-------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = CASS 8164.16.00 09/18
12/28/2018 21:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:16
02 Std. Gas	0.083	21:16
03 Room Air	0.000	21:17
04 Std. Gas	0.083	21:17
05 Room Air	0.000	21:18
06 Std. Gas	0.083	21:18
07 Room Air	0.000	21:19

Lot No = 13518080A6
Cyl No = 14
Exp Date = 08/05/2020
County = 09 Oper No. = 090801



Operator Signature
KATHRYN ALLEN

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = CASS 8164.16.00 09/18
12/28/2018 21:21

Test	AC	Time
01 Room Air	0.000	21:22
02 *Subject Test	RFI*	21:22
03 Room Air	0.000	21:23

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST NEW MACHINE
Sub DOB = 01/01/1981
Sub Sex = Female Weight = NA
Test = DUI Cit = TESTING
Dr. Lic. = ND/TESTING
Lot No = 13518080A6
Cyl No = 14
Expiration Date = 08/05/2020
County = 09 Oper No. = 090801



Operator Signature
KATHRYN ALLEN

Remarks:

Form 106-I8000