

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.0 I: 0.0
B: 44.9 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 636 psi P (gauge): 650 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: DR3841 Lot #: MILLI Q H₂O Exp date: NA
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: DR3378 Lot #: 201712B Exp date: 12.12.19
Known value: 0.020 AC
Results: 0.020 AC 0.020 AC 0.019 AC
3. High AC (Level 1,C):
Sim ser #: DR7351 Lot #: 201803G Exp date: 3.22.20
Known value: 0.250 AC
Results: 0.250 AC 0.249 AC 0.251 AC
CEE

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR3451 Lot #: ICS 5 Exp date: NA
Known value: 0.10% AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR5188 Lot #: ACE 1 Exp date: NA
Known value: 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR3455 Lot #: MET 2 Exp date: NA
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR5189 Lot #: IPA 2 Exp date: NA
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0796 Std Dev: 0.0006 Rel Std Dev: 0.76%

Maintenance/Repairs/Notes: INSPECTION AFTER RECEIVING
FROM CMI INC.

Instrument is acceptable to be used in the field.


Field Inspector's Signature

6.7.2019
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:00

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

24 characters per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

32 characters per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

42 characters per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Menfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34418080A2
Standard Cyl #? 45
Standard Expiration? 02/05/2021
Oper No? 666666

Flow Cal. Date: 06/07/2019
Slope 638
Intercept -611758

IR Calibration Date: 06/07/2019
3um 9um

0th Coef(*100):	-9257	-21772
1st Coef(*100):	261014	138791
2nd Coef(*100):	3804	1556
H2O adj(mg/l*10k):	438	640

***** Test End *****

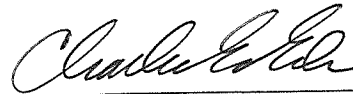

Operator Signature
CHARLES EDER

Remarks: *INTERNAL
PRINT TEST*

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:03

Test	AC	Time
01 Room Air	0.000	15:04
02 *Subject Test	REF*	15:04
03 Room Air	0.000	15:05

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = R KEY REFUSAL
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: R KEY REFUSAL

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:05

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

DIAGNOSTIC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:13
02 Std. Sol.	0.000	15:14
03 Room Air	0.000	15:14
04 Std. Sol.	0.000	15:15
05 Room Air	0.000	15:15
06 Std. Sol.	0.000	15:16
07 Room Air	0.000	15:16

08 Sim Temp = 34.0°C

Simul Ser No = DR3841
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

ZERO AC
0.000 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:18

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:18
02 Std. Sol.	0.020	15:19
03 Room Air	0.000	15:19
04 Std. Sol.	0.020	15:20
05 Room Air	0.000	15:21
06 Std. Sol.	0.019	15:21
07 Room Air	0.000	15:22

08 Sim Temp = 34.0°C

Simul Ser No = DR3378
Std Sol No = 201712B
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:24
02 Std. Sol.	0.249	15:25
03 Room Air	0.000	15:25
04 Std. Sol.	0.249	15:26
05 Room Air	0.000	15:27
06 Std. Sol.	0.251	15:27
07 Room Air	0.000	15:28

08 Sim Temp = 34.0°C

Simul Ser No = DR7351
Std Sol No = 201803G
County = 08 Oper No. = 666666

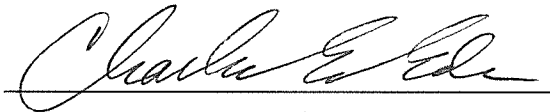

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.250 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:37

Test	AC	Time
01 Room Air	0.000	15:38
02 *Subject Test	INT*	15:38
03 Room Air	0.000	15:39

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:39

Test	AC	Time
01 Room Air	0.000	15:40
02 *Subject Test	INT*	15:40
03 Room Air	0.000	15:41

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

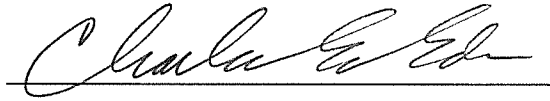

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:42

Test	AC	Time
01 Room Air	0.000	15:42
02 *Subject Test	INT*	15:43
03 Room Air	0.000	15:43

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10% METHANOL

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:44

Test	AC	Time
01 Room Air	0.000	15:44
02 *Subject Test	INT*	15:45
03 Room Air	0.000	15:45

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10% ISOPROPANOL

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:46

Test	AC	Time
01 Room Air	AMB*	15:47
02 Room Air	0.000	15:48

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *AMBIENT FAIL*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:48

Test	AC	Time
01 Room Air	0.000	15:49
02 Invalid Sample	X.XXX	15:49
03 Room Air	0.000	15:50

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INVALID SAMPLE
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INVALID SAMPLE*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:50

Test	AC	Time
01 Room Air	0.000	15:51
02 *Subject Test	ABT*	15:51
03 Room Air	0.000	15:52

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = SEQUENCE ABORT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

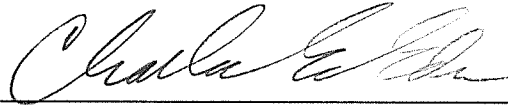

Operator Signature
CHARLES EDER

Remarks: SEQUENCE ABORTED

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:52

Test	AC	Time
01 Room Air	0.000	15:53
02 Subject Test 1	0.027	15:53
03 Room Air	0.000	15:54
04 Reported AC	0.027	15:53

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = ABA MODE
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *ABA MODE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 15:54

Test	AC	Time
01 Diagnostic	OK	15:55
02 Room Air	0.000	15:55
03 Subject Test 1	0.014	15:56
04 Room Air	0.000	15:58
05 Std. Gas	0.090*	15:59
06 Room Air	0.000	15:59

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = CAL CHK OUT TOL
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 16:00

Test	AC	Time
01 Diagnostic	OK	16:01
02 Room Air	0.000	16:01
03 *Subject Test	RFI*	16:01
04 Room Air	0.000	16:02

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 16:02

Test	AC	Time
01 Diagnostic	OK	16:03
02 Room Air	0.000	16:03
03 *Subject Test	0.000*	16:06
04 Room Air	0.000	16:09
05 Std. Gas	0.082	16:10
06 Room Air	0.000	16:11
07 *Subject Test	0.000*	16:14
08 Room Air	0.000	16:15

No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = DEFICIENT SAMP
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DEFICIENT SAMPLES

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 16:15

Test	AC	Time
01 Diagnostic	OK	16:16
02 Room Air	0.000	16:16
03 Reference	IPS*	16:16
04 Room Air	0.000	16:17

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = IMPROPER SAMPLE
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

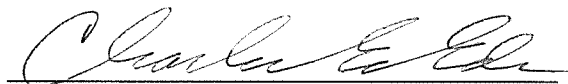

Operator Signature
CHARLES EDER

Remarks:

IMPROPER SAMPLE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 16:17

Test	AC	Time
01 Diagnostic	OK	16:18
02 Room Air	0.000	16:18
03 Subject Test 1	0.101*	16:19
04 Room Air	0.000	16:21
05 Std. Gas	0.083	16:22
06 Room Air	0.000	16:24
07 Subject Test 2	0.148*	16:24
08 Room Air	0.000	16:25

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *DIFFERENCE TOO GREAT*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL, 8164.16.00 09/18
06/07/2019 16:26

Test	AC	Time
01 Diagnostic	OK	16:26
02 Room Air	0.000	16:27
03 Subject Test 1	0.100	16:27
04 Room Air	0.000	16:30
05 Std. Gas	0.081	16:31
06 Room Air	0.000	16:32
07 Subject Test 2	0.098	16:33
08 Room Air	0.000	16:34
09 Reported AC	0.098	16:33

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = CMS MODE
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:


CMS MODE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = TOXL 8164.16.00 09/18
06/07/2019 16:34

Test	AC	Time
01 Room Air	0.000	16:35
02 Std. Gas	0.081	16:35
03 Room Air	0.000	16:36
04 Std. Gas	0.079	16:36
05 Room Air	0.000	16:36
06 Std. Gas	0.079	16:37
07 Room Air	0.000	16:37
08 Std. Gas	0.079	16:38
09 Room Air	0.000	16:38
10 Std. Gas	0.079	16:39
11 Room Air	0.000	16:39
12 Std. Gas	0.080	16:39
13 Room Air	0.000	16:40
14 Std. Gas	0.080	16:40
15 Room Air	0.000	16:41
16 Std. Gas	0.080	16:41
17 Room Air	0.000	16:42
18 Std. Gas	0.079	16:42
19 Room Air	0.000	16:43
20 Std. Gas	0.080	16:43
21 Room Air	0.000	16:44
22 Std. Gas	0.079	16:44
23 Room Air	0.000	16:44
24 Std. Gas	0.080	16:45
25 Room Air	0.000	16:45
26 Std. Gas	0.080	16:46
27 Room Air	0.000	16:46
28 Std. Gas	0.080	16:47
29 Room Air	0.000	16:47
30 Std. Gas	0.079	16:47
31 Room Air	0.000	16:48
32 Std. Gas	0.080	16:48
33 Room Air	0.000	16:49
34 Std. Gas	0.080	16:49
35 Room Air	0.000	16:50
36 Std. Gas	0.080	16:50
37 Room Air	0.000	16:51
38 Std. Gas	0.079	16:51
39 Room Air	0.000	16:51
40 Std. Gas	0.079	16:52
41 Room Air	0.000	16:52
42 Cal Check Stats:		
43 Average	0.0796	
44 Std Dev	0.0006	
45 Rel Std Dev	0.76%	

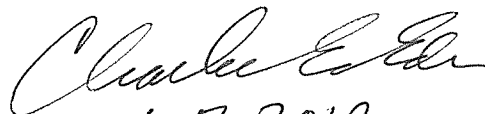
6.7.2019

Operator Signature
CHARLES EDER

Remarks: STABILITY TEST
0.080 AC

Form 106-I8000

LOT: 34418080A2

CYL: 45

Exp: 2/5/21