



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-007093	Instrument Location Rolette County Sheriff Department
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Jeffrey Bauske</i>	Date 8/19/2019
--	-------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Charles E. Edr</i>	Certified Date 8.26.19
--	---------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = ROLL 8164.16.00 09/18
08/19/2019 11:24

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13518080A6
Standard Cyl #? 033
Standard Expiration? 08/05/2020
Oper No? 120501

Flow Cal. Date: 06/07/2019
Slope 638
Intercept -611758

IR Calibration Date: 06/07/2019

	3um	9um
0th Coef(*100):	-9257	-21772
1st Coef(*100):	261014	138791
2nd Coef(*100):	3804	1556
H2O adj(mg/l*10k):	438	640

***** Printer Test End *****



Operator Signature
JEFFREY BAUSKE

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = ROLL 8164.16.00 09/18
08/19/2019 11:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:26
02 Std. Gas	0.082	11:26
03 Room Air	0.000	11:27
04 Std. Gas	0.082	11:27
05 Room Air	0.000	11:28
06 Std. Gas	0.081	11:28
07 Room Air	0.000	11:28

Lot No = 13518080A6
Cyl No = 033
Exp Date = 08/05/2020
County = 40 Oper No. = 120501

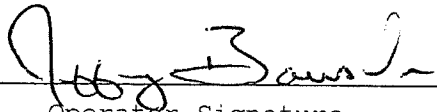

Operator Signature
JEFFREY BAUSKE

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = ROLL 8164.16.00 09/18
08/19/2019 11:30

Test	AC	Time
01 Room Air	0.000	11:33
02 *Subject Test	RFI*	11:33
03 Room Air	RFI*	11:33

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 01/01/2000
Sub Sex = Male Weight = 225
Test = DUI Cit = 123456789
Dr. Lic. = ND/TES001234
Lot No = 13518080A6
Cyl No = 033
Expiration Date = 08/05/2020
County = 40 Oper No. = 120501

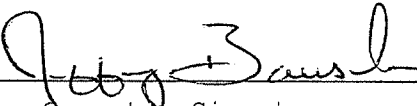
Operator Signature
JEFFREY BAUSKE

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = ROLL 8164.16.00 09/18
08/19/2019 11:50

Test	AC	Time
01 Room Air	0.000	11:51
02 *Subject Test	RFI*	11:51
03 Room Air	0.000	11:52

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 01/01/2000
Sub Sex = Male Weight = 225
Test = DUI Cit = RFI TEST
Dr. Lic. = ND/TES001234
Lot No = 13518080A6
Cyl No = 033
Expiration Date = 08/05/2020
County = 40 Oper No. = 120501


Operator Signature
JEFFREY BAUSKE

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007093
Location = ROLL 8164.16.00 09/18
08/19/2019 11:52

Test	AC	Time
01 Room Air	0.000	11:53
02 *Subject Test	RFI*	11:54
03 Room Air	0.000	11:54

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 01/01/2000
Sub Sex = Male Weight = 225
Test = DUI Cit = RFI TEST
Dr. Lic. = ND/TES001234
Lot No = 13518080A6
Cyl No = 033
Expiration Date = 08/05/2020
County = 40 Oper No. = 120501



Operator Signature
JEFFREY BAUSKE

Remarks:

Form 106-I8000