



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59281 (06/2018)

Serial Number 80-007098	Instrument Location WAPD
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): NONE
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 4-12-19
-------------------------------	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date 9.16.19
---------------------------	----------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007098
Location = NAPO 8164.16.00 09/18
09/12/2019 13:42

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13518080A6
Standard Cyl #? 024
Standard Expiration? 08/05/2020
Oper No? 131100

Flow Cal. Date: 09/26/2018
Slope 684
Intercept -588446

IR Calibration Date: 09/26/2018

	3um	9um
0th Coef(*100):	-40948	-15551
1st Coef(*100):	237555	134140
2nd Coef(*100):	2571	1433
H2O adj(mg/l*10k):	1097	457

***** Printer Test End *****



Operator Signature
MICHAEL MAROLT

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007098
Location = NAPO 8164.16.00 09/18
09/12/2019 13:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:52
02 Std. Gas	0.080	13:52
03 Room Air	0.000	13:53
04 Std. Gas	0.080	13:53
05 Room Air	0.000	13:54
06 Std. Gas	0.080	13:54
07 Room Air	0.000	13:55

Lot No = 13518080A6
Cyl No = 024
Exp Date = 08/05/2020
County = 24 Oper No. = 131100

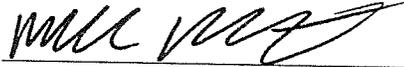

Operator Signature
MICHAEL MAROLT

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007098
Location = NAPO 8164.16.00 09/18
09/12/2019 13:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	13:57
02 Room Air	RFI*	13:57

*Invalid Test
Inhibited - RFI

Lot No = 13518080A6
Cyl No = 024
Exp Date = 08/05/2020
County = 24 Oper No. = 131100

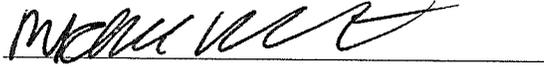

Operator Signature
MICHAEL MAROLT

Remarks:

Form 106-I8000