



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Allen, Kathryn Marie</i>		
Location <i>Cass County Jail</i>		Intoxilyzer® Serial Number <i>80-007097</i>
Gas Lot Number <i>17618080A3</i>	Gas Cylinder Number <i>7a</i>	Gas Expiration Date <i>8-5-2020</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0. *080* AC
 B. 0. *080* AC
 C. 0. *080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i> #3687	Date <i>11-20-19</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>11-20-2019</i>

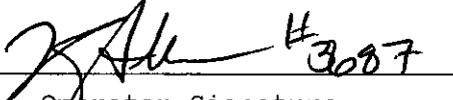
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = CASS 8164.16.00 09/18
11/20/2019 02:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	02:57
02 Std. Gas	0.080	02:57
03 Room Air	0.000	02:58
04 Std. Gas	0.080	02:58
05 Room Air	0.000	02:58
06 Std. Gas	0.080	02:59
07 Room Air	0.000	02:59

Lot No = 17618080A3
Cyl No = 72
Exp Date = 08/05/2020
County = 09 Oper No. = 090801


Operator Signature
KATHRYN ALLEN

Remarks:

Form 106-I8000