

ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59282 (08/2018)

Chemical Test Operator Name (Print) Travis L. Collins			
Location		Intoxilyzer® Serial Number	
HETT		80-007095	
Gas Lot Number	Gas Cylinder Number		Gas Expiration Date
24119080A1	25		11/05/2021

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

- A. 0. 0 1 AC
- B. 0.08 AC
- C. 0.080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 🗵 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ★ 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature	I Collind	Date 5/13/2020
Reviewed By (Crime Laboratory Use Only)	Charla Exter	Date 5/14/2020

FORM 105-G

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007095 Location = HETT 8164.16.00 09/18 05/13/2020

17:02

DRY CAL CHECK

Test			AC	Time	
01	Room	Air	0.000	17:03	
02	Std.	Gas	0.081	17:03	
03	Room	Air	0.000	17:04	
04	Std.	Gas	0.081	17:04	
05	Room	Air	0.000	17:05	
06	Std.	Gas	0.080	17:05	
07	Room	Air	0.000	17:06	

Lot. No = 24119080A1

Cyl No = 25

Exp Date = 11/05/2021

County = 01

Oper No. = 020533

Operator Signature TRAVIS COLLINS

Remarks:

Form 106-I8000