

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-007092

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 985 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.250 AC
Sim. Ser #: MP3067 Lot #: 201911B Exp. Date: 11.5.21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS 6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 24119080A1 Cyl #: 9 Exp. Date: 11.5.21

Test 1	<u>0.080</u>	Test 4	<u>0.081</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.080</u>	Test 9	<u>0.081</u>
Average <u>0.080</u>					

C. Remarks/Maintenance:

ANNUAL INSPECTION
REPLACED GAS REGULATOR - GAS CYLINDER WAS READING 0PSE.
POSSIBLE LEAK. CALIBRATED TANK SENSOR.
SPEAKER VOLUME LOW BUT AUDIBLE.

Instrument is acceptable to be used in the field.

[Signature]
Breath Analyst Signature

6.8.20
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = TOXL 8164.16.00 09/18
06/08/2020 14:55

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:56
02 Std. Sol.	0.018	14:57
03 Room Air	0.000	14:57
04 Std. Sol.	0.019	14:58
05 Room Air	0.000	14:58
06 Std. Sol.	0.019	14:59
07 Room Air	0.000	15:00

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = 20070
County = 08 Oper No. = 666666

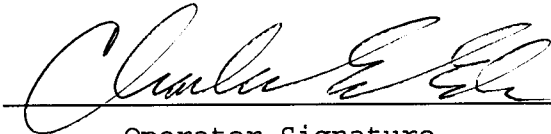

Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = TOXL 8164.16.00 09/18
06/08/2020 15:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:01
02 Std. Sol.	0.250	15:02
03 Room Air	0.000	15:02
04 Std. Sol.	0.251	15:03
05 Room Air	0.000	15:04
06 Std. Sol.	0.251	15:04
07 Room Air	0.000	15:05

08 Sim Temp = 34.0°C

Simul Ser No = MP3067
Std Sol No = 201911B
County = 08 Oper No. = 666666

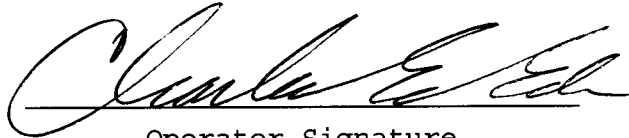

Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.250 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = TOXL 8164.16.00 09/18
06/08/2020 15:06

Test	AC	Time
01 Room Air	0.000	15:07
02 *Subject Test	INT*	15:07
03 Room Air	0.000	15:08

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 9
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

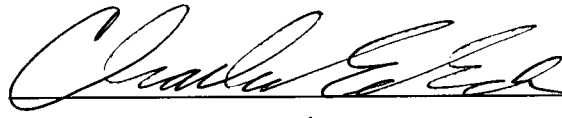

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = TOXL 8164.16.00 09/18
06/08/2020 15:08

Test	AC	Time
01 Diagnostic	OK	15:09
02 Room Air	0.000	15:09
03 *Subject Test	RFI*	15:10
04 Room Air	0.000	15:10

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 9
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

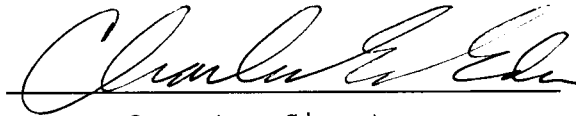

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = TOXL 8164.16.00 09/18
06/08/2020 15:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:11
02 Std. Gas	0.080	15:11
03 Room Air	0.000	15:12
04 Std. Gas	0.080	15:12
05 Room Air	0.000	15:13
06 Std. Gas	0.080	15:13
07 Room Air	0.000	15:13

Lot No = 24119080A1
Cyl No = 9
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = TOXL 8164.16.00 09/18
06/08/2020 15:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:15
02 Std. Gas	0.081	15:15
03 Room Air	0.000	15:15
04 Std. Gas	0.080	15:16
05 Room Air	0.000	15:16
06 Std. Gas	0.080	15:17
07 Room Air	0.000	15:17

Lot No = 24119080A1
Cyl No = 9
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = TOXL 8164.16.00 09/18
06/08/2020 15:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:18
02 Std. Gas	0.080	15:18
03 Room Air	0.000	15:19
04 Std. Gas	0.080	15:19
05 Room Air	0.000	15:20
06 Std. Gas	0.081	15:20
07 Room Air	0.000	15:21

Lot No = 24119080A1
Cyl No = 9
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000