

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number QQ = Q Instrument Location	27
Reason for Install/Repair	a ID
☐ Other (Specify)	
Check When Done:	
1. Surge Protector Installed/Property Grounded.	
2. Telephone Line Connected to Intoxilyzer® 8000.	
3. Breath Tube Heated.	
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).	
5. Scan/Enter Gas Cylinder Information (Level 1, Function S).	
8. Run Tests:	
A. Print Test (Level1, Function P).	
B. ACA Test (Level 1, Function C).	
C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radi	io During Test).
7. Repair and/or Maintenance Performed (if any):	
8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by	the Intoxilyzer® for Use.
9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the	e Agency.
10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Rep 104-G), Print Test, ACA Test, and RFI Test.	air Checkout (SFN59281, Form
FieldInspector Signature	Date
I here	06/10/2020
Crime Laboratory Use Only	
This installation has been reviewed and the instrument is approved to be used for the analysis of breath to other the Field Inspector performed the installation. This record on file at the Office of Attorney General, County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.	determine alcohol concentration from Crime Laboratory Division, in the
Reviewed/Certified By	Certified Date 6 • 12 • 2020

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007098
Location = TIOG 8164.16.00 09/18
06/10/2020 14:20

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? Standard Cyl #? 24119080A1 Standard Expiration? 11/05/2021 130977 Oper No?

Flow Cal. Date: 09/26/2018 Slope 684 Intercept -588446

IR Calibration Date: 09/26/2018
3um 9um
Oth Coef(*100): -40948 -15551

Oth Coef(*100): -40948 -15551 1st Coef(*100): 237555 134140 2nd Coef(*100): 2571 1433 H2O adj(mg/l*10k): 1097 457

******* Printer Test End *******

Operator Signature JOSHUA NELSON

Remarks:

I certify this is a true and accurate copy.

Form 106-I8000

from hele

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007098
Location = TIOG 8164.16.00 09/18
06/10/2020 14:26

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? Standard Cyl #? 24119080A1 Standard Expiration? 11/05/2021 130977 Oper No?

Flow Cal. Date: 09/26/2018 Slope 684 Intercept -588446

 0th Coef(*100):
 -40948
 -15551

 1st Coef(*100):
 237555
 134140

 2nd Coef(*100):
 2571
 1433

 H2O adj(mg/l*10k):
 1097
 457

****** Printer Test End *******

Operator Signature
JOSHUA NELSON

Remarks:

I certify this is a true and accurate copy.

Form 106-I8000

SN 80-007098

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-007098 Location = TIOG 8164.16.00 09/18 06/10/2020

14:34

DRY CAL CHECK

Test			AC	Time
	Room		0.000	14:35
02	std.	Gas	0.081	14:35
03	Room	Air	0.000	14:36
04	Std.	Gas	0.080	14:36
05	Room	Air	0.000	14:37
06	Std.	Gas	0.082	14:37
07	Room	Air	0.000	14:38

Lot No = 24119080A1

Cyl No = 2

Exp Date = 11/05/2021

County = 53

Oper No. = 130977

Operator Signature JOSHUA NELSON

Remarks:

Form 106-I8000

I certify this is a true and accurate copy.

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 Location = TIOG 06/10/2020

SN 80-007098 8164.16.00 09/18 14:31

DRY CAL CHECK

Test		AC	Time
01	Room Air	0.000	14:32
02	Std. Gas	0.081	14:32
03	Room Air	0.000	14:33
04	Reference	RFI*	14:33
05	Room Air	0.000	14:33

*Invalid Test Inhibited - RFI

Lot No = 24119080A1

Cyl No = 2

Exp Date = 11/05/2021

County = 53

Oper No. = 130977

Operator Signature JOSHUA NELSON

Remarks:

Form 106-I8000

I certify this is a true and accurate copy.