

**INTOXILYZER® 8000 INITIAL INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-007096

**A. GENERAL Setup and Checks:**

- Power On
  - 5 min. countdown during initial warm-up
  - 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
- Password Security:
  - Level 1 (Operator)     Level 2 (Field Inspector)     Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

**B. LEVEL 1 Setup and Checks:**

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
- Barcode Reader (Level 1,B):
  - Operator ID     ND License     R key: Refusal. Attach record.

**C. LEVEL 2 Setup and Checks:**

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
- Ser Number/Version Info (Level 2,V):
  - Inst Ser#: 80-007096    DSP Ver: 1362.14
  - Core Ver: 8164.16    CPLD Ver: 7

**D. LEVEL 3 Setup and Checks:**

- DVM Monitor (Level 3,D,D): 3: 13266    s: 2    C: 47.00  
9: 13436    s: 1    B: 45    0  
(RFI indicator with keyed radio): 1
- Function (F) Keys: :  F5 – Tone                                     F9 - Fan  
 F6 – Green LED                                     F10 – Pump  
 F7 – Red LED                                         F11 – Solenoid
- Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 957    CF: 1.05
- Flow Monitor (Level 3,D,F): Flow Sensor, R: 215    F: 0.000    L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 46.0 I: 0.0  
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 224 psi P (gauge): 250 psi
7.  General Setup (Level 3,S,G)
8.  Configure Start Test (Level 3,S,T)
9.  Network (Level 3,S,N)

**E. TESTS (attach all test records):**

**Configure wet bath simulator standards before running tests (Level 1,S):**

1. Zero AC (Level 1,C):  
Sim ser #: MP3002 Lot #: MILLI Q H2O Exp date: NA  
Known value: 0.000 AC  
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):  
Sim ser #: MP3064 Lot #: 20070 Exp date: 2.13.22  
Known value: 0.020 AC  
Results: 0.018 AC 0.017 AC 0.018 AC
3. High AC (Level 1,C): 9  
Sim ser #: MP3069 Lot #: 20902D Exp date: 2.20.21  
Known value: 0.300 AC  
Results: 0.298 AC 0.299 AC 0.299 AC

**Configure dry gas standard before running tests (Level 1,S):**

4. Ethanol + 0.05% Acetone (Level 1,B):  
Sim ser #: DR3451 Lot #: IC56 Exp date: NA  
Known value: 0.10% AC + 0.05% ACETONE  
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):  
Sim ser #: DR5188 Lot #: ACE1 Exp date: NA  
Known value: 0.05% ACETONE  
 Display reads "Interferent Detect"
6. Methanol (Level 1,B): 3455 MET2  
Sim ser #: DR5189 Lot #: IPA2 CEE Exp date: NA  
Known value: 0.10% METHANOL  
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):  
Sim ser #: DR5189 Lot #: IPA2 Exp date: NA  
Known value: 0.10% ISOPROPANOL  
 Display reads "Interferent Detect"

- 8.  Ambient Fail (Level 1,B): Display and test record read correct.
- 9.  Invalid Sample (Level 1,B): Display and test record read correct.
- 10.  Sequence Aborted (Level 1,B): Display and test record read correct.
- 11.  ABA (Level 1,B): Test record reads correct.
- 12.  Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13.  RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15.  Deficient Samples (CMS mode): Display and test record read correct.
- 16.  Improper Sample (CMS mode): Display and test record read correct.
- 17.  Difference Too Great (CMS mode): Display and test record read correct.
- 18.  CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0797 Std Dev: 0.0005 Rel Std Dev: 0.59%

Maintenance/Repairs/Notes: INITIAL INSPECTION DUE TO  
INSTRUMENT BEING OPENED UP DURING REPAIR  
OF INTERNAL PRINTER.

Instrument is acceptable to be used in the field.

Charles E. Ed  
Field Inspector's Signature

7/27/20  
Date

NA  
Reviewed by

NA  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-007096  
Location = TOXL 8164.16.00 09/18  
07/27/2020 10:50

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN

24 characters per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN

32 characters per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN


42 characters per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN

Current Instrument Setup  
Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On MemFull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 34418080A2  
Standard Cyl #? 7  
Standard Expiration? 02/05/2021  
Oper No? 666666

Flow Cal. Date: 10/02/2018  
Slope 695  
Intercept -692605

IR Calibration Date: 10/02/2018  
3um 9um  
-----  
0th Coef(x100): -33544 -18475  
1st Coef(x100): 261464 138054  
2nd Coef(x100): 3020 1612  
H2O adj(mg/l)x10k): 573 386

\*\*\*\*\* Test End \*\*\*\*\*

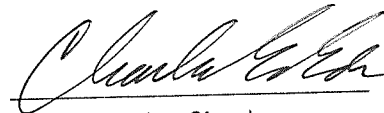

Operator Signature  
CHARLES EDER

Remarks: *Internal  
PRINT TEST*

Form 106-18000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-007096  
Location = TOXL                8164.16.00 09/18  
07/27/2020                       10:52

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#%&\*()\_+?

Current Instrument Setup


Data Entry Mode:               Enabled  
Start Test Sequence:         DABACABA  
Display Prelim Rslt?         Yes  
Display Third Digit?         Yes  
Inhib Printer(Y/N)?         No  
Display Volume?               No  
Disable On Memfull?         Yes  
# of Print Copies?            1  
Select Std (D/W/I)?         Dry  
Standard Value?               0.080  
Standard Lot #?               34418080A2  
Standard Cyl #?               7  
Standard Expiration?         02/05/2021  
Oper No?                       666666

Flow Cal. Date:               10/02/2018  
Slope                         695  
Intercept                    -692605

IR Calibration Date:         10/02/2018  
                                  3um               9um

	3um	9um
0th Coef(*100):	-33544	-18475
1st Coef(*100):	261464	138054
2nd Coef(*100):	3020	1612
H2O adj(mg/l*10k):	573	386

\*\*\*\*\* Printer Test End \*\*\*\*\*

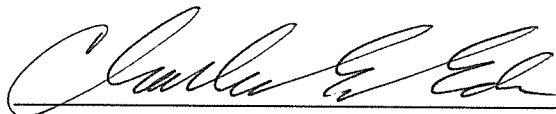

Operator Signature  
CHARLES EDER

Remarks:

EXTERNAL PRINT  
TEST

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      10:54

Test	AC	Time
01 Room Air	0.000	10:55
02 *Subject Test	REF*	10:55
03 Room Air	0.000	10:55

\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = R KEY REFUSAL  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 7  
Expiration Date = 02/05/2021  
County = 08      Oper No. = 666666

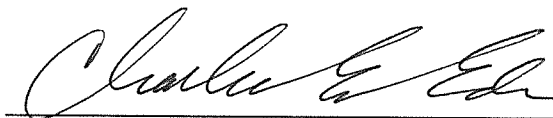

Operator Signature  
CHARLES EDER

Remarks: "R" KEY REFUSAL

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      10:56

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*DIAGNOSTIC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      11:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:05
02 Std. Sol.	0.000	11:05
03 Room Air	0.000	11:06
04 Std. Sol.	0.000	11:07
05 Room Air	0.000	11:07
06 Std. Sol.	0.000	11:08
07 Room Air	0.000	11:08

08 Sim Temp = 34.0°C

Simul Ser No = MP3002  
Std Sol No = MILLIQ H2O  
County = 08      Oper No. = 666666

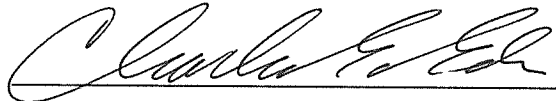

Operator Signature  
CHARLES EDER

Remarks: ZERO AC  
0.000 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      11:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:34
02 Std. Sol.	0.018	11:35
03 Room Air	0.000	11:35
04 Std. Sol.	0.017	11:36
05 Room Air	0.000	11:37
06 Std. Sol.	0.018	11:37
07 Room Air	0.000	11:38

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = 20070  
County = 08      Oper No. = 666666

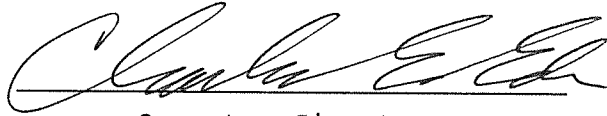

Operator Signature  
CHARLES EDER

Remarks:      *Low AC*  
                    *0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      11:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:41
02 Std. Sol.	0.298	11:41
03 Room Air	0.000	11:42
04 Std. Sol.	0.299	11:43
05 Room Air	0.000	11:43
06 Std. Sol.	0.299	11:44
07 Room Air	0.000	11:45

08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      11:47

Test	AC	Time
01 Room Air	0.000	11:48
02 *Subject Test	INT*	11:48
03 Room Air	0.000	11:49

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT'  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666

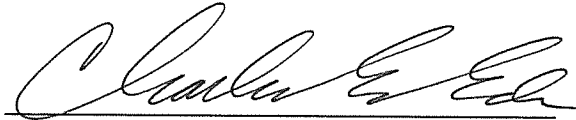

Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10AC + 0.05% ACETONE*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007096  
Location = TOXL                    8164.16.00 09/18  
07/27/2020                            11:50

Test	AC	Time
01 Room Air	0.000	11:51
02 *Subject Test	INT*	11:51
03 Room Air	0.000	11:52

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08                            Oper No. = 666666

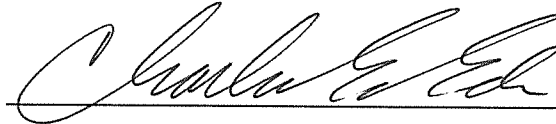

Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      11:54

Test	AC	Time
01 Room Air	0.000	11:54
02 *Subject Test	INT*	11:55
03 Room Air	0.000	11:55

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10% METHANOL*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      11:55

Test	AC	Time
01 Room Air	0.000	11:56
02 *Subject Test	INT*	11:57
03 Room Air	0.000	11:57

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10% ISOPROPANOL*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      11:59

Test	AC	Time
01 Room Air	AMB*	11:59
02 Room Air	0.000	12:00

\*Invalid Test  
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = AMBIENT FAIL  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *AMBIENT FAIL*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      12:01

Test	AC	Time
01 Room Air	0.000	12:02
02 Invalid Sample	X.XXX	12:02
03 Room Air	0.000	12:03

\*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INVALID SAMPLE  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666

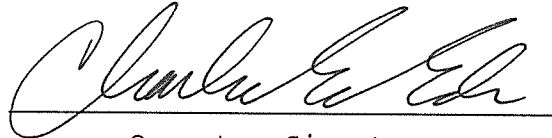

Operator Signature  
CHARLES EDER

Remarks: *INVALID SAMPLE*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007096  
Location = TOXL                 8164.16.00 09/18  
07/27/2020                        12:05

Test	AC	Time
01 Room Air	0.000	12:06
02 *Subject Test	ABT*	12:06
03 Room Air	0.000	12:06

\*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = SEQ ABORTED  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: SEQUENCE ABORTED

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      12:07

Test	AC	Time
01 Room Air	0.000	12:08
02 Subject Test 1	0.019	12:08
03 Room Air	0.000	12:08
04 Reported AC	0.019	12:08

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = ABA MODE  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *ABA MODE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      12:09

Test	AC	Time
01 Diagnostic	OK	12:10
02 Room Air	0.000	12:10
03 Subject Test 1	0.019	12:11
04 Room Air	0.000	12:13
05 Std. Gas	0.088*	12:14
06 Room Air	0.000	12:15

\*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = CAL CHK OUT TOL  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
OUT OF TOLERANCE

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007096  
Location = TOXL                    8164.16.00 09/18  
07/27/2020                            12:16

Test	AC	Time
01 Diagnostic	OK	12:17
02 Room Air	0.000	12:17
03 *Subject Test	RFI*	12:17
04 Room Air	0.000	12:18

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007096  
Location = TOXL                    8164.16.00 09/18  
07/27/2020                            12:19

Test	AC	Time
01 Diagnostic	OK	12:20
02 Room Air	0.000	12:20
03 *Subject Test	0.000*	12:23
04 Room Air	0.000	12:25
05 Std. Gas	0.080	12:26
06 Room Air	0.000	12:28
07 *Subject Test	0.000*	12:31
08 Room Air	0.000	12:32

No RFI Detected

\*Deficient Sample - Value Printed was  
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = DEFICIENT SAMP  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08                            Oper No. = 666666

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

DEFICIENT  
SAMPLES

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00.09/18  
07/27/2020      12:32

Test	AC	Time
01 Diagnostic	OK	12:33
02 Room Air	0.000	12:33
03 Reference	IPS*	12:33
04 Room Air	0.000	12:34

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = IMPROPER SAMPLE  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

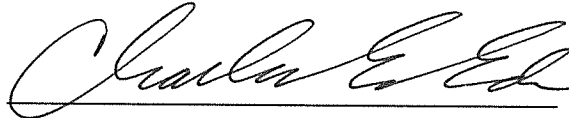

Operator Signature  
CHARLES EDER

Remarks:

*IMPROPER SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      13:13

Test	AC	Time
01 Diagnostic	OK	13:14
02 Room Air	0.000	13:15
03 Subject Test 1	0.019*	13:15
04 Room Air	0.000	13:17
05 Std. Gas	0.079	13:18
06 Room Air	0.000	13:20
07 Subject Test 2	0.147*	13:22
08 Room Air	0.000	13:23

\*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = DIFF TOO GREAT  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

DIFFERENCE  
TOO GREAT

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007096  
Location = TOXL                8164.16.00 09/18  
07/27/2020                      13:24

Test	AC	Time
01 Diagnostic	OK	13:25
02 Room Air	0.000	13:25
03 Subject Test 1	0.019	13:26
04 Room Air	0.000	13:28
05 Std. Gas	0.079	13:29
06 Room Air	0.000	13:31
07 Subject Test 2	0.018	13:31
08 Room Air	0.000	13:32
09 Reported AC	0.018	13:31

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = CMS MODE  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 9  
Expiration Date = 11/05/2021  
County = 08                              Oper No. = 666666

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: CMS MODE

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = TOXL      8164.16.00 09/18  
07/27/2020      13:33

Test	AC	Time
01 Room Air	0.000	13:33
02 Std. Gas	0.080	13:34
03 Room Air	0.000	13:34
04 Std. Gas	0.080	13:34
05 Room Air	0.000	13:35
06 Std. Gas	0.080	13:35
07 Room Air	0.000	13:36
08 Std. Gas	0.079	13:36
09 Room Air	0.000	13:37
10 Std. Gas	0.080	13:37
11 Room Air	0.000	13:38
12 Std. Gas	0.079	13:38
13 Room Air	0.000	13:38
14 Std. Gas	0.079	13:39
15 Room Air	0.000	13:39
16 Std. Gas	0.079	13:40
17 Room Air	0.000	13:40
18 Std. Gas	0.080	13:40
19 Room Air	0.000	13:41
20 Std. Gas	0.080	13:41
21 Room Air	0.000	13:42
22 Std. Gas	0.080	13:42
23 Room Air	0.000	13:43
24 Std. Gas	0.080	13:43
25 Room Air	0.000	13:44
26 Std. Gas	0.080	13:44
27 Room Air	0.000	13:44
28 Std. Gas	0.080	13:45
29 Room Air	0.000	13:45
30 Std. Gas	0.079	13:46
31 Room Air	0.000	13:46
32 Std. Gas	0.080	13:47
33 Room Air	0.000	13:47
34 Std. Gas	0.080	13:47
35 Room Air	0.000	13:48
36 Std. Gas	0.080	13:48
37 Room Air	0.000	13:49
38 Std. Gas	0.080	13:49
39 Room Air	0.000	13:50
40 Std. Gas	0.079	13:50
41 Room Air	0.000	13:51
42 Cal Check Stats:		
43 Average	0.0797	
44 Std Dev	0.0005	
45 Rel Std Dev	0.59%	



Operator Signature  
CHARLES EDER

0.080 AC

Remarks:

LOT: 24119080 A1

CYL: 9

Form 106-I8000

Exp: 11/5/2021