

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-007097

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 461 psi Regulator: 475 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.020 AC
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
- b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.300 AC
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS 6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 24119080A1 Cyl #: 6 Exp. Date: 11.5.21

Test 1	<u>0.083</u>	Test 4	<u>0.083</u>	Test 7	<u>0.083</u>
Test 2	<u>0.082</u>	Test 5	<u>0.083</u>	Test 8	<u>0.082</u>
Test 3	<u>0.083</u>	Test 6	<u>0.083</u>	Test 9	<u>0.083</u>
Average	<u>0.083</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION - REPLACED
EXTERNAL BATTERY

OFFICER STAYED DURING INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr
Breath Analyst Signature

4/7/2021
Date

Kelli L. Hieb
Reviewed by

4-12-2021
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 09:42

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 24119080A1
Standard Cyl #? 6
Standard Expiration? 11/05/2021
Oper No? 666666

Flow Cal. Date: 09/03/2020
Slope 660
Intercept -636264

IR Calibration Date: 09/03/2020

	3um	9um
0th Coef(*100):	-36810	-25919
1st Coef(*100):	254044	139068
2nd Coef(*100):	1858	1258
H2O adj(mg/l*10k):	903	584

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 09:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:45
02 Std. Sol.	0.020	09:46
03 Room Air	0.000	09:47
04 Std. Sol.	0.020	09:47
05 Room Air	0.000	09:48
06 Std. Sol.	0.020	09:49
07 Room Air	0.000	09:49


08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 20070

County = 08

Oper No. = 666666

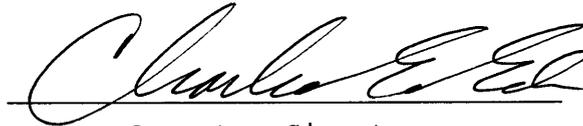

Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 09:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:53
02 Std. Sol.	0.307	09:53
03 Room Air	0.000	09:54
04 Std. Sol.	0.307	09:55
05 Room Air	0.000	09:55
06 Std. Sol.	0.305	09:56
07 Room Air	0.000	09:56

08 Sim Temp = 34.0°C

Simul Ser No = MP5317
Std Sol No = 202012A
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
 0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 09:58

Test	AC	Time
01 Room Air	0.000	09:59
02 *Subject Test	INT*	09:59
03 Room Air	0.000	10:00

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 6
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

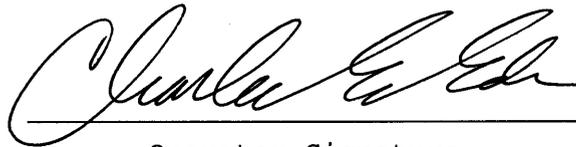

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 10:01

Test	AC	Time
01 Diagnostic	OK	10:02
02 Room Air	0.000	10:02
03 *Subject Test	RFI*	10:02
04 Room Air	0.000	10:03

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 6
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

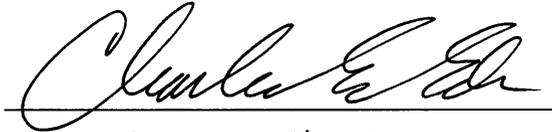

Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 10:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:04
02 Std. Gas	0.083	10:05
03 Room Air	0.000	10:05
04 Std. Gas	0.082	10:05
05 Room Air	0.000	10:06
06 Std. Gas	0.083	10:06
07 Room Air	0.000	10:07

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666

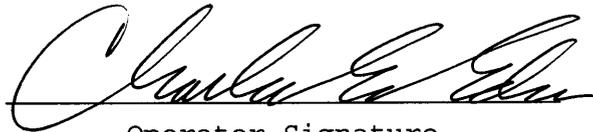

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 10:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:08
02 Std. Gas	0.083	10:08
03 Room Air	0.000	10:09
04 Std. Gas	0.083	10:09
05 Room Air	0.000	10:09
06 Std. Gas	0.083	10:10
07 Room Air	0.000	10:10

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666

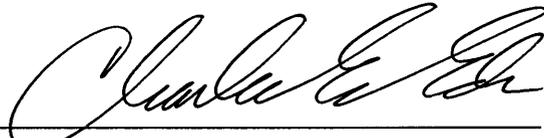

Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = TOXL 8164.16.00 09/18
04/07/2021 10:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:11
02 Std. Gas	0.083	10:12
03 Room Air	0.000	10:12
04 Std. Gas	0.082	10:12
05 Room Air	0.000	10:13
06 Std. Gas	0.083	10:13
07 Room Air	0.000	10:14

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000