

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-007092

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 197 psi Regulator: 225 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.030 AC  
Sim. Ser #: MP5289 Lot #: 201911E Exp. Date: 11.19.21
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.400 AC  
Sim. Ser #: MP3062 Lot #: 202103E Exp. Date: 3.4.23

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3847 Lot #: ICS 6 Exp. Date: NA

Display reads "Interferent Detect"

3.  RFI Check (CMS Mode)  Display reads "RFI Detect"

4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC

Gas Cyl Lot #: 02621080A1 Cyl #: 26 Exp. Date: 3.5.23

Test 1	<u>0.080</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.079</u>	Test 6	<u>0.080</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, SPEAKER VOLUME IS LOW BUT STILL AUDIBLE.

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

Date

Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007092  
Location = TOXL                 8164.16.00 09/18  
06/03/2021                        11:07

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:                Enabled  
Start Test Sequence:            DABACABA  
Display Prelim Rslt?            Yes  
Display Third Digit?            Yes  
Inhib Printer(Y/N)?            No  
Display Volume?                No  
Disable On Memfull?            Yes  
# of Print Copies?              1  
Select Std (D/W/I)?            Dry  
Standard Value?                0.080  
Standard Lot #?                 24119080A1  
Standard Cyl #?                3  
Standard Expiration?          11/05/2021  
Oper No?                        666666

Flow Cal. Date:                 09/26/2018  
Slope                            670  
Intercept                        -596123

IR Calibration Date:            09/26/2018  
                                  3um                            9um

-----  
0th Coef(\*100):                -27453                        -17291  
1st Coef(\*100):                248629                        134601  
2nd Coef(\*100):                2770                            1393  
H2O adj(mg/l\*10k):            872                             414

\*\*\*\*\* Printer Test End \*\*\*\*\*

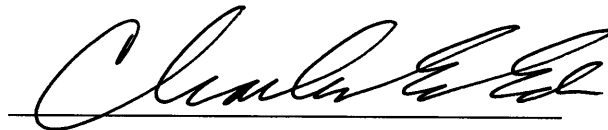

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007092  
Location = TOXL      8164.16.00 09/18  
06/03/2021      11:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Sol.	0.029	11:16
03 Room Air	0.000	11:16
04 Std. Sol.	0.028	11:17
05 Room Air	0.000	11:18
06 Std. Sol.	0.029	11:18
07 Room Air	0.000	11:19

08 Sim Temp = 34.0°C

Simul Ser No = MP5289

Std Sol No = 201911E

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

LOW AC

0.030 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007092  
Location = TOXL      8164.16.00 09/18  
06/03/2021      11:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:20
02 Std. Sol.	0.401	11:21
03 Room Air	0.000	11:22
04 Std. Sol.	0.399	11:22
05 Room Air	0.000	11:23
06 Std. Sol.	0.398	11:24
07 Room Air	0.000	11:24

08 Sim Temp = 34.0°C

Simul Ser No = MP3062

Std Sol No = 202103E

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007092  
Location = TOXL      8164.16.00 09/18  
06/03/2021      11:28

Test	AC	Time
01 Room Air	0.000	11:29
02 *Subject Test	INT*	11:29
03 Room Air	0.000	11:30

\*Invalid Test  
Interferent Detected

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male      Weight = NA  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08      Oper No. = 666666

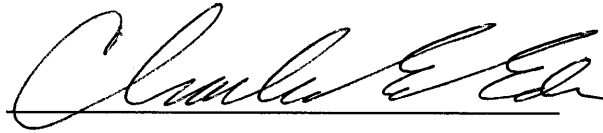

Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007092  
Location = TOXL      8164.16.00 09/18  
06/03/2021      11:34

Test	AC	Time
01 Diagnostic	OK	11:35
02 Room Air	0.000	11:35
03 *Subject Test	RFI*	11:35
04 Room Air	0.000	11:36

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male      Weight = NA  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007092  
Location = TOXL      8164.16.00 09/18  
06/03/2021      11:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:41
02 Std. Gas	0.080	11:42
03 Room Air	0.000	11:42
04 Std. Gas	0.080	11:43
05 Room Air	0.000	11:43
06 Std. Gas	0.079	11:44
07 Room Air	0.000	11:44

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666

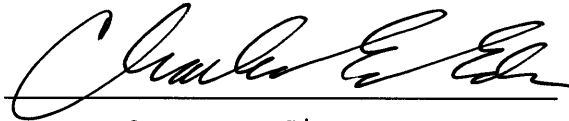

Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007092  
Location = TOXL      8164.16.00 09/18  
06/03/2021      11:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:45
02 Std. Gas	0.080	11:45
03 Room Air	0.000	11:46
04 Std. Gas	0.080	11:46
05 Room Air	0.000	11:47
06 Std. Gas	0.080	11:47
07 Room Air	0.000	11:47

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007092  
Location = TOXL      8164.16.00 09/18  
06/03/2021      11:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:48
02 Std. Gas	0.080	11:49
03 Room Air	0.000	11:49
04 Std. Gas	0.080	11:50
05 Room Air	0.000	11:50
06 Std. Gas	0.080	11:51
07 Room Air	0.000	11:51

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000