



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tracy L. Meidinger</i>		
Location <i>UND-PD</i>	Intoxilyzer® Serial Number <i>80-007092</i>	
Gas Lot Number <i>200 02621080A1</i>	Gas Cylinder Number <i>17</i>	Gas Expiration Date <i>03-05-2023</i>

Check When Done:

- TM 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- TM 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.079* AC
 - B. *0.080* AC
 - C. *0.080* AC
- TM 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- TM 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- TM 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Tracy L. Meidinger</i>	Date <i>06-04-21</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ed</i>	Date <i>6.9.2021</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007092
Location = UNDP 8164.16.00 09/18
06/04/2021 10:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:33
02 Std. Gas	0.079	10:34
03 Room Air	0.000	10:34
04 Std. Gas	0.080	10:35
05 Room Air	0.000	10:35
06 Std. Gas	0.080	10:36
07 Room Air	0.000	10:36

Lot No = 02621080A1
Cyl No = 17
Exp Date = 03/05/2023
County = 18 Oper No. = 119409


Operator Signature
TRACY MEIDINGER

Remarks:

Form 106-I8000