

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-007093

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 933 psi Regulator: 950 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
  - a.  Low AC (Level 1,C): Known Value  $\leq$  0.03 AC: 0.070 AC  
Sim. Ser #: MP5290 Lot #: 21340 Exp. Date: 8/24/23
  - b.  High AC (Level 1,C): Known Value  $\geq$  0.25 AC: 0.300 AC  
Sim. Ser #: MP3067 Lot #: 21470 Exp. Date: 10/20/23

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR5114 Lot #: 1656 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 02621080A1 Cyl #: 31 Exp. Date: 3/5/23

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual inspection.  
Replaced breath tube o-ring + simulator return o-ring.  
Left instrument in mountain home for annual inspection.  
Replaced external battery.

Instrument is acceptable to be used in the field.

Janelle Putschillo  
Breath Analyst Signature

6/27/2022  
Date

[Signature]  
Reviewed by

070122  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007093  
Location = TOXL                    8164.16.00 09/18  
06/27/2022                            14:02

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            02621080A1  
Standard Cyl #?            24  
Standard Expiration?       03/05/2023  
Oper No?                    132128

Flow Cal. Date:            06/07/2019  
Slope                        638  
Intercept                   -611758

IR Calibration Date:       12/16/2019  
                                    3um                    9um

	3um	9um
0th Coef(*100):	-14610	-32356
1st Coef(*100):	264615	138933
2nd Coef(*100):	2992	1364
H2O adj(mg/l*10k):	369	621

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007093  
Location = TOXL                    8164.16.00 09/18  
06/27/2022                            14:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:15
02 Std. Sol.	0.017	14:16
03 Room Air	0.000	14:17
04 Std. Sol.	0.017	14:17
05 Room Air	0.000	14:18
06 Std. Sol.	0.017	14:19
07 Room Air	0.000	14:19

08 Sim Temp = 34.0°C

Simul Ser No = MP5290  
Std Sol No = 21340  
County = 08                            Oper No. = 132128

  
\_\_\_\_\_  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Low AC check 0.020*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007093  
Location = TOXL                    8164.16.00 09/18  
06/27/2022                            14:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:22
02 Std. Sol.	0.291	14:23
03 Room Air	0.000	14:24
04 Std. Sol.	0.294	14:24
05 Room Air	0.000	14:25
06 Std. Sol.	0.294	14:26
07 Room Air	0.000	14:26

08 Sim Temp = 34.0°C

Simul Ser No = MP3067  
Std Sol No = 21420  
County = 08                            Oper No. = 132128

  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Interferent JP*  
*High Ac Check 0.300*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007093  
Location = TOXL      8164.16.00 09/18  
06/27/2022      14:31

Test	AC	Time
01 Room Air	0.000	14:32
02 *Subject Test	INT*	14:34
03 Room Air	0.000	14:35

\*Invalid Test  
Interferent Detected

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male      Weight = NA  
Test = DUI      Cit = INTERFERENT CK  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 31  
Expiration Date = 03/05/2023  
County = 08      Oper No. = 132128



Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Interferent check*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007093  
Location = TOXL                    8164.16.00 09/18  
06/27/2022                            14:37

Test	AC	Time
01 Diagnostic	OK	14:38
02 Room Air	0.000	14:38
03 *Subject Test	RFI*	14:39
04 Room Air	0.000	14:39

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male                            Weight = NA  
Test = DUI                                    Cit = RFI CHK  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 31  
Expiration Date = 03/05/2023  
County = 08                                    Oper No. = 132128

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
\_\_\_\_\_  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *RFI check*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007093  
Location = TOXL                 8164.16.00 09/18  
06/27/2022                        14:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Gas	0.080	14:43
03 Room Air	0.000	14:43
04 Std. Gas	0.079	14:44
05 Room Air	0.000	14:44
06 Std. Gas	0.079	14:45
07 Room Air	0.000	14:45

Lot No = 02621080A1  
Cyl No = 31  
Exp Date = 03/05/2023  
County = 08                            Oper No. = 132128

  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: ~~Low AC check~~ <sup>gap</sup> #1-3  
Dry Calibration check

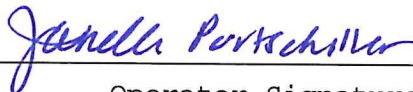
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007093  
Location = TOXL      8164.16.00 09/18  
06/27/2022      14:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:46
02 Std. Gas	0.080	14:46
03 Room Air	0.000	14:47
04 Std. Gas	0.080	14:47
05 Room Air	0.000	14:48
06 Std. Gas	0.080	14:48
07 Room Air	0.000	14:49

Lot No = 02621080A1  
Cyl No = 31  
Exp Date = 03/05/2023  
County = 08      Oper No. = 132128

  
\_\_\_\_\_  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Dry calibration checks #4-6*




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007093  
Location = TOXL      8164.16.00 09/18  
06/27/2022      14:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:53
02 Std. Gas	0.080	14:53
03 Room Air	0.000	14:54
04 Std. Gas	0.080	14:54
05 Room Air	0.000	14:55
06 Std. Gas	0.079	14:55
07 Room Air	0.000	14:56

Lot No = 02621080A1  
Cyl No = 31  
Exp Date = 03/05/2023  
County = 08      Oper No. = 132128

  
\_\_\_\_\_  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check # 7-9*