



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

Serial Number <u>80-007091</u>	Instrument Location <u>Mott</u>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Gas Cylinder Exchange</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): Cylinder Replacement
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

NOTE: DATE ON TEST RECORDS IS 11-08-2020 CEE

Field Inspector Signature <u>[Signature]</u>	Date <u>8/11/20</u> <u>11/8/20</u> CEE
Reviewed By (Crime Laboratory Use Only) <u>[Signature]</u>	Date <u>11/17/20</u>

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By: [Signature] Certified Date: 11/17/20

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007091
Location = MOTT 8164.16.00 09/18
11/08/2020 13:36

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05620080A1
Standard Cyl #? 41
Standard Expiration? 04/05/2022
Oper No? 132666

Flow Cal. Date: 09/25/2018
Slope 681
Intercept -651030

IR Calibration Date: 09/25/2018
 3um 9um

0th Coef(*100): -39121 -20906
1st Coef(*100): 240667 132345
2nd Coef(*100): 3960 1516
H2O adj(mg/l*10k): 899 428

***** Printer Test End *****



Operator Signature
JEDEDIAH KOHLER

Remarks: Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007091
Location = MOTT 8164.16.00 09/18
11/08/2020 13:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:14
02 Std. Gas	0.079	13:14
03 Room Air	0.000	13:15
04 Std. Gas	0.078	13:15
05 Room Air	0.000	13:16
06 Std. Gas	0.079	13:16
07 Room Air	0.000	13:16

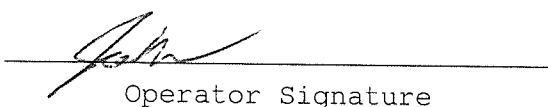
Lot No = 05620080A1

Cyl No = 41

Exp Date = 04/05/2022

County = 21

Oper No. = 132666


Operator Signature
JEDEDIAH KOHLER

Remarks:

ACA Function Test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007091
Location = MOTT 8164.16.00 09/18
11/08/2020 13:39

Test	AC	Time
01 Room Air	RFI*	13:42
02 Room Air	0.000	13:42

*Invalid Test
Inhibited - RFI

Sub Name = KOHLER, JED S
Sub DOB = 08/20/1987
Sub Sex = Male Weight = 165
Test = OTH Cit = NA
Dr. Lic. = ND/NA
Lot No = 05620080A1
Cyl No = 41
Expiration Date = 04/05/2022
County = 21 Oper No. = 132666



Operator Signature
JEDEDIAH KOHLER

Remarks:

RFI test

Form 106-I8000