



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tim Jordan</i>		
Location <i>Lisbon</i>	Intoxilyzer® Serial Number <i>80-007086</i> ^{SEP} <i>12 Apr 22</i>	
Gas Lot Number <i>34921080A1</i>	Gas Cylinder Number <i>60</i>	Gas Expiration Date <i>215/2024</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.082 AC
 - B. 0.082 AC
 - C. 0.082 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Tim Jordan</i>	Date <i>3/30/22</i>
Reviewed By (Crime Laboratory Use Only) <i>Janelle Pertschiller</i>	Date <i>12 Apr 22</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007086
Location = LISB 8164.16.00 09/18
03/30/2022 13:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:23
02 Std. Gas	0.082	13:23
03 Room Air	0.000	13:24
04 Std. Gas	0.082	13:24
05 Room Air	0.000	13:25
06 Std. Gas	0.082	13:25
07 Room Air	0.000	13:26


Lot No = 34921080A1

Cyl No = 60

Exp Date = 02/05/2024

County = 37

Oper No. = 020607

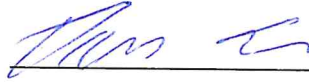

Operator Signature
TIMOTHY JORDAN

Remarks: *ACA + PST*

Form 106-I8000