

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006507

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR5114 Lot No. 201505B Rep AC 0.030 ✓
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3068 Lot No. 14102 Rep AC 0.400 ✓
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS 4 Solution 0.10 AC + 0.05% ACETONE ✓
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016


Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>

Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 708 psi Reg ~700 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

11 APR 16
Date
11 Apr 2016
Date


Field Inspector's Signature
Deb Sheaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 14:42

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 13314080A3
Standard Cyl #? 28
Standard Expiration? 06/05/2016
Oper No? 666666

Flow Cal. Date: 08/20/2015
Slope 659
Intercept -658315

IR Calibration Date: 08/20/2015
 3um 9um

0th Coef(*100): -9452 -11722
1st Coef(*100): 258808 139588
2nd Coef(*100): 4476 1674
H2O adj(mg/l*10k): 499 376

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 14:51

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:52
02 Std. Sol.	0.028	14:53
03 Room Air	0.000	14:53
04 Std. Sol.	0.029	14:54
05 Room Air	0.000	14:54
06 Std. Sol.	0.029	14:55
07 Room Air	0.000	14:56

08 Sim Temp = 34.0°C

Simul Ser No = DR5114
Std Sol No = 201505B
County = 08 Oper No. = 666666

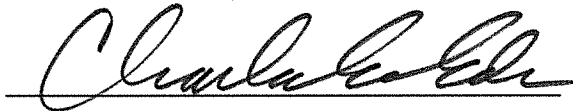

Operator Signature
CHARLES EDER

Remarks:

Low AC
0.030 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 14:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:58
02 Std. Sol.	0.404	14:59
03 Room Air	0.000	15:00
04 Std. Sol.	0.407	15:00
05 Room Air	0.000	15:01
06 Std. Sol.	0.407	15:02
07 Room Air	0.000	15:02

08 Sim Temp = 34.0°C

Simul Ser No = MP3068

Std Sol No = 14102

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 15:04

Test	AC	Time
01 Room Air	0.000	15:04
02 *Subject Test	INT*	15:05
03 Room Air	0.000	15:06

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 15:06

Test	AC	Time
01 Diagnostic	OK	15:07
02 Room Air	0.000	15:08
03 *Subject Test	RFI*	15:08
04 Room Air	0.000	15:08

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13314080A3
Cyl No = 28
Expiration Date = 06/05/2016
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 15:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:10
02 Std. Gas	0.080	15:11
03 Room Air	0.000	15:11
04 Std. Gas	0.080	15:12
05 Room Air	0.000	15:12
06 Std. Gas	0.080	15:13
07 Room Air	0.000	15:13

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 15:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:14
02 Std. Gas	0.080	15:14
03 Room Air	0.000	15:15
04 Std. Gas	0.080	15:15
05 Room Air	0.000	15:16
06 Std. Gas	0.080	15:16
07 Room Air	0.000	15:17

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = TOXL 8164.13.00 06/09
04/11/2016 15:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:17
02 Std. Gas	0.080	15:18
03 Room Air	0.000	15:18
04 Std. Gas	0.080	15:19
05 Room Air	0.000	15:19
06 Std. Gas	0.081	15:20
07 Room Air	0.000	15:20

Lot No = 13314080A3
Cyl No = 28
Exp Date = 06/05/2016
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC