

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006508

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
  - B. Is the diagnostic check complete? ✓
  - C. Is the time, date, and year correct (re-set if necessary)? ✓
  - D. Print test. (Attach test record.) ✓
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5114 Lot No. 201505B Rep AC 0.030 ✓
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP306B Lot No. 14102 Rep AC 0.400 ✓
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. ICS4 Solution 0.10AC+0.05%ACETONE ✓
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 13314080A3 Cyl No. 028 Exp. Date 6/5/2016

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 735 psi Reg ~750 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

11 APR. 16  
Date  
11 Apr 2016  
Date

Cleaveland E. Eden  
Field Inspector's Signature  
Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = TOXL              8164.13.00 06/09  
04/11/2016                      13:27

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&\*( )\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&\*( )\_+?

Current Instrument Setup

Data Entry Mode:              Enabled  
Start Test Sequence:          DABACABA  
Display Prelim Rslt?          Yes  
Display Third Digit?          Yes  
Inhib Printer(Y/N)?          No  
Display Volume?                No  
Disable On Memfull?          Yes  
# of Print Copies?            1  
Select Std (D/W/I)?          Dry  
Standard Value?                0.080  
Standard Lot #?                13314080A3  
Standard Cyl #?                28  
Standard Expiration?         06/05/2016  
Oper No?                        666666

Flow Cal. Date:                08/20/2015  
Slope                            644  
Intercept                        -568181

IR Calibration Date:          08/20/2015  
    3um    9um

	3um	9um
0th Coef(*100):	-22365	-18627
1st Coef(*100):	256969	136095
2nd Coef(*100):	3168	1444
H2O adj(mg/l*10k):	693	453

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = TOXL      8164.13.00 06/09  
04/11/2016      13:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:35
02 Std. Sol.	0.028	13:36
03 Room Air	0.000	13:36
04 Std. Sol.	0.027	13:37
05 Room Air	0.000	13:38
06 Std. Sol.	0.028	13:38
07 Room Air	0.000	13:39

08 Sim Temp = 34.0°C

Simul Ser No = DR5114

Std Sol No = 201505B

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC

0.030 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = TOXL      8164.13.00 06/09  
04/11/2016      13:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:53
02 Std. Sol.	0.400	13:54
03 Room Air	0.000	13:54
04 Std. Sol.	0.398	13:55
05 Room Air	0.000	13:56
06 Std. Sol.	0.399	13:57
07 Room Air	0.000	13:57

08 Sim Temp = 34.0°C

Simul Ser No = MP3068  
Std Sol No = 14102  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = TOXL      8164.13.00 06/09  
04/11/2016      14:12

Test	AC	Time
01 Room Air	0.000	14:13
02 *Subject Test	INT*	14:14
03 Room Air	0.000	14:14

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08      Oper No. = 666666

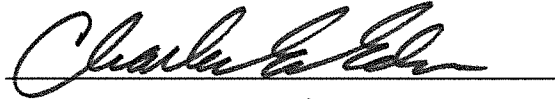

Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = TOXL      8164.13.00 06/09  
04/11/2016      14:16

Test	AC	Time
01 Diagnostic	OK	14:16
02 Room Air	0.000	14:17
03 *Subject Test	RFI*	14:17
04 Room Air	0.000	14:17

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 13314080A3  
Cyl No = 28  
Expiration Date = 06/05/2016  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-006508  
Location = TOXL 8164.13.00 06/09  
04/11/2016 14:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:19
02 Std. Gas	0.080	14:20
03 Room Air	0.000	14:20
04 Std. Gas	0.081	14:21
05 Room Air	0.000	14:21
06 Std. Gas	0.081	14:22
07 Room Air	0.000	14:22

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08 Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = TOXL      8164.13.00 06/09  
04/11/2016      14:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:24
02 Std. Gas	0.080	14:24
03 Room Air	0.000	14:25
04 Std. Gas	0.081	14:25
05 Room Air	0.000	14:26
06 Std. Gas	0.081	14:26
07 Room Air	0.000	14:27

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = TOXL      8164.13.00 06/09  
04/11/2016      14:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:27
02 Std. Gas	0.081	14:28
03 Room Air	0.000	14:28
04 Std. Gas	0.081	14:29
05 Room Air	0.000	14:29
06 Std. Gas	0.081	14:29
07 Room Air	0.000	14:30

Lot No = 13314080A3  
Cyl No = 28  
Exp Date = 06/05/2016  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC