



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (03-2014)

Serial Number <b>80-0065-11</b>	Instrument Location <b>TRAIL CO. S.O. (Hill)</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): NONE, NEW INTOXILYZER
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <b>Ray Weber</b>	Date <b>5/13/2016</b> ✓
Reviewed By (Crime Laboratory Use Only) <b>Deb Shenaver</b>	Date <b>16 May 2016</b>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: **Charles E. Eden**    Certified Date: **09 Nov. 16**



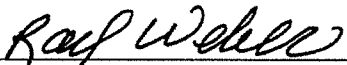
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006511  
Location = HILL      8164.13.00 06/09  
05/13/2016      01:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:15
02 Std. Gas	0.080	01:16
03 Room Air	0.000	01:17
04 Std. Gas	0.078	01:17
05 Room Air	0.000	01:18
06 Std. Gas	0.079	01:18
07 Room Air	0.000	01:19

Lot No = 19615080A5  
Cyl No = 32  
Exp Date = 09/05/2017  
County = 49      Oper No. = 058405

  
\_\_\_\_\_  
Operator Signature  
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006511  
Location = HILL      8164.13.00 06/09  
05/13/2016      01:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:21
02 Std. Gas	RFI*	01:21
03 Room Air	0.000	01:22

\*Invalid Test  
Inhibited - RFI

Lot No = 19615080A5  
Cyl No = 32  
Exp Date = 09/05/2017  
County = 49      Oper No. = 058405

  
\_\_\_\_\_  
Operator Signature  
RAY WEBER

Remarks: