



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number <i>80-006571</i>	Instrument Location <i>TRAIL CO. S.O. (Hill)</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *NONE, NEW INTOXILYZER*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Ray Weber</i>	Date <i>5/13/2016</i> ✓
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>16 May 2016</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = HILL 8164.13.00 06/09
05/13/2016 01:13

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 32
Standard Expiration? 09/05/2017
Oper No? 058405

Flow Cal. Date: 08/21/2015
Slope 688
Intercept -714727

IR Calibration Date: 08/21/2015
 3um 9um

	3um	9um
0th Coef(*100):	-19744	-15946
1st Coef(*100):	272125	142950
2nd Coef(*100):	2975	1383
H2O adj(mg/l*10k):	725	394

***** Printer Test End *****



Operator Signature
RAY WEBER

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = HILL 8164.13.00 06/09
05/13/2016 01:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:15
02 Std. Gas	0.080	01:16
03 Room Air	0.000	01:17
04 Std. Gas	0.078	01:17
05 Room Air	0.000	01:18
06 Std. Gas	0.079	01:18
07 Room Air	0.000	01:19

Lot No = 19615080A5
Cyl No = 32
Exp Date = 09/05/2017
County = 49 Oper No. = 058405



Operator Signature
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = HILL 8164.13.00 06/09
05/13/2016 01:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:21
02 Std. Gas	RFI*	01:21
03 Room Air	0.000	01:22

*Invalid Test
Inhibited - RFI

Lot No = 19615080A5
Cyl No = 32
Exp Date = 09/05/2017
County = 49 Oper No. = 058405


Operator Signature
RAY WEBER

Remarks:

Form 106-I8000