



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number <i>80-006511</i>	Instrument Location <i>Hill, TRAIL county S.D.</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *None, moved Intoxilyzer*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Ray Weiler</i>	Date <i>11-21-16</i>
--	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shanaver</i>	Certified Date <i>20 Dec 16</i>
--	------------------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = HILL 8164.14.00 09/16
11/21/2016 17:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:45
02 Std. Gas	0.082	17:45
03 Room Air	0.000	17:45
04 Std. Gas	0.082	17:46
05 Room Air	0.000	17:46
06 Std. Gas	0.082	17:47
07 Room Air	0.000	17:47

Lot No = 19615080A5
Cyl No = 32
Exp Date = 09/05/2017
County = 49 Oper No. = 058405


Operator Signature
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = HILL 8164.14.00 09/16
11/21/2016 17:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:50
02 Std. Gas	0.082	17:51
03 Room Air	0.000	17:51
04 Std. Gas	RFI*	17:52
05 Room Air	0.000	17:52

*Invalid Test
Inhibited - RFI

Lot No = 19615080A5
Cyl No = 32
Exp Date = 09/05/2017
County = 49 Oper No. = 058405



Operator Signature
RAY WEBER

Remarks:

Form 106-I8000