



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) Travis Collins		
Location HETT	Intoxilyzer® Serial Number 80006510	
Gas Lot Number 09216080A3	Gas Cylinder Number 040	Gas Expiration Date 5/5/2018

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0. 082 AC
  - B. 0. 082 AC
  - C. 0. 082 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>T Collins</i>	Date <i>12/07/2016</i>
Reviewed By (Crime Laboratory Use Only) <i>Debs Shanaver</i>	Date <i>20 Dec 16</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006510  
Location = HETT                    8164.14.00 09/16  
12/07/2016                            16:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:19
02 Std. Gas	0.082	16:19
03 Room Air	0.000	16:20
04 Std. Gas	0.082	16:20
05 Room Air	0.000	16:21
06 Std. Gas	0.082	16:21
07 Room Air	0.000	16:22

Lot No = 09216080A3  
Cyl No = 40  
Exp Date = 05/05/2018  
County = 01                            Oper No. = 020533



Operator Signature  
TRAVIS COLLINS

Remarks:

Form 106-I8000