

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006680

A. GENERAL Setup and Checks:

- Power On
 - 5 min. countdown during initial warm-up
 - 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
 - Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):
 - Operator ID ND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-006680 DSP Ver: 1362.14
 - Core Ver: 8164.14 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

- DVM Monitor (Level 3,D,D): 3: 12854 s: 1 c: 47.00
9: 13437 s: 0 B: 44 0
(RFI indicator with keyed radio): L
2. Function (F) Keys: : F5 – Tone F9 - Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 949 CF: 1.06
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 238 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 48.5 I: 0.0
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 862 psi P (gauge): ~875 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: MP3064 Lot #: MILLIQ H₂O Exp date: N/A
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: MP3003 Lot #: 201603H Exp date: 3.16.2018
Known value: 0.010 AC
Results: 0.010 AC 0.010 AC 0.010 AC
3. High AC (Level 1,C):
Sim ser #: MP3063 Lot #: 16102 Exp date: 3.22.2018
Known value: 0.400 AC
Results: 0.390 AC 0.391 AC 0.391 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR5135 Lot #: ICS4 Exp date: 9.9.2017
Known value: 0.10 AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR5143 Lot #: ACE2 Exp date: 9.29.2017
Known value: 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR3377 Lot #: MET2 Exp date: 9.29.2017
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR3453 Lot #: IPA2 Exp date: 9.29.17
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0830 Std Dev: 0.0004 Rel Std Dev: 0.48%

Maintenance/Repairs/Notes: _____

Instrument is acceptable to be used in the field.

Charles E. E...
Field Inspector's Signature

1/23/2017
Date

Deb Shanaver
Reviewed by

24 Jan 17
Date

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-00668C
Location = TOXL 8164.14.00 09/16
01/23/2017 12:03

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMN

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMN

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMN

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMN

Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.081
Standard Lot #? 81
Standard Cyl #? 123456
Standard Expiration? 11/11/2016
Oper No? 666666

Flow Cal. Date: 09/06/2016
Slope 681
Intercept -620591

IR Calibration Date: 09/06/2016

	3um	9um
0th Coef(*100):	-14265	-21271
1st Coef(*100):	260743	129687
2nd Coef(*100):	3906	1392
H2O adj(mg/l)*10K):	742	684

***** Test End *****



Operator Signature
CHARLES EDER

Remarks: *Internal*
PRINT TEST

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:09

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.081
Standard Lot #? 81
Standard Cyl #? 123456
Standard Expiration? 11/11/2016
Oper No? 666666

Flow Cal. Date: 09/06/2016
Slope 681
Intercept -620591

IR Calibration Date: 09/06/2016

	3um	9um
0th Coef(*100):	-14265	-21271
1st Coef(*100):	260743	129687
2nd Coef(*100):	3906	1392
H2O adj(mg/l*10k):	742	684

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: EXTERNAL PRINT
 TEST


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:07

Test	AC	Time
01 Room Air	0.000	12:08
02 *Subject Test	REF*	12:09
03 Room Air	0.000	12:09

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = R KEY REFUSAL
Dr. Lic. = ND/DIS821456
Lot No = 81
Cyl No = 123456
Expiration Date = 11/11/2016
County = 08 Oper No. = 666666

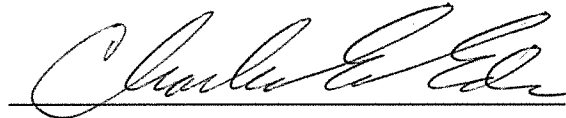

Operator Signature
CHARLES EDER

Remarks: "R" KEY REFUSAL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:10

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *DIAGNOSTIC*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:17
02 Std. Sol.	0.000	12:18
03 Room Air	0.000	12:18
04 Std. Sol.	0.000	12:19
05 Room Air	0.000	12:19
06 Std. Sol.	0.000	12:20
07 Room Air	0.000	12:20

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: ZERO AC
 0.000 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:23
02 Std. Sol.	0.010	12:24
03 Room Air	0.000	12:24
04 Std. Sol.	0.010	12:25
05 Room Air	0.000	12:26
06 Std. Sol.	0.010	12:26
07 Room Air	0.000	12:27

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 201603H
County = 08 Oper No. = 666666

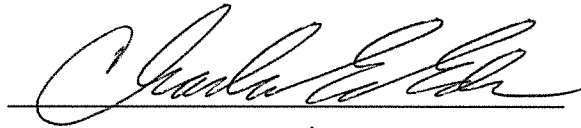

Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.010 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:36

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:36
02 Std. Sol.	0.390	12:37
03 Room Air	0.000	12:38
04 Std. Sol.	0.391	12:38
05 Room Air	0.000	12:39
06 Std. Sol.	0.391	12:40
07 Room Air	0.000	12:40

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666

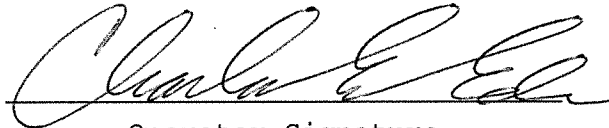

Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.400 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:42

Test	AC	Time
01 Room Air	0.000	12:43
02 *Subject Test	INT*	12:43
03 Room Air	0.000	12:44

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.10AC+0.05ACE
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:44

Test	AC	Time
01 Room Air	0.000	12:45
02 *Subject Test	INT*	12:45
03 Room Air	0.000	12:46

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.05 ACETONE
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:50

Test	AC	Time
01 Room Air	0.000	12:51
02 *Subject Test	INT*	12:52
03 Room Air	0.000	12:52

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.10 METHANOL
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10% METHANOL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:53

Test	AC	Time
01 Room Air	0.000	12:54
02 *Subject Test	INT*	12:54
03 Room Air	0.000	12:55

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.10 IPA
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: 0.10% ISOPROPANOL



Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 12:56

Test	AC	Time
01 Room Air	AMB*	12:57
02 Room Air	0.000	12:58

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

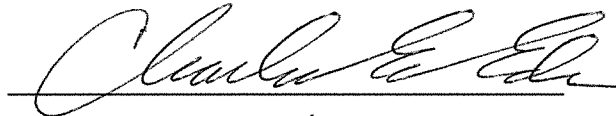

Operator Signature
CHARLES EDER

Remarks: *AMBIENT FAIL*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:00

Test	AC	Time
01 Room Air	0.000	13:01
02 Invalid Sample	X.XXX	13:01
03 Room Air	0.000	13:02

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INVALID SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INVALID SAMPLE*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:02

Test	AC	Time
01 Room Air	0.000	13:03
02 *Subject Test	ABT*	13:03
03 Room Air	0.000	13:04

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = SEQUENCE ABORT
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

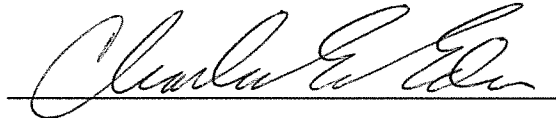

Operator Signature
CHARLES EDER

Remarks:

SEQUENCE ABORTED

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:05

Test	AC	Time
01 Room Air	0.000	13:05
02 Subject Test 1	0.026	13:06
03 Room Air	0.000	13:06
04 Reported AC	0.026	13:06

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = ABA MODE
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: ABA MODE
SINGLE BREATH TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:06

Test	AC	Time
01 Diagnostic	OK	13:07
02 Room Air	0.000	13:08
03 Subject Test 1	0.256	13:08
04 Room Air	0.000	13:11
05 Std. Gas	0.092*	13:12
06 Room Air	0.000	13:12

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CHK OUT OF TOL
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

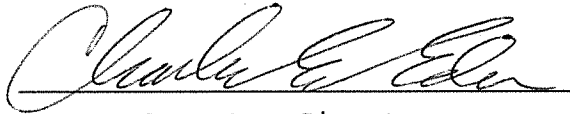

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:15

Test	AC	Time
01 Diagnostic	OK	13:15
02 Room Air	0.000	13:16
03 Subject Test 1	0.252	13:16
04 Room Air	0.000	13:19
05 Std. Gas	0.083	13:20
06 Room Air	0.000	13:21
07 *Subject Test	RFI*	13:21
08 Room Air	0.000	13:22

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI DETECTED
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI DETECTED*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:22

Test	AC	Time
01 Diagnostic	OK	13:23
02 Room Air	0.000	13:24
03 *Subject Test	0.000*	13:27
04 Room Air	0.000	13:32
05 Std. Gas	0.083	13:33
06 Room Air	0.000	13:34
07 *Subject Test	0.000*	13:37
08 Room Air	0.000	13:38
No RFI Detected		

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DEFICIENT SAMP
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DEFICIENT
SAMPLES

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:39

Test	AC	Time
01 Diagnostic	OK	13:39
02 Room Air	IPS*	13:40
03 Room Air	0.000	13:40

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = IMPROPER SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

*IMPROPER
SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:40

Test	AC	Time
01 Diagnostic	OK	13:41
02 Room Air	0.000	13:42
03 Subject Test 1	0.252*	13:42
04 Room Air	0.000	13:44
05 Std. Gas	0.084	13:45
06 Room Air	0.000	13:47
07 Subject Test 2	0.202*	13:47
08 Room Air	0.000	13:48

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DIFFERENCE
TOO * GREAT
CEE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:49


Test	AC	Time
01 Diagnostic	OK	13:49
02 Room Air	0.000	13:50
03 Subject Test 1	0.249	13:51
04 Room Air	0.000	13:53
05 Std. Gas	0.083	13:54
06 Room Air	0.000	13:55
07 Subject Test 2	0.248	13:56
08 Room Air	0.000	13:57
09 Reported AC	0.248	13:56

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CMS MODE
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 38
Expiration Date = 05/05/2018
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

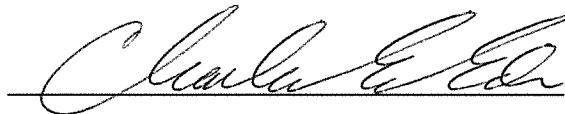

Operator Signature
CHARLES EDER

Remarks:

CMS MODE
EVIDENTIAL BREATH TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
01/23/2017 13:58

Test	AC	Time
01 Room Air	0.000	13:59
02 Std. Gas	0.082	13:59
03 Room Air	0.000	14:00
04 Std. Gas	0.082	14:00
05 Room Air	0.000	14:01
06 Std. Gas	0.083	14:01
07 Room Air	0.000	14:02
08 Std. Gas	0.083	14:02
09 Room Air	0.000	14:02
10 Std. Gas	0.083	14:03
11 Room Air	0.000	14:03
12 Std. Gas	0.083	14:04
13 Room Air	0.000	14:04
14 Std. Gas	0.083	14:05
15 Room Air	0.000	14:05
16 Std. Gas	0.083	14:05
17 Room Air	0.000	14:06
18 Std. Gas	0.083	14:06
19 Room Air	0.000	14:07
20 Std. Gas	0.083	14:07
21 Room Air	0.000	14:08
22 Std. Gas	0.083	14:08
23 Room Air	0.000	14:09
24 Std. Gas	0.083	14:09
25 Room Air	0.000	14:09
26 Std. Gas	0.084	14:10
27 Room Air	0.000	14:10
28 Std. Gas	0.083	14:11
29 Room Air	0.000	14:11
30 Std. Gas	0.083	14:12
31 Room Air	0.000	14:12
32 Std. Gas	0.083	14:12
33 Room Air	0.000	14:13
34 Std. Gas	0.083	14:13
35 Room Air	0.000	14:14
36 Std. Gas	0.083	14:14
37 Room Air	0.000	14:15
38 Std. Gas	0.083	14:15
39 Room Air	0.000	14:16
40 Std. Gas	0.083	14:16
41 Room Air	0.000	14:16
42 Cal Check Stats:		
43 Average	0.0830	
44 Std Dev	0.0004	
45 Rel Std Dev	0.48%	



Operator Signature
CHARLES EDER

Remarks: STABILITY TEST

Form 106-I8000

LOT: 09216080A3

CYL: 038

EXP: 5/5/2018