



5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.0 I: 0.0  
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 729 psi P (gauge): ~740 psi
7.  General Setup (Level 3,S,G)
8.  Configure Start Test (Level 3,S,T)
9.  Network (Level 3,S,N)

**E. TESTS (attach all test records):**

**Configure wet bath simulator standards before running tests (Level 1,S):**

1. Zero AC (Level 1,C):  
Sim ser #: MP3064 Lot #: MILLI Q H<sub>2</sub>O Exp date: NA  
Known value: 0.000 AC  
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):  
Sim ser #: MP3060 Lot #: 201507F Exp date: 7/28/17  
Known value: 0.020 AC  
Results: 0.020 AC 0.020 AC 0.020 AC
3. High AC (Level 1,C):  
Sim ser #: MP3063 Lot #: 16102 Exp date: 3/22/18  
Known value: 0.400 AC  
Results: 0.393 AC 0.394 AC 0.394 AC

**Configure dry gas standard before running tests (Level 1,S):**

4. Ethanol + 0.05% Acetone (Level 1,B):  
Sim ser #: DR5135 Lot #: ICS<sup>CBE</sup>#4 Exp date: 9/9/17  
Known value: 0.10AC + 0.05% ACETONE  
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):  
Sim ser #: DR5143 Lot #: ACE2 Exp date: 9/29/17  
Known value: 0.05% ACETONE  
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):  
Sim ser #: DR3377 Lot #: MET2 Exp date: 9/29/17  
Known value: 0.10% METHANOL  
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):  
Sim ser #: DR3453 Lot #: IPA2 Exp date: 9/29/17  
Known value: 0.10% ISOPROPANOL  
 Display reads "Interferent Detect"

- 8.  Ambient Fail (Level 1,B): Display and test record read correct.
- 9.  Invalid Sample (Level 1,B): Display and test record read correct.
- 10.  Sequence Aborted (Level 1,B): Display and test record read correct.
- 11.  ABA (Level 1,B): Test record reads correct.
- 12.  Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13.  RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15.  Deficient Samples (CMS mode): Display and test record read correct.
- 16.  Improper Sample (CMS mode): Display and test record read correct.
- 17.  Difference Too Great (CMS mode): Display and test record read correct.
- 18.  CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0816 Std Dev: 0.0005 Rel Std Dev: 0.62%

Maintenance/Repairs/Notes: \_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

Instrument is acceptable to be used in the field.

Charles E. Edin  
Field Inspector's Signature

3/22/2017  
Date

Deb Shanaver  
Reviewed by

24 Mar 17  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006687  
Location = TOXL      8164.14.00 09/16  
03/21/2017      16:37

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN

24 characters per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN

32 characters per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN

42 characters per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN


Current Instrument Setup  
Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On MemFull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 17316080A2  
Standard Cyl #? 26  
Standard Expiration? 08/05/2018  
Oper No? 666666

Flow Cal. Date: 09/06/2016  
Slope: 697  
Intercept: -689849

IR Calibration Date: 09/06/2016  
3um      9um

0th Coef(\*100): -14968      -21526  
1st Coef(\*100): 263834      133395  
2nd Coef(\*100): 2721      1501  
H2O adj(mg/l\*10K): 582      571

\*\*\*\*\* Test End \*\*\*\*\*

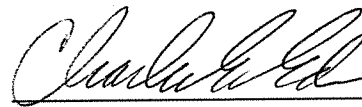

Operator Signature  
CHARLES EDER

Remarks: *Internal*  
*Print Test*  
Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-006687  
Location = TOXL                8164.14.00 09/16  
03/21/2017                      16:38

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:               Enabled  
Start Test Sequence:           DABACABA  
Display Prelim Rslt?           Yes  
Display Third Digit?           Yes  
Inhib Printer(Y/N)?            No  
Display Volume?                No  
Disable On Memfull?            Yes  
# of Print Copies?              1  
Select Std (D/W/I)?            Dry  
Standard Value?                0.080  
Standard Lot #?                 17316080A2  
Standard Cyl #?                 26  
Standard Expiration?           08/05/2018  
Oper No?                        666666

Flow Cal. Date:                 09/06/2016  
Slope                            697  
Intercept                       -689849

IR Calibration Date:            09/06/2016

	3um	9um
0th Coef(*100):	-14968	-21526
1st Coef(*100):	263834	133395
2nd Coef(*100):	2721	1501
H2O adj(mg/l*10k):	582	571

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

EXTERNAL  
PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006687  
Location = TOXL      8164.14.00 09/16  
03/21/2017      16:39

Test	AC	Time
01 Room Air	0.000	16:40
02 *Subject Test	REF*	16:40
03 Room Air	0.000	16:41

\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = R KEY REFUSAL  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

R KEY  
REFUSAL

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006687  
Location = TOXL      8164.14.00 09/16  
03/21/2017      16:41

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666

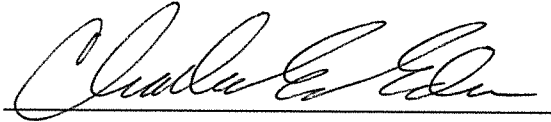

Operator Signature  
CHARLES EDER

Remarks:

DIAGNOSTIC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                 8164.14.00 09/16  
03/22/2017                        10:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:26
02 Std. Sol.	0.000	10:27
03 Room Air	0.000	10:28
04 Std. Sol.	0.000	10:28
05 Room Air	0.000	10:29
06 Std. Sol.	0.000	10:29
07 Room Air	0.000	10:30

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = MILLIQ H2O  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:    0.000 AC  
                 ZERO AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            10:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:33
02 Std. Sol.	0.020	10:34
03 Room Air	0.000	10:34
04 Std. Sol.	0.020	10:35
05 Room Air	0.000	10:36
06 Std. Sol.	0.020	10:36
07 Room Air	0.000	10:37

08 Sim Temp = 34.0°C

Simul Ser No = MP3060  
Std Sol No = 201507F  
County = 08                            Oper No. = 666666

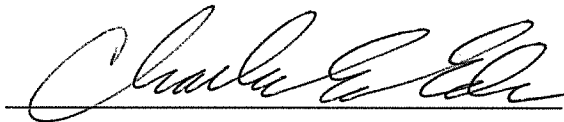

Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                 8164.14.00 09/16  
03/22/2017                        10:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:45
02 Std. Sol.	0.393	10:46
03 Room Air	0.000	10:47
04 Std. Sol.	0.394	10:47
05 Room Air	0.000	10:48
06 Std. Sol.	0.394	10:49
07 Room Air	0.000	10:49

08 Sim Temp = 34.0°C

Simul Ser No = MP3063  
Std Sol No = 16102  
County = 08                            Oper No. = 666666

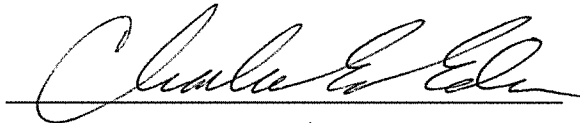

Operator Signature  
CHARLES EDER

Remarks:    *HIGH AC*  
                  *0.400 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            10:51

Test	AC	Time
01 Room Air	0.000	10:52
02 *Subject Test	INT*	10:53
03 Room Air	0.000	10:53

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent*  
*0.10AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            10:58

Test	AC	Time
01 Room Air	0.000	10:58
02 *Subject Test	INT*	10:59
03 Room Air	0.000	10:59

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*

*0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            11:00

Test	AC	Time
01 Room Air	0.000	11:00
02 *Subject Test	INT*	11:01
03 Room Air	0.000	11:02

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent*  
*0.10% METHANOL*  
Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                 8164.14.00 09/16  
03/22/2017                        11:02

Test	AC	Time
01 Room Air	0.000	11:02
02 *Subject Test	INT*	11:03
03 Room Air	0.000	11:04

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                 Weight = 150  
Test = DUI                         Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                         Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*  
*0.10% ISOPROPNOL*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            11:05

Test	AC	Time
01 Room Air	AMB*	11:06
02 Room Air	0.000	11:07

\*Invalid Test  
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = AMBIENT FAIL  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

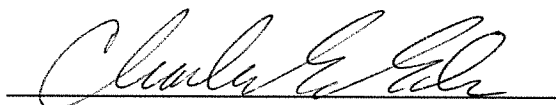

Operator Signature  
CHARLES EDER

Remarks: *AMBIENT FAIL*








Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                8164.14.00 09/16  
03/22/2017                        11:15

Test	AC	Time
01 Room Air	0.000	11:16
02 Subject Test 1	0.010	11:16
03 Room Air	0.000	11:16
04 Reported AC	0.010	11:16

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = ABA TEST MODE  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

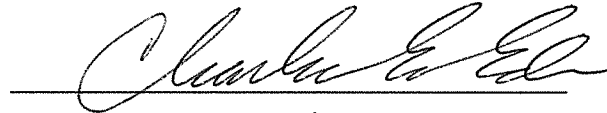

Operator Signature  
CHARLES EDER

Remarks:

ABA MODE







Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            12:57

Test	AC	Time
01 Diagnostic	OK	12:58
02 Room Air	0.000	12:58
03 Subject Test 1	0.145	12:59
04 Room Air	0.000	13:01
05 Std. Gas	0.082	13:02
06 Room Air	0.000	13:03
07 Reference	IPS*	13:03
08 Room Air	0.000	13:04

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = IMPROPER SAMP  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*Improper  
Sample*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = TOXL                 8164.14.00 09/16  
03/22/2017                        13:05

Test	AC	Time
01 Diagnostic	OK	13:06
02 Room Air	0.000	13:06
03 Subject Test 1	0.145*	13:07
04 Room Air	0.000	13:10
05 Std. Gas	0.081	13:11
06 Room Air	0.000	13:12
07 Subject Test 2	0.100*	13:13
08 Room Air	0.000	13:14

\*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = DIFF TOO GREAT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: DIFFERENCE TOO  
                  GREAT

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            15:09


Test	AC	Time
01 Diagnostic	OK	15:10
02 Room Air	0.000	15:11
03 Subject Test 1	0.000	15:11
04 Room Air	0.000	15:13
05 Std. Gas	0.082	15:14
06 Room Air	0.000	15:16
07 Subject Test 2	0.000	15:16
08 Room Air	0.000	15:18
09 Reported AC	0.000	15:11

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = CMS MODE  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*CMS MODE*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006687  
Location = TOXL                    8164.14.00 09/16  
03/22/2017                            13:16

Test	AC	Time
01 Room Air	0.000	13:16
02 Std. Gas	0.081	13:17
03 Room Air	0.000	13:17
04 Std. Gas	0.082	13:18
05 Room Air	0.000	13:18
06 Std. Gas	0.081	13:19
07 Room Air	0.000	13:19
08 Std. Gas	0.082	13:19
09 Room Air	0.000	13:20
10 Std. Gas	0.082	13:20
11 Room Air	0.000	13:21
12 Std. Gas	0.082	13:21
13 Room Air	0.000	13:22
14 Std. Gas	0.082	13:22
15 Room Air	0.000	13:23
16 Std. Gas	0.081	13:23
17 Room Air	0.000	13:23
18 Std. Gas	0.082	13:24
19 Room Air	0.000	13:24
20 Std. Gas	0.081	13:25
21 Room Air	0.000	13:25
22 Std. Gas	0.081	13:25
23 Room Air	0.000	13:26
24 Std. Gas	0.082	13:26
25 Room Air	0.000	13:27
26 Std. Gas	0.081	13:27
27 Room Air	0.000	13:28
28 Std. Gas	0.082	13:28
29 Room Air	0.000	13:29
30 Std. Gas	0.081	13:29
31 Room Air	0.000	13:29
32 Std. Gas	0.082	13:30
33 Room Air	0.000	13:30
34 Std. Gas	0.082	13:31
35 Room Air	0.000	13:31
36 Std. Gas	0.081	13:32
37 Room Air	0.000	13:32
38 Std. Gas	0.082	13:32
39 Room Air	0.000	13:33
40 Std. Gas	0.082	13:33
41 Room Air	0.000	13:34
42 Cal Check Stats:		
43 Average	0.0816	
44 Std Dev	0.0005	
45 Rel Std Dev	0.62%	



CHARLES EDER

Remarks: STABILITY TEST  
0.080 AC

Form 106-I8000

LOT: 17316 080 A2

CYL: 026

EXP: 08/05/2018