

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006668

A. GENERAL Setup and Checks:

- Power On
 5 min. countdown during initial warm-up
 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
 Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):
 Operator ID ND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
Inst Ser#: 80-006668 DSP Ver: 1362.14
Core Ver: 8164.14 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

- DVM Monitor (Level 3,D,D): 3: 12535 s: 1 c: 47.06
9: 13069 s: 1 B: 45 0
(RFI indicator with keyed radio): 1
2. Function (F) Keys: : F5 – Tone F9 - Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 957 CF: 1.05
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 201 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.5 I: 0.0
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 56 psi P (gauge): ~50 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: MP3064 Lot #: MILLI QH₂O Exp date: NA
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: MP3060 Lot #: 201501A Exp date: 01.06.17
Known value: 0.015 AC
Results: 0.016 AC 0.017 AC 0.017 AC
3. High AC (Level 1,C):
Sim ser #: MP3063 Lot #: 16102 Exp date: 03.22.18
Known value: 0.400 AC
Results: 0.398 AC 0.400 AC 0.398 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR5135 Lot #: ICS4 Exp date: 9.9.2017
Known value: 0.10 AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR5143 Lot #: ACE2 Exp date: 9/29/2017
Known value: 0.10 CEE 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR3377 Lot #: MET2 Exp date: 9/29/2017
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR3453 Lot #: IPA2 Exp date: 9/29/2017
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0824 Std Dev: 0.0007 Rel Std Dev: 0.91%

Maintenance/Repairs/Notes: INTERNAL PRINT TEST WAS RE-RUN
DUE TO TAPE ^{SEE} OBSCURE, OBSCURING SOME OF THE ORIGINAL
PRINT TEST. 1/24/17 Charles Eder

Instrument is acceptable to be used in the field.

Charles Eder
Charles Eder
Field Inspector's Signature

24 JAN 2017

05 JAN 2017

Date

Deb Shanaver
Reviewed by

13 Jan '17 DS
Date 27 Mar 17

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:04

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMNPO

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMNQRSTUWX

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMNQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef


Data entry mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 3
Standard Expiration? 09/05/2017
Oper No? 666666

Flow Cal. Date: 08/15/2016
Slope 690
Intercept -609656

IR Calibration Date: 08/15/2016
3um 9um

0th Coef(*100): -21473 -25013
1st Coef(*100): 254225 130517
2nd Coef(*100): 2814 1195
H2O adj(ng/l*10K): 694 580

***** Test End *****



Operator Signature
CHARLES EDER

Remarks: *Internal*
Print TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/24/2017 12:08

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMNQRSTUWX

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMNQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMNQRSTUWXYZ1234567890ABCDEF


Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 26
Standard Expiration? 09/05/2017
Oper No? 666666

Flow Cal. Date: 08/15/2016
Slope 690
Intercept -609656

IR Calibration Date: 08/15/2016
3um 9um

0th Coef(*100): -21473 -25013
1st Coef(*100): 254225 130517
2nd Coef(*100): 2814 1195
H2O adj(mg/l*10K): 694 580

***** Test End *****

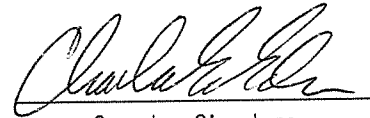

Operator Signature
CHARLES EDER

Remarks: INTERNAL
PRINT TEST

Form 106-18000

IT WAS NOTED THAT
TAPE HAD OBSCURED
SOME OF THE ORIGINAL
PRINT.

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:05

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 3
Standard Expiration? 09/05/2017
Oper No? 666666

Flow Cal. Date: 08/15/2016
Slope 690
Intercept -609656

IR Calibration Date: 08/15/2016
 3um 9um

0th Coef(*100): -21473 -25013
1st Coef(*100): 254225 130517
2nd Coef(*100): 2814 1195
H2O adj(mg/l*10k): 694 580

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

*External
Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:08

Test	AC	Time
01 Room Air	0.000	16:09
02 *Subject Test	REF*	16:09
03 Room Air	0.000	16:10

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = R KEY REFUSAL
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

"R" KEY REFUSAL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:11

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

DIAGNOSTIC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:19
02 Std. Sol.	0.000	16:20
03 Room Air	0.000	16:20
04 Std. Sol.	0.000	16:21
05 Room Air	0.000	16:22
06 Std. Sol.	0.000	16:22
07 Room Air	0.000	16:23

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666

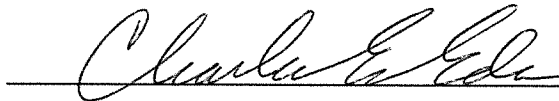

Operator Signature
CHARLES EDER

Remarks:

ZERO AC
0.000 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:24

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:25
02 Std. Sol.	0.016	16:25
03 Room Air	0.000	16:26
04 Std. Sol.	0.017	16:27
05 Room Air	0.000	16:27
06 Std. Sol.	0.017	16:28
07 Room Air	0.000	16:28

08 Sim Temp = 34.0°C

Simul Ser No = MP3060

Std Sol No = 201501A

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC

0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:31
02 Std. Sol.	0.398	16:31
03 Room Air	0.000	16:32
04 Std. Sol.	0.400	16:33
05 Room Air	0.000	16:33
06 Std. Sol.	0.398	16:34
07 Room Air	0.000	16:34

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
 0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:37

Test	AC	Time
01 Room Air	0.000	16:38
02 *Subject Test	INT*	16:38
03 Room Air	0.000	16:39

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.10AC+0.05ACE
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:40

Test	AC	Time
01 Room Air	0.000	16:41
02 *Subject Test	INT*	16:41
03 Room Air	0.000	16:42

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.05 ACETONE
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent

0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:42

Test	AC	Time
01 Room Air	0.000	16:43
02 *Subject Test	INT*	16:43
03 Room Air	0.000	16:44

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.10 METHANOL
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent METHANOL
0.10% ISOPROPANOL OEE

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:45

Test	AC	Time
01 Room Air	0.000	16:46
02 *Subject Test	INT*	16:46
03 Room Air	0.000	16:47

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = 0.10 IPA
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10% ISOPROPANOL

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:48

Test	AC	Time
01 Room Air	AMB*	16:48
02 Room Air	0.000	16:49

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

*AMBIENT
FAIL*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:50

Test	AC	Time
01 Room Air	0.000	16:50
02 Invalid Sample	X.XXX	16:51
03 Room Air	0.000	16:51

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INVALID SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Invalid Sample*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:52

Test	AC	Time
01 Room Air	0.000	16:53
02 *Subject Test	ABT*	16:53
03 Room Air	0.000	16:54

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = SEQ ABORTED
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: SEQUENCE
 ABORTED

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/04/2017 16:54

Test	AC	Time
01 Room Air	0.000	16:55
02 Subject Test 1	0.000	16:55
03 Room Air	0.000	16:55
04 Reported AC	0.000	16:55

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = ABA MODE
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: ABA MODE
 (SINGLE BREATH TEST)

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/05/2017 09:12

Test	AC	Time
01 Diagnostic	OK	09:13
02 Room Air	0.000	09:13
03 Subject Test 1	0.215	09:14
04 Room Air	0.000	09:16
05 Std. Gas	0.092*	09:17
06 Room Air	0.000	09:18

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CAL OUT TOL
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 3
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/05/2017 09:28

Test	AC	Time
01 Diagnostic	OK	09:29
02 Room Air	0.000	09:29
03 Subject Test 1	0.214	09:30
04 Room Air	0.000	09:32
05 Std. Gas	0.082	09:33
06 Room Air	0.000	09:34
07 *Subject Test	RFI*	09:34
08 Room Air	0.000	09:35

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI DETECTED
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 26
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI DETECTED*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/05/2017 10:06

Test	AC	Time
01 Diagnostic	OK	10:07
02 Room Air	0.000	10:08
03 *Subject Test	0.000*	10:11
04 Room Air	0.000	10:13
05 Std. Gas	0.082	10:14
06 Room Air	0.000	10:15
07 *Subject Test	0.000*	10:18
08 Room Air	0.000	10:19

No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DEFICIENT SAMP
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 26
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DEFICIENT
SAMPLES

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/05/2017 10:39

Test	AC	Time
01 Diagnostic	OK	10:40
02 Room Air	IPS*	10:40
03 Room Air	0.000	10:40

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = IMPROPER SAMP
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 26
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

*IMPROPER
SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/05/2017 10:41

Test	AC	Time
01 Diagnostic	OK	10:42
02 Room Air	0.000	10:42
03 Subject Test 1	0.214*	10:43
04 Room Air	0.000	10:45
05 Std. Gas	0.082	10:46
06 Room Air	0.000	10:48
07 Subject Test 2	0.278*	10:48
08 Room Air	0.000	10:49

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 26
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DIFFERENCE
 TOO GREAT

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/05/2017 11:10

Test	AC	Time
01 Diagnostic	OK	11:11
02 Room Air	0.000	11:11
03 Subject Test 1	0.277	11:12
04 Room Air	0.000	11:14
05 Std. Gas	0.082	11:15
06 Room Air	0.000	11:17
07 Subject Test 2	0.276	11:17
08 Room Air	0.000	11:18
09 Reported AC	0.276	11:17

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CMS MODE
Dr. Lic. = ND/DIS821456
Lot No = 19615080A5
Cyl No = 26
Expiration Date = 09/05/2017
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

CMS MODE
(EVIDENTIAL TEST)

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006668
Location = TOXL 8164.14.00 09/16
01/05/2017 11:20

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Gas	0.080	11:21
03 Room Air	0.000	11:21
04 Std. Gas	0.082	11:22
05 Room Air	0.000	11:22
06 Std. Gas	0.082	11:23
07 Room Air	0.000	11:23
08 Std. Gas	0.082	11:24
09 Room Air	0.000	11:24
10 Std. Gas	0.082	11:24
11 Room Air	0.000	11:25
12 Std. Gas	0.082	11:25
13 Room Air	0.000	11:26
14 Std. Gas	0.082	11:26
15 Room Air	0.000	11:27
16 Std. Gas	0.082	11:27
17 Room Air	0.000	11:28
18 Std. Gas	0.082	11:28
19 Room Air	0.000	11:28
20 Std. Gas	0.083	11:29
21 Room Air	0.000	11:29
22 Std. Gas	0.083	11:30
23 Room Air	0.000	11:30
24 Std. Gas	0.083	11:30
25 Room Air	0.000	11:31
26 Std. Gas	0.082	11:31
27 Room Air	0.000	11:32
28 Std. Gas	0.082	11:32
29 Room Air	0.000	11:33
30 Std. Gas	0.083	11:33
31 Room Air	0.000	11:34
32 Std. Gas	0.083	11:34
33 Room Air	0.000	11:34
34 Std. Gas	0.083	11:35
35 Room Air	0.000	11:35
36 Std. Gas	0.083	11:36
37 Room Air	0.000	11:36
38 Std. Gas	0.083	11:37
39 Room Air	0.000	11:37
40 Std. Gas	0.083	11:37
41 Room Air	0.000	11:38
42 Cal Check Stats:		
43 Average	0.0824	
44 Std Dev	0.0007	
45 Rel Std Dev	0.91%	



Operator Signature
CHARLES EDER

Remarks: STABILITY TEST

Form 106-I8000

Lot#: 19615080A5
Exp: 9/5/2017
Cyl: 026