

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006679

Location: TOXL

Check When Done

- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3060 Lot No. 201507F Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. ICS4 Solution 0.10 AC + 0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 026 Exp. Date 08/05/2018
Test 1 0.081 Test 4 0.080 Test 7 0.080
Test 2 0.081 Test 5 0.081 Test 8 0.080
Test 3 0.080 Test 6 0.081 Test 9 0.080
Average 0.080

J. Tank pressure, Level 3 Func D Sub G P: 563 psi Reg 550 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4.4.2017
Date

04 Apr 17
Date

Charles E. Edr
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006679
Location = TOXL 8164.14.00 09/16
04/04/2017 12:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:44
02 Std. Sol.	0.017	12:45
03 Room Air	0.000	12:45
04 Std. Sol.	0.018	12:46
05 Room Air	0.000	12:46
06 Std. Sol.	0.018	12:47
07 Room Air	0.000	12:48

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 201507F
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006679
Location = TOXL 8164.14.00 09/16
04/04/2017 12:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:50
02 Std. Sol.	0.391	12:51
03 Room Air	0.000	12:51
04 Std. Sol.	0.393	12:52
05 Room Air	0.000	12:52
06 Std. Sol.	0.392	12:53
07 Room Air	0.000	12:54

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: HIGH AC
 0.400 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006679
Location = TOXL 8164.14.00 09/16
04/04/2017 12:56

Test	AC	Time
01 Room Air	0.000	12:57
02 *Subject Test	INT*	12:57
03 Room Air	0.000	12:58

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

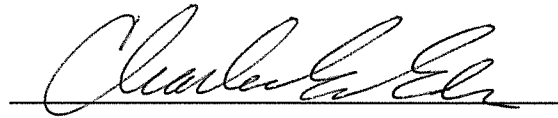

Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.10AC + 0.05% ACETONE
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006679
Location = TOXL 8164.14.00 09/16
04/04/2017 13:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:02
02 Std. Gas	0.081	13:02
03 Room Air	0.000	13:02
04 Std. Gas	0.081	13:03
05 Room Air	0.000	13:03
06 Std. Gas	0.080	13:04
07 Room Air	0.000	13:04

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006679
Location = TOXL 8164.14.00 09/16
04/04/2017 13:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:05
02 Std. Gas	0.080	13:06
03 Room Air	0.000	13:06
04 Std. Gas	0.081	13:07
05 Room Air	0.000	13:07
06 Std. Gas	0.081	13:08
07 Room Air	0.000	13:08

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006679
Location = TOXL 8164.14.00 09/16
04/04/2017 13:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:09
02 Std. Gas	0.080	13:09
03 Room Air	0.000	13:10
04 Std. Gas	0.080	13:10
05 Room Air	0.000	13:11
06 Std. Gas	0.080	13:11
07 Room Air	0.000	13:12

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC