

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006691

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN MP3060 Lot No. 201507F Rep AC 0.020 ✓
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5135 Lot No. IC54 Solution 0.10AC + 0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 026 Exp. Date 8/5/2018

Test 1 0.081 Test 4 0.082 Test 7 0.081

Test 2 0.080 Test 5 0.082 Test 8 0.081

Test 3 0.081 Test 6 0.081 Test 9 0.082

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 540 psi Reg ~550 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4.4.2017  
Date

04 Apr 17  
Date

Charles E. Ed  
Field Inspector's Signature

D. b Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006691  
Location = TOXL                    8164.14.00 09/16  
04/04/2017                                    13:50

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            17316080A2  
Standard Cyl #?            26  
Standard Expiration?        08/05/2018  
Oper No?                    666666

Flow Cal. Date:            09/07/2016  
Slope                        715  
Intercept                   -719337

IR Calibration Date:        09/07/2016  
  3um                    9um

	3um	9um
0th Coef(*100):	-11169	-23079
1st Coef(*100):	265834	136373
2nd Coef(*100):	1816	1210
H2O adj(mg/l*10k):	497	550

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
04/04/2017      13:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:52
02 Std. Sol.	0.016	13:53
03 Room Air	0.000	13:54
04 Std. Sol.	0.017	13:54
05 Room Air	0.000	13:55
06 Std. Sol.	0.017	13:56
07 Room Air	0.000	13:56

08 Sim Temp = 34.0°C

Simul Ser No = MP3060  
Std Sol No = 201507F  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
04/04/2017      14:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:09
02 Std. Sol.	0.397	14:10
03 Room Air	0.000	14:11
04 Std. Sol.	0.397	14:11
05 Room Air	0.000	14:12
06 Std. Sol.	0.397	14:13
07 Room Air	0.000	14:13

08 Sim Temp = 34.0°C

Simul Ser No = MP3063  
Std Sol No = 16102  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: HIGH AC  
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
04/04/2017      14:15

Test	AC	Time
01 Room Air	0.000	14:16
02 *Subject Test	INT*	14:17
03 Room Air	0.000	14:17

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*  
*0.10% AC + 0.05% ACETONE*  
*CEE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
04/04/2017      14:18

Test	AC	Time
01 Diagnostic	OK	14:18
02 Room Air	0.000	14:19
03 *Subject Test	RFI*	14:19
04 Room Air	0.000	14:20

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
04/04/2017      14:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:24
02 Std. Gas	0.081	14:25
03 Room Air	0.000	14:25
04 Std. Gas	0.080	14:26
05 Room Air	0.000	14:26
06 Std. Gas	0.081	14:26
07 Room Air	0.000	14:27

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666

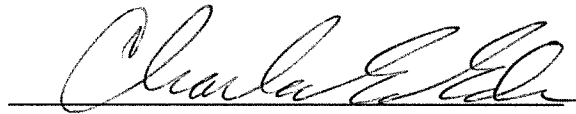

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
04/04/2017      14:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:28
02 Std. Gas	0.082	14:28
03 Room Air	0.000	14:29
04 Std. Gas	0.082	14:29
05 Room Air	0.000	14:30
06 Std. Gas	0.081	14:30
07 Room Air	0.000	14:31

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666

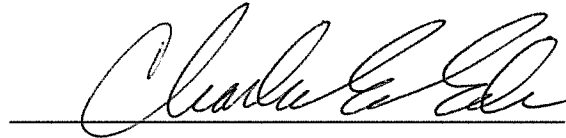

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
04/04/2017      14:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Gas	0.081	14:32
03 Room Air	0.000	14:32
04 Std. Gas	0.081	14:33
05 Room Air	0.000	14:33
06 Std. Gas	0.082	14:34
07 Room Air	0.000	14:34

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666

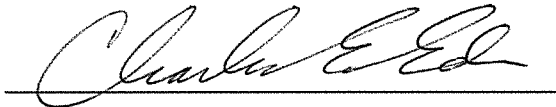

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000