

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006670

Location: TOXL

- Check When Done
- A. Is the breath tube heated? X
 - B. Is the diagnostic check complete? X
 - C. Is the time, date, and year correct (re-set if necessary)? X
 - D. Print test. (Attach test record.) X
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.030 X
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 16102 Rep AC 0.400 X
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. 1CS4 Solution 0.1002% AC
and 0.05 ACE X
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) X
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 cyl No. 26 Exp. Date 5 Aug 18

Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>

Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 312 psi Reg 325 psi

K. Remarks/Maintenance record: Annual Inspection.

6 Apr 17
Date

11 Apr 17
Date

Robert J. Meyer-Kimms
Field Inspector's Signature

Dub Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 14:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 26
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 08/15/2016
Slope 691
Intercept -681667

IR Calibration Date: 08/15/2016
 3um 9um

0th Coef(*100): -36090 -26909
1st Coef(*100): 270161 131597
2nd Coef(*100): 3492 1526
H2O adj (mg/l*10k): 790 617

***** Printer Test End *****

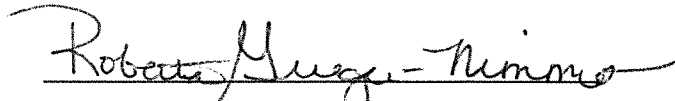

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 15:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Sol.	0.018	15:03
03 Room Air	0.000	15:03
04 Std. Sol.	0.018	15:04
05 Room Air	0.000	15:04
06 Std. Sol.	0.018	15:05
07 Room Air	0.000	15:06

08 Sim Temp = 34.0°C

Simul Ser No = MP3003

Std Sol No = 201610A

County = 08

Oper No. = 888888

Robert Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

Low AC
0.020

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 15:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:09
02 Std. Sol.	0.392	15:10
03 Room Air	0.000	15:11
04 Std. Sol.	0.395	15:11
05 Room Air	0.000	15:12
06 Std. Sol.	0.396	15:13
07 Room Air	0.000	15:13

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 888888

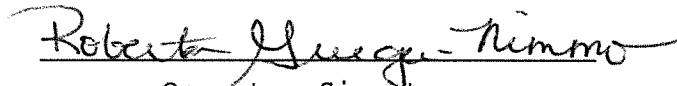
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.400


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 15:17

Test	AC	Time
01 Room Air	0.000	15:18
02 *Subject Test	INT*	15:18
03 Room Air	0.000	15:19

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

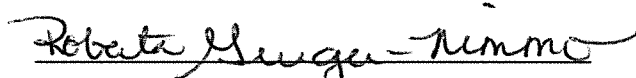

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.100 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 15:24

Test	AC	Time
01 Diagnostic	OK	15:25
02 Room Air	RFI*	15:26
03 Room Air	0.000	15:26

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

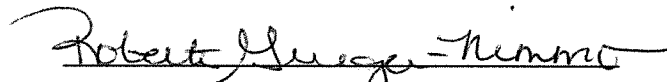

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 15:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:29
02 Std. Gas	0.082	15:30
03 Room Air	0.000	15:30
04 Std. Gas	0.082	15:30
05 Room Air	0.000	15:31
06 Std. Gas	0.082	15:31
07 Room Air	0.000	15:32

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

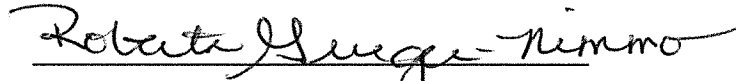
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 15:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:33
02 Std. Gas	0.082	15:33
03 Room Air	0.000	15:34
04 Std. Gas	0.082	15:34
05 Room Air	0.000	15:35
06 Std. Gas	0.082	15:35
07 Room Air	0.000	15:35

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = TOXL 8164.14.00 09/16
04/06/2017 15:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:36
02 Std. Gas	0.082	15:37
03 Room Air	0.000	15:37
04 Std. Gas	0.082	15:37
05 Room Air	0.000	15:38
06 Std. Gas	0.082	15:38
07 Room Air	0.000	15:39

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

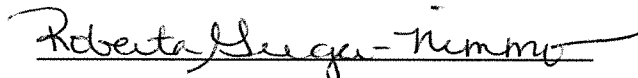

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080

Form 106-I8000